

THERAPEUTIC SHOES  
**POST-OP**  
MAY 2016







## Dear Customers,

We are happy to present you our therapeutic shoes in a completely new light.

The catalogue at hand contains all FIOR & GENTZ therapeutic shoes especially designed for treatment after an operation or injury.

Besides the classification into bandage and relief shoes, the table of contents gives an overview of the different shaft and sole designs. All the shoes' details can be found on each shoe's particular product page. Matching insoles are also described on these pages.

You can find information on shoe sizes and measurement methods from page 50 and matching accessory parts, such as protective caps, from page 52 on.

Our goal is to provide you with products enabling you to treat your patients the best possible way. You will find further therapeutic shoes in our **product catalogues DIABETES** and **HOMECARE**.

With best regards from Lüneburg  
Jörg Fior and Ralf Gentz

All our products meet our high standard of quality. This quality standard has been controlled and certified by an independent certification organisation according to the international standards ISO 9001 and ISO 13485.





## SALES TEAM IN-HOUSE

In order to provide you with up to date contact details of our distributors in foreign countries, we have listed all our current distributors online at:  
[www.fior-gentz.com/contact/export](http://www.fior-gentz.com/contact/export)

**Ana María von Corvin**  
English/Spanish



**Michael Krüger**  
English



**Anna-Lena Kühn**  
English/French



## SALES TEAM BENELUX

**Luc Schols**  
Dutch/English/French



### IN-HOUSE SALES TEAM

For any questions, orders or customer data changes just call +49 4131 24445-0. We are available on workdays from 8.00am to 5.00pm (Central European Time).

If you place your order by 4.15pm, we will dispatch the goods on that same day. Express deliveries on Saturday are also possible.



# TABLE OF CONTENTS

## INTRODUCTION

WELCOME.....	2-3
SERVICE AND SUPPORT.....	4-5
SHOE CHARACTERISTICS.....	8-9

## SHOES FOR BANDAGES

### ASYMMETRIC SHOES WITH TALL SHAFT

Rigid, extra strong rocker bottom sole with pitch, 30mm thick

FLENSBURG.....	10-11
ERFURT.....	12-13
AUGSBURG.....	14-15

### SYMMETRIC SHOES WITH LOW SHAFT

Rigid, extra strong rocker bottom sole with pitch, 30mm thick

BREMEN.....	16
TOKIO.....	17
STUTTGART.....	18
MADRID.....	19

### SYMMETRIC SHOES WITH TALL SHAFT

Rigid, extra strong rocker bottom sole with an even top side, 30mm thick

AMSTERDAM.....	20
FREIBURG.....	21
OSLO.....	22
HAMBURG.....	23
REGENSBURG.....	24
ROSTOCK.....	25

## FOREFOOT RELIEF SHOES

### SYMMETRIC SHOES WITH TALL SHAFT

Rigid, extra strong rocker bottom sole with an even top side, 40mm thick

KOPENHAGEN.....	26
FRANKFURT.....	27
HELSINKI.....	28
LÜBECK.....	29

## SYMMETRIC SHOES WITH TALL SHAFT

8° wedge sole (60:40) with forefoot rocker, 30mm thick

LONDON.....	30
MAINZ.....	31

8° wedge sole (60:40) with forefoot support, 30mm thick

HANNOVER.....	32
MAILAND.....	33
KÖLN.....	34
STOCKHOLM.....	35

10° wedge sole (50:50) with forefoot support, 30mm thick

STENDAL.....	36-37
--------------	-------

10° wedge sole (50:50) with forefoot support, 40mm thick

LÜNEBURG.....	38-39
---------------	-------

10° wedge sole (50:50) without forefoot support, 40mm thick

BERLIN.....	40-41
BORDEAUX.....	42-43

10° wedge sole (30:70) without forefoot support, 30mm thick

KAISERSLAUTERN.....	44-45
---------------------	-------

## HEEL RELIEF SHOES

### SYMMETRIC SHOES WITH TALL SHAFT

5° wedge sole (20:80) without heel support, 30mm thick

MÜNCHEN.....	46
MARSEILLE.....	47
WUPPERTAL.....	48-49

### ADDITIONAL INFORMATION

INNER SHOE DIMENSIONS.....	50-51
SPARE AND ACCESSORY PARTS.....	52-58
MORE SHOE CATALOGUES.....	59







## SOLE

The rigid soles are made of solid EVA material. This construction prevents flexion of the metatarsophalangeal joints and ensures the patient's foot being stabilised and immobilised. That is how an existent wound is relieved and the wound's healing supported in an optimal way. The soles can be adapted to the patient's needs by grinding and/or adhering material.



## WASHABLE

All bandage and relief shoes can be washed at 40°C.



## REINFORCED HEEL CAP

All bandage and forefoot relief shoes with closed heel area have a reinforced heel cap which provides support, cushions the heel and ensures comfortable stability.



## INSOLES

The insoles made of PA or EVA material are placed or attached into the correspondent shoes. Due to reworkable insoles, the inner volume and the hardness of the cushioning can be adapted to individual needs for space and relief. The insoles can be replaced by individual foot orthotics, if applicable.

## BREATHABLE

The upper fabric of all shoes is breathable.

## ADAPTABLE INNER VOLUME

Due to the generous hook and loop or instep fasteners, the inner volume can be adapted to the patient's individual need for space.





## PRODUCT INFORMATION

### INDICATIONS

FLENSBURG is particularly suited for patients with wounds at the foot/forefoot area on the sole of the foot (e.g. caused by traumatic injuries), swellings at the foot area (e.g. caused by inflammations) or joint pain in the foot area (e.g. caused by rheumatism or arthrosis) who need a closed shoe with an individually adaptable inner volume and a tall shaft.

### SHOE TYPES

The asymmetric bandage shoe FLENSBURG is available in a STANDARD and WIDE version.

### SOLE DESIGN


FLENSBURG has a 30mm thick, rigid and extra strong rocker bottom sole with pitch.

### INSOLES

Inside the shoe you will find three 5mm thick insoles made of EVA material (STANDARD F00053-XX and WIDE F00054-XX). The top insole has an antimicrobial coating.

### SPECIAL FEATURES

FLENSBURG, especially the WIDE shoe, provides much space for bandages and other specific demands. By using the dorsal zip, this shoe can be put on and taken off without even using the hook and loop fasteners. In addition, this shoe has an integrated toe cap for more stability and protection in the toe area.

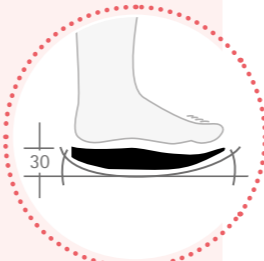

## FLENSBURG STANDARD

Shoe size	Art. no. pair	Art. no. left shoe	Art. no. right shoe
35	TS5311-35	TS5311-35/L	TS5311-35/R
36	TS5311-36	TS5311-36/L	TS5311-36/R
37	TS5311-37	TS5311-37/L	TS5311-37/R
38	TS5311-38	TS5311-38/L	TS5311-38/R
39	TS5311-39	TS5311-39/L	TS5311-39/R
40	TS5311-40	TS5311-40/L	TS5311-40/R
41	TS5311-41	TS5311-41/L	TS5311-41/R
42	TS5311-42	TS5311-42/L	TS5311-42/R
43	TS5311-43	TS5311-43/L	TS5311-43/R
44	TS5311-44	TS5311-44/L	TS5311-44/R
45	TS5311-45	TS5311-45/L	TS5311-45/R
46	TS5311-46	TS5311-46/L	TS5311-46/R
47	TS5311-47	TS5311-47/L	TS5311-47/R
48	TS5311-48	TS5311-48/L	TS5311-48/R
49	TS5311-49	TS5311-49/L	TS5311-49/R
50	TS5311-50	TS5311-50/L	TS5311-50/R

## FLENSBURG WIDE

Shoe size	Art. no. pair	Art. no. left shoe	Art. no. right shoe
35	TS5911-35	TS5911-35/L	TS5911-35/R
36	TS5911-36	TS5911-36/L	TS5911-36/R
37	TS5911-37	TS5911-37/L	TS5911-37/R
38	TS5911-38	TS5911-38/L	TS5911-38/R
39	TS5911-39	TS5911-39/L	TS5911-39/R
40	TS5911-40	TS5911-40/L	TS5911-40/R
41	TS5911-41	TS5911-41/L	TS5911-41/R
42	TS5911-42	TS5911-42/L	TS5911-42/R
43	TS5911-43	TS5911-43/L	TS5911-43/R
44	TS5911-44	TS5911-44/L	TS5911-44/R
45	TS5911-45	TS5911-45/L	TS5911-45/R
46	TS5911-46	TS5911-46/L	TS5911-46/R
47	TS5911-47	TS5911-47/L	TS5911-47/R
48	TS5911-48	TS5911-48/L	TS5911-48/R
49	TS5911-49	TS5911-49/L	TS5911-49/R
50	TS5911-50	TS5911-50/L	TS5911-50/R





## ERFURT STANDARD

Shoe size	Art. no. pair	Art. no. left shoe	Art. no. right shoe
35	TS5310-35	TS5310-35/L	TS5310-35/R
36	TS5310-36	TS5310-36/L	TS5310-36/R
37	TS5310-37	TS5310-37/L	TS5310-37/R
38	TS5310-38	TS5310-38/L	TS5310-38/R
39	TS5310-39	TS5310-39/L	TS5310-39/R
40	TS5310-40	TS5310-40/L	TS5310-40/R
41	TS5310-41	TS5310-41/L	TS5310-41/R
42	TS5310-42	TS5310-42/L	TS5310-42/R
43	TS5310-43	TS5310-43/L	TS5310-43/R
44	TS5310-44	TS5310-44/L	TS5310-44/R
45	TS5310-45	TS5310-45/L	TS5310-45/R
46	TS5310-46	TS5310-46/L	TS5310-46/R
47	TS5310-47	TS5310-47/L	TS5310-47/R
48	TS5310-48	TS5310-48/L	TS5310-48/R
49	TS5310-49	TS5310-49/L	TS5310-49/R
50	TS5310-50	TS5310-50/L	TS5310-50/R

## ERFURT WIDE

Shoe size	Art. no. pair	Art. no. left shoe	Art. no. right shoe
35	TS5410-35	TS5410-35/L	TS5410-35/R
36	TS5410-36	TS5410-36/L	TS5410-36/R
37	TS5410-37	TS5410-37/L	TS5410-37/R
38	TS5410-38	TS5410-38/L	TS5410-38/R
39	TS5410-39	TS5410-39/L	TS5410-39/R
40	TS5410-40	TS5410-40/L	TS5410-40/R
41	TS5410-41	TS5410-41/L	TS5410-41/R
42	TS5410-42	TS5410-42/L	TS5410-42/R
43	TS5410-43	TS5410-43/L	TS5410-43/R
44	TS5410-44	TS5410-44/L	TS5410-44/R
45	TS5410-45	TS5410-45/L	TS5410-45/R
46	TS5410-46	TS5410-46/L	TS5410-46/R
47	TS5410-47	TS5410-47/L	TS5410-47/R
48	TS5410-48	TS5410-48/L	TS5410-48/R
49	TS5410-49	TS5410-49/L	TS5410-49/R
50	TS5410-50	TS5410-50/L	TS5410-50/R

## PRODUCT INFORMATION

### INDICATIONS

ERFURT is particularly suited for patients with wounds at the foot/forefoot area on the sole of the foot (e.g. caused by traumatic injuries), swellings at the foot area (e.g. caused by inflammations) or joint pain in the foot area (e.g. caused by rheumatism or arthrosis) who need a closed shoe with a fastener system that can be widely opened and a tall shaft.

### SHOE TYPES

The asymmetric bandage shoe ERFURT is available in a STANDARD and WIDE version.

### SOLE DESIGN


ERFURT has a 30mm thick, rigid and extra strong rocker bottom sole with pitch.

### INSOLES

Inside the shoe you will find three 5mm thick insoles made of EVA material (STANDARD F00053-XX and WIDE F00054-XX). The top insole has an antimicrobial coating.

### SPECIAL FEATURES

ERFURT, especially the WIDE shoe, provides much space for bandages and other specific demands. By using the dorsal zip, this shoe can be put on and taken off without even opening the instep fastener. In addition, this shoe has an integrated toe cap for more stability and protection in the toe area.

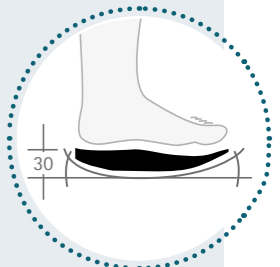





## PRODUCT INFORMATION

### INDICATIONS

AUGSBURG is particularly suited for patients with wounds at the foot/forefoot area on the sole of the foot (e.g. caused by traumatic injuries), swellings at the foot area (e.g. caused by inflammations) or joint pain in the foot area (e.g. caused by rheumatism or arthrosis) who need a shoe with an open toe area and a tall shaft.

### SHOE TYPES

The asymmetric bandage shoe AUGSBURG is available in a STANDARD and WIDE version.

### SOLE DESIGN

AUGSBURG has a 30mm thick, rigid and extra strong rocker bottom sole with pitch.

### INSOLES

Inside the shoe you will find three 5mm thick insoles made of EVA material (STANDARD F00053-XX and WIDE F00054-XX). The top insole has an antimicrobial coating.

### SPECIAL FEATURES

AUGSBURG, especially the WIDE shoe, provides much space for bandages and other specific demands. Besides, and if necessary, a protective forefoot cap can be attached to the toe area that keeps the foot dry and warm.



## AUGSBURG STANDARD

Shoe size	Art. no. pair	Art. no. left shoe	Art. no. right shoe
35	TS5381-35	TS5381-35/L	TS5381-35/R
36	TS5381-36	TS5381-36/L	TS5381-36/R
37	TS5381-37	TS5381-37/L	TS5381-37/R
38	TS5381-38	TS5381-38/L	TS5381-38/R
39	TS5381-39	TS5381-39/L	TS5381-39/R
40	TS5381-40	TS5381-40/L	TS5381-40/R
41	TS5381-41	TS5381-41/L	TS5381-41/R
42	TS5381-42	TS5381-42/L	TS5381-42/R
43	TS5381-43	TS5381-43/L	TS5381-43/R
44	TS5381-44	TS5381-44/L	TS5381-44/R
45	TS5381-45	TS5381-45/L	TS5381-45/R
46	TS5381-46	TS5381-46/L	TS5381-46/R
47	TS5381-47	TS5381-47/L	TS5381-47/R
48	TS5381-48	TS5381-48/L	TS5381-48/R
49	TS5381-49	TS5381-49/L	TS5381-49/R
50	TS5381-50	TS5381-50/L	TS5381-50/R

## AUGSBURG WIDE

Shoe size	Art. no. pair	Art. no. left shoe	Art. no. right shoe
35	TS5481-35	TS5481-35/L	TS5481-35/R
36	TS5481-36	TS5481-36/L	TS5481-36/R
37	TS5481-37	TS5481-37/L	TS5481-37/R
38	TS5481-38	TS5481-38/L	TS5481-38/R
39	TS5481-39	TS5481-39/L	TS5481-39/R
40	TS5481-40	TS5481-40/L	TS5481-40/R
41	TS5481-41	TS5481-41/L	TS5481-41/R
42	TS5481-42	TS5481-42/L	TS5481-42/R
43	TS5481-43	TS5481-43/L	TS5481-43/R
44	TS5481-44	TS5481-44/L	TS5481-44/R
45	TS5481-45	TS5481-45/L	TS5481-45/R
46	TS5481-46	TS5481-46/L	TS5481-46/R
47	TS5481-47	TS5481-47/L	TS5481-47/R
48	TS5481-48	TS5481-48/L	TS5481-48/R
49	TS5481-49	TS5481-49/L	TS5481-49/R
50	TS5481-50	TS5481-50/L	TS5481-50/R





PRODUCT INFORMATION

INDICATIONS

BREMEN is particularly suited for patients with wounds at the foot/forefoot area on the sole of the foot (e.g. caused by traumatic injuries), swellings at the foot area (e.g. caused by inflammations) or operations after other injuries who need a closed shoe with a generous fastener system.

SHOE TYPE

The bandage shoe BREMEN is symmetric and can be worn on the left as well as on the right.

SOLE DESIGN


BREMEN has a 30mm thick, rigid and extra strong rocker bottom sole with pitch.

INSOLES

Inside the shoe you will find an insole made of EVA material (F00064-XX) which has an antimicrobial coating.

SPECIAL FEATURES

BREMEN provides much space for bandages of different thickness and can be put on and taken off comfortably by using the generous instep fastener. In addition, this shoe has an integrated toe cap for more stability and protection in the toe area.

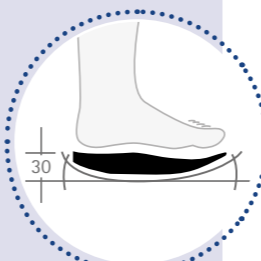

PRODUCT INFORMATION

INDICATIONS

TOKIO is particularly suited for patients with wounds at the foot/forefoot area on the sole of the foot (e.g. caused by traumatic injuries), swellings at the foot area (e.g. caused by inflammations) or operations after other injuries who need a closed shoe with a generous fastener system.

SHOE TYPE

The bandage shoe TOKIO is symmetric and can be worn on the left as well as on the right.

SOLE DESIGN


TOKIO has a 30mm thick, rigid and extra strong rocker bottom sole with pitch.

INSOLES

The shoe is delivered with a simple, 10mm thick insole made of EVA material.

SPECIAL FEATURES

TOKIO provides much space for bandages of different thickness and can be put on and taken off comfortably by using the generous instep fastener. In addition, this shoe has an integrated toe cap for more stability and protection in the toe area.

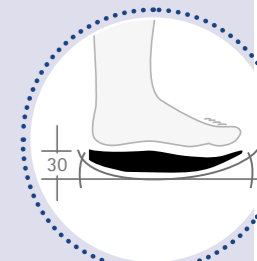

ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
XXS 32-33	89	209	TS1151-XXS	L 40-41	106	268	TS1151-L
XS 34-35	92	230	TS1151-XS	XL 42-43	109	285	TS1151-XL
S 36-37	100	236	TS1151-S	XXL 44-46	110	301	TS1151-XXL
M 38-39	102	254	TS1151-M				

ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
XXS 33-34	86	228	TS1150-XXS	L 41-42	102	276	TS1150-L
XS 35-36	92	239	TS1150-XS	XL 43-44	105	283	TS1150-XL
S 37-38	94	252	TS1150-S	XXL 45-47	111	309	TS1150-XXL
M 39-40	99	270	TS1150-M				





## PRODUCT INFORMATION

### INDICATIONS

STUTTGART is particularly suited for patients with wounds at the foot/forefoot area on the sole of the foot (e.g. caused by traumatic injuries), swellings at the foot area (e.g. caused by inflammations) or corrective osteotomies at the forefoot area (e.g. hallux valgus, hammer or claw toes) or other injuries who need a shoe with an open toe area.

### SHOE TYPE

The bandage shoe STUTTGART is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN

STUTTGART has a 30mm thick, rigid and extra strong rocker bottom sole with pitch.

### INSOLES

Inside the shoe you will find an insole made of EVA material (F00063-XX) which has an antimicrobial coating.

### SPECIAL FEATURES

STUTTGART has an open toe area and can be adapted to different needs by using the three long hook and loop fasteners.



## PRODUCT INFORMATION

### INDICATIONS

MADRID is particularly suited for patients with wounds at the foot/forefoot area on the sole of the foot (e.g. caused by traumatic injuries), swellings at the foot area (e.g. caused by inflammations) or corrective osteotomies at the forefoot area (e.g. hallux valgus, hammer or claw toes) or other injuries who need a shoe with an open toe area.

### SHOE TYPE

The bandage shoe MADRID is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN


MADRID has a 30mm thick, rigid and extra strong rocker bottom sole with pitch.

### INSOLES

The shoe is delivered with a simple, 10mm thick insole made of EVA material.

### SPECIAL FEATURES

MADRID has an open toe area and can be adapted to different needs by using the two long hook and loop fasteners.

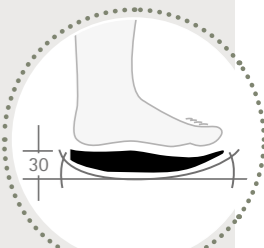

## ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
XXS 33-34	85	226	TS1111-XXS	L 41-42	104	285	TS1111-L
XS 35-36	91	241	TS1111-XS	XL 43-44	105	301	TS1111-XL
S 37-38	98	258	TS1111-S	XXL 45-47	112	318	TS1111-XXL
M 39-40	101	273	TS1111-M				

## ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
XXS 33-34	87	230	TS1000-XXS	L 41-42	104	286	TS1000-L
XS 35-36	90	243	TS1000-XS	XL 43-44	108	301	TS1000-XL
S 37-38	97	257	TS1000-S	XXL 45-47	112	316	TS1000-XXL
M 39-40	100	273	TS1000-M				





## PRODUCT INFORMATION

### INDICATIONS

AMSTERDAM is particularly suited for patients with wounds at the foot/forefoot area on the sole of the foot (e.g. caused by traumatic injuries), swellings at the foot area (e.g. caused by inflammations) or corrective osteotomies at the forefoot area (e.g. hallux valgus, hammer or claw toes) or other injuries who need a shoe with an open toe area and a tall shaft.

### SHOE TYPE

The bandage shoe AMSTERDAM is symmetric and can be worn on the left as well as on the right. AMSTERDAM is also available in a KIDS version.

### SOLE DESIGN

AMSTERDAM has a 30mm thick, rigid rocker bottom sole with an even top side.

### INSOLES

The shoe is delivered with a simple, 10mm thick insole made of EVA material.

### SPECIAL FEATURES

AMSTERDAM provides an open toe area with a strap. If necessary, a protective forefoot cap can be attached there to keep the foot dry and warm. With the practical hook and loop fastener in the heel area, the shaft width can be adapted. Furthermore, AMSTERDAM is an appropriate **TwinShoe** and thus height compensation for the relief shoes LONDON and MAILAND.



## PRODUCT INFORMATION

### INDICATIONS

FREIBURG is particularly suited for patients with wounds at the foot/forefoot area on the sole of the foot (e.g. caused by traumatic injuries), swellings at the foot area (e.g. caused by inflammations) or corrective osteotomies at the forefoot area (e.g. hallux valgus, hammer or claw toes) or other injuries who need a shoe with an open toe area and a tall shaft.

### SHOE TYPE

The bandage shoe FREIBURG is symmetric and can be worn on the left as well as on the right. FREIBURG is also available in a KIDS version.

### SOLE DESIGN

FREIBURG has a 30mm thick, rigid rocker bottom sole with an even top side.

### INSOLES

Inside the shoe you will find an insole made of EVA material (F00060-XX) which has an antimicrobial coating.

### SPECIAL FEATURES

FREIBURG provides an open toe area with a strap. If necessary, a protective forefoot cap can be attached there to keep the foot dry and warm. With the practical hook and loop fastener in the heel area, the shaft width can be adapted. Furthermore, FREIBURG is an appropriate **TwinShoe** and thus height compensation for the relief shoes MAINZ and HANNOVER.




## ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece		
	a	c			a	c			
XSK	31-33	86	221	TS6100-XSK	L	40-42	98	280	TS6100-L
XS	31-33	86	220	TS6100-XS	XL	43-45	106	302	TS6100-XL
S	34-36	92	216	TS6100-S	XXL	46-48	116	320	TS6100-XXL
M	37-39	96	261	TS6100-M					

## ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece		
	a	c			a	c			
XSK	31-33	83	223	TS6101-XSK	L	40-42	98	285	TS6101-L
XS	31-33	83	223	TS6101-XS	XL	43-45	100	301	TS6101-XL
S	34-36	93	244	TS6101-S	XXL	46-48	116	323	TS6101-XXL
M	37-39	95	262	TS6101-M					



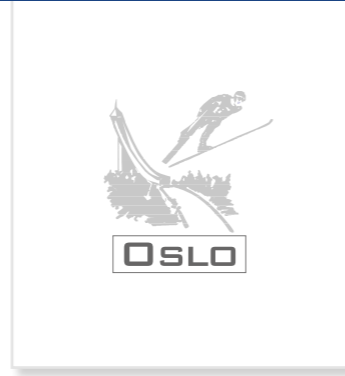

PRODUCT INFORMATION

INDICATIONS

OSLO is particularly suited for patients with wounds at the foot/forefoot area on the sole of the foot (e.g. caused by traumatic injuries), swellings at the foot area (e.g. caused by inflammations) or operations after other injuries who need a shoe with an open forefoot area and a tall shaft.

SHOE TYPE

The bandage shoe OSLO is symmetric and can be worn on the left as well as on the right.

SOLE DESIGN


OSLO has a 30mm thick, rigid rocker bottom sole with an even top side.

INSOLES

The shoe is delivered with a simple, 10mm thick insole made of EVA material.

SPECIAL FEATURES

OSLO provides an open forefoot area with a low framing brim. If necessary, a protective forefoot cap can be attached there to keep the foot dry and warm. With the practical hook and loop fastener in the heel area, the shaft width can be adapted. Furthermore, OSLO is an appropriate **TwinShoe** and thus height compensation for the relief shoe STOCKHOLM.

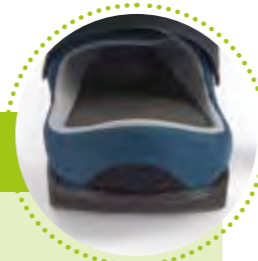

PRODUCT INFORMATION

INDICATIONS

HAMBURG is particularly suited for patients with wounds at the foot/forefoot area on the sole of the foot (e.g. caused by traumatic injuries), swellings at the foot area (e.g. caused by inflammations) or operations after other injuries who need a shoe with an open forefoot area and a tall shaft.

SHOE TYPE

The bandage shoe HAMBURG is symmetric and can be worn on the left as well as on the right.

SOLE DESIGN


HAMBURG has a 30mm thick, rigid rocker bottom sole with an even top side.

INSOLES

Inside the shoe you will find an insole made of EVA material (F00062-XX) which has an antimicrobial coating.

SPECIAL FEATURES

HAMBURG provides an open forefoot area with a low framing brim. If necessary, a protective forefoot cap can be attached there to keep the foot dry and warm. With the practical hook and loop fastener in the heel area, the shaft width can be adapted. Furthermore, HAMBURG is an appropriate **TwinShoe** and thus height compensation for the relief shoe KÖLN.

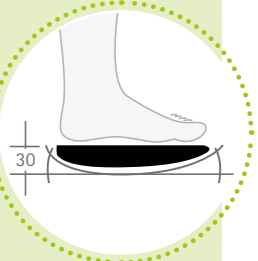

ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece		
	a	c			a	c			
XS	30-32	79	215	TS6120-XS	L	39-41	99	274	TS6120-L
S	33-35	93	235	TS6120-S	XL	42-44	103	300	TS6120-XL
M	36-38	97	257	TS6120-M	XXL	45-47	116	320	TS6120-XXL

ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece		
	a	c			a	c			
XS	30-32	88	216	TS6121-XS	L	39-41	98	281	TS6121-L
S	33-35	92	237	TS6121-S	XL	42-44	104	297	TS6121-XL
M	36-38	96	262	TS6121-M	XXL	45-47	118	322	TS6121-XXL





PRODUCT INFORMATION

**INDICATIONS**

REGENSBURG is particularly suited for patients with operations after an ankle joint fracture who need an individually adaptable shoe with tall shaft which is suitable for an external fixator.

**SHOE TYPE**

The bandage shoe REGENSBURG is symmetric and can be worn on the left as well as on the right.

**SOLE DESIGN**


REGENSBURG has a 30mm thick rocker bottom sole with an even top side.

**INSOLES**

Inside the shoe you will find an insole made of EVA material (F00060-XX) which has an antimicrobial coating.

**SPECIAL FEATURES**

REGENSBURG has additional seams in the ankle area so that the material can be cut neatly there in order to achieve more space, if needed [e.g. for an external fixator].

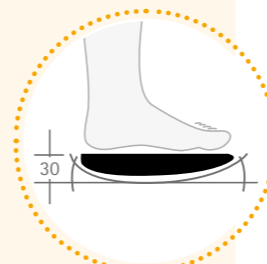

PRODUCT INFORMATION

**INDICATIONS**

ROSTOCK is particularly suited for patients with wounds at the heel area on the heel (e.g. caused by traumatic injuries) who need a shoe with an open heel area and a tall shaft.

**SHOE TYPE**

The bandage shoe ROSTOCK is symmetric and can be worn on the left as well as on the right.

**SOLE DESIGN**


ROSTOCK has a 30mm thick rocker bottom sole with an even top side.

**INSOLES**

Inside the shoe you will find an insole made of EVA material (F00069-XX) which has an antimicrobial coating.

**SPECIAL FEATURES**

ROSTOCK provides an open forefoot area with a low framing brim. If necessary, a protective forefoot cap can be attached there to keep the foot dry and warm.

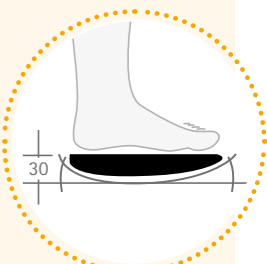

ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
XS 31-33	82	219	TS6141-XS	L 40-42	94	275	TS6141-L
S 34-36	92	240	TS6141-S	XL 43-45	98	290	TS6141-XL
M 37-39	94	252	TS6141-M	XXL 46-48	115	315	TS6141-XXL

ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
XS 30-32	85	220	TS6131-XS	L 39-41	90	272	TS6131-L
S 33-35	86	238	TS6131-S	XL 42-44	97	293	TS6131-XL
M 36-38	90	256	TS6131-M	XXL 45-47	112	311	TS6131-XXL





## PRODUCT INFORMATION

### INDICATIONS

KOPENHAGEN is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or corrective osteotomies at the forefoot area (e.g. hallux valgus, hammer or claw toes) or other injuries who need a shoe with an open toe area, a tall shaft and a thick sole.

### SHOE TYPE

The forefoot relief shoe KOPENHAGEN is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN


KOPENHAGEN has a 40mm thick, rigid and extra strong rocker bottom sole with an even top side.

### INSOLES

The shoe is delivered with a simple, 10mm thick insole made of EVA material.

### SPECIAL FEATURES

KOPENHAGEN provides an open toe area with a strap. If necessary, a protective forefoot cap can be attached there to keep the foot dry and warm. With the practical hook and loop fastener in the heel area, the shaft width can be adapted.

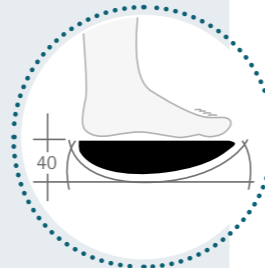

## ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
XS 31-33	84	224	F07100-XS	L 40-42	100	280	F07100-L
S 34-36	91	239	F07100-S	XL 43-45	109	303	F07100-XL
M 37-39	94	258	F07100-M	XXL 46-48	116	319	F07100-XXL

## PRODUCT INFORMATION

### INDICATIONS

FRANKFURT is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or corrective osteotomies at the forefoot area (e.g. hallux valgus, hammer or claw toes) or other injuries who need a shoe with an open toe area, a tall shaft and a thick sole.

### SHOE TYPE

The forefoot relief shoe FRANKFURT is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN


FRANKFURT has a 40mm thick, rigid and extra strong rocker bottom sole with an even top side.

### INSOLES

Inside the shoe you will find an insole made of EVA material (F00060-XX) which has an antimicrobial coating.

### SPECIAL FEATURES

FRANKFURT provides an open toe area with a strap. If necessary, a protective forefoot cap can be attached there to keep the foot dry and warm. With the practical hook and loop fastener in the heel area, the shaft width can be adapted.

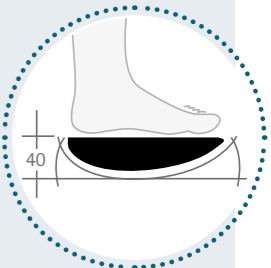

## ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
XS 31-33	85	222	F07101-XS	L 40-42	98	283	F07101-L
S 34-36	92	240	F07101-S	XL 43-45	101	296	F07101-XL
M 37-39	96	260	F07101-M	XXL 46-48	119	320	F07101-XXL





## PRODUCT INFORMATION

### INDICATIONS

HELSINKI is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or operations after other injuries who need a shoe with an open forefoot area, a tall shaft and a thick sole.

### SHOE TYPE

The forefoot relief shoe HELSINKI is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN



HELSINKI has a 40mm thick, rigid and extra strong rocker bottom sole with an even top side.

### INSOLES

The shoe is delivered with a simple, 10mm thick insole made of EVA material.

### SPECIAL FEATURES

HELSINKI provides an open forefoot area with a low framing brim. If necessary, a protective forefoot cap can be attached there to keep the foot dry and warm.

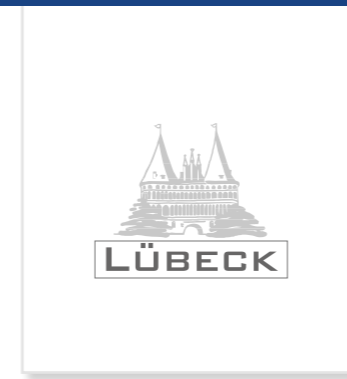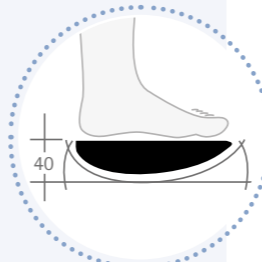

## PRODUCT INFORMATION

### INDICATIONS

LÜBECK is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or operations after other injuries who need a shoe with an open forefoot area, a tall shaft and a thick sole.

### SHOE TYPE

The forefoot relief shoe LÜBECK is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN


LÜBECK has a 40mm thick, rigid and extra strong rocker bottom sole with an even top side.

### INSOLES

Inside the shoe you will find an insole made of EVA material (F00062-XX) which has an antimicrobial coating.

### SPECIAL FEATURES

LÜBECK provides an open forefoot area with a low framing brim. If necessary, a protective forefoot cap can be attached there to keep the foot dry and warm.

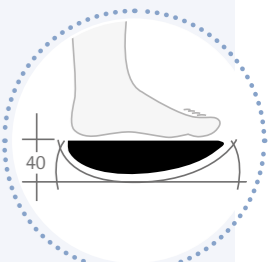

## ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
XS 30-32	88	212	F07120-XS	L 39-41	95	269	F07120-L
S 33-35	92	230	F07120-S	XL 42-44	106	288	F07120-XL
M 36-38	93	249	F07120-M	XXL 45-47	115	309	F07120-XXL

## ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
XS 30-32	86	215	F07121-XS	L 39-41	99	277	F07121-L
S 33-35	93	240	F07121-S	XL 42-44	102	299	F07121-XL
M 36-38	95	260	F07121-M	XXL 45-47	114	319	F07121-XXL





PRODUCT INFORMATION

INDICATIONS

LONDON is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or corrective osteotomies at the forefoot area (e.g. hallux valgus, hammer or claw toes) or other injuries who need a shoe with an open toe area and a tall shaft.

SHOE TYPE

The forefoot relief shoe LONDON is symmetric and can be worn on the left as well as on the right.

SOLE DESIGN


LONDON has a 30mm thick, rigid 8° wedge sole (60:40) with forefoot rocker. Due to the wedge sole, the foot's heel to toe movement is changed and the plantar pressure on the forefoot area is immensely reduced.

INSOLES

The shoe is delivered with a simple, 10mm thick insole made of EVA material.

SPECIAL FEATURES

LONDON provides an open toe area with a strap. If necessary, a protective forefoot cap can be attached there to keep the foot dry and warm. With the practical hook and loop fastener in the heel area, the shaft width can be adapted. Furthermore, AMSTERDAM can be worn as **TwinShoe** and thus height compensation for LONDON.

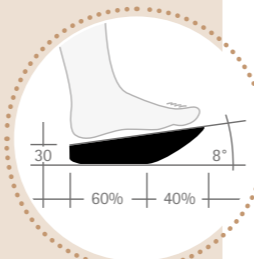

ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
XS 31-33	88	224	F01600-XS	L 40-42	99	279	F01600-L
S 34-36	92	239	F01600-S	XL 43-45	105	303	F01600-XL
M 37-39	94	261	F01600-M	XXL 46-48	116	323	F01600-XXL



PRODUCT INFORMATION

INDICATIONS

MAINZ is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or corrective osteotomies at the forefoot area (e.g. hallux valgus, hammer or claw toes) or other injuries who need a shoe with an open toe area and a tall shaft.

SHOE TYPE

The forefoot relief shoe MAINZ is symmetric and can be worn on the left as well as on the right.

SOLE DESIGN


MAINZ has a 30mm thick, rigid 8° wedge sole (60:40) with forefoot rocker. Due to the wedge sole, the foot's heel to toe movement is changed and the plantar pressure on the forefoot area is immensely reduced.

INSOLES

Inside the shoe you will find an insole made of EVA material (F00060-XX) which has an antimicrobial coating.

SPECIAL FEATURES

MAINZ provides an open toe area with a strap. If necessary, a protective forefoot cap can be attached there to keep the foot dry and warm. With the practical hook and loop fastener in the heel area, the shaft width can be adapted. Furthermore, FREIBURG can be worn as **TwinShoe** and thus height compensation for MAINZ.

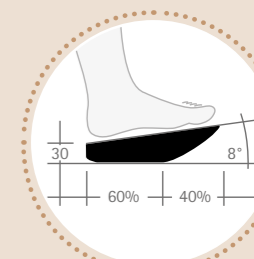

ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
XS 31-33	84	222	F01601-XS	L 40-42	97	285	F01601-L
S 34-36	93	245	F01601-S	XL 43-45	103	303	F01601-XL
M 37-39	95	265	F01601-M	XXL 46-48	114	325	F01601-XXL





KIDS



## PRODUCT INFORMATION

### INDICATIONS

HANNOVER is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or corrective osteotomies at the forefoot area (e.g. hallux valgus, hammer or claw toes) or other injuries who need a shoe with an open toe area, a tall shaft and forefoot support.

### SHOE TYPE

The forefoot relief shoe HANNOVER is symmetric and can be worn on the left as well as on the right. HANNOVER is also available in a KIDS version.

### SOLE DESIGN


HANNOVER has a 30mm thick, rigid 8° wedge sole (60:40) with forefoot support which changes the foot's heel to toe movement and relieves the forefoot. The rolling off line can be shifted by grinding the sole.

### INSOLES

Inside the shoe you will find an insole made of EVA material (F00060-XX) which has an antimicrobial coating.

### SPECIAL FEATURES

HANNOVER provides an open toe area with a strap. If necessary, a protective forefoot cap can be attached there to keep the foot dry and warm. With the practical hook and loop fastener in the heel area, the shaft width can be adapted. Furthermore, FREIBURG can be worn as **TwinShoe** and thus height compensation for HANNOVER.

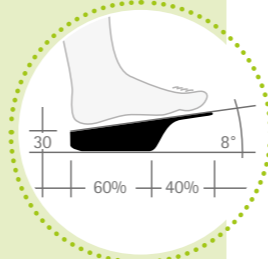

## ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece		
	a	c			a	c			
XSK	31-33	86	221	F01101-XSK	L	40-42	96	282	F01101-L
XS	31-33	83	222	F01101-XS	XL	43-45	100	298	F01101-XL
S	34-36	92	245	F01101-S	XXL	46-48	113	325	F01101-XXL
M	37-39	94	261	F01101-M					



KIDS



## PRODUCT INFORMATION

### INDICATIONS

MAILAND is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or corrective osteotomies at the forefoot area (e.g. hallux valgus, hammer or claw toes) or other injuries who need a shoe with an open toe area, a tall shaft and forefoot support.

### SHOE TYPE

The forefoot relief shoe MAILAND is symmetric and can be worn on the left as well as on the right. MAILAND is also available in a KIDS version.

### SOLE DESIGN


MAILAND has a 30mm thick, rigid 8° wedge sole (60:40) with forefoot support which changes the foot's heel to toe movement and relieves the forefoot. The rolling off line can be shifted by grinding the sole.

### INSOLES

The shoe is delivered with a simple, 10mm thick insole made of EVA material.

### SPECIAL FEATURES


MAILAND provides an open toe area with a strap. If necessary, a protective forefoot cap can be attached there to keep the foot dry and warm. With the practical hook and loop fastener in the heel area, the shaft width can be adapted. Furthermore, AMSTERDAM can be worn as **TwinShoe** and thus height compensation for MAILAND.

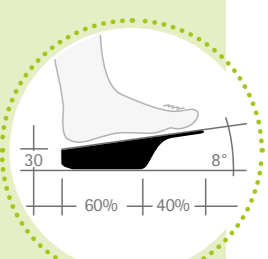

## ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece		
	a	c			a	c			
XSK	31-33	88	224	F01100-XSK	L	40-42	100	281	F01100-L
XS	31-33	89	223	F01100-XS	XL	43-45	104	304	F01100-XL
S	34-36	90	242	F01100-S	XXL	46-48	119	323	F01100-XXL
M	37-39	92	261	F01100-M					



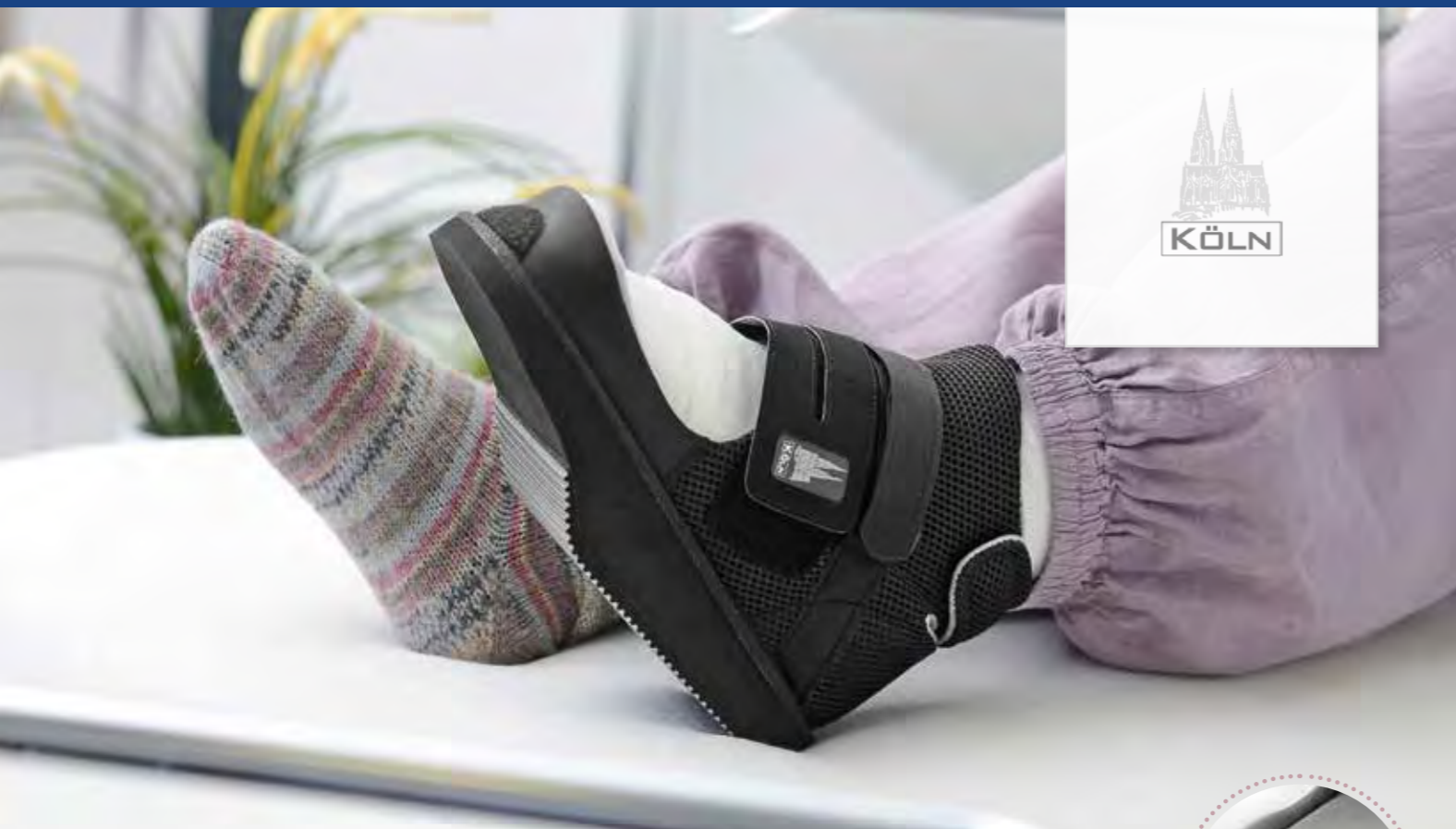

## PRODUCT INFORMATION

### INDICATIONS

KÖLN is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or (operations after) other injuries who need a shoe with an open forefoot area, a tall shaft and forefoot support.

### SHOE TYPE

The forefoot relief shoe KÖLN is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN


KÖLN has a 30mm thick, rigid 8° wedge sole (60:40) with forefoot support which changes the foot's heel to toe movement and relieves the forefoot. The rolling off line can be shifted by grinding the sole.

### INSOLES

Inside the shoe you will find an insole made of EVA material (F00062-XX) which has an antimicrobial coating.

### SPECIAL FEATURES

KÖLN provides an open forefoot area with a low framing brim. If necessary, a protective forefoot cap can be attached there to keep the foot dry and warm. With the practical hook and loop fastener in the heel area, the shaft width can be adapted. Furthermore, HAMBURG can be worn as **TwinShoe** and thus height compensation for KÖLN.

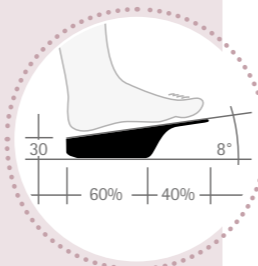

## PRODUCT INFORMATION

### INDICATIONS

STOCKHOLM is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or (operations after) other injuries who need a shoe with an open forefoot area, a tall shaft and forefoot support.

### SHOE TYPE

The forefoot relief shoe STOCKHOLM is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN


STOCKHOLM has a 30mm thick, rigid 8° wedge sole (60:40) with forefoot support which changes the foot's heel to toe movement and relieves the forefoot. The rolling off line can be shifted by grinding the sole.

### INSOLES

The shoe is delivered with a simple, 10mm thick insole made of EVA material.

### SPECIAL FEATURES

STOCKHOLM provides an open forefoot area with a low framing brim. If necessary, a protective forefoot cap can be attached there to keep the foot dry and warm. With the practical hook and loop fastener in the heel area, the shaft width can be adapted. Furthermore, OSLO can be worn as **TwinShoe** and thus height compensation for STOCKHOLM.

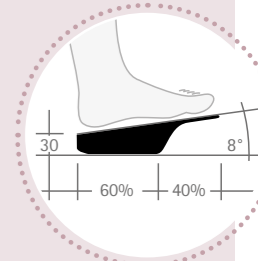

## ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
XS 30-32	82	219	F01121-XS	L 39-41	94	275	F01121-L
S 33-35	92	240	F01121-S	XL 42-44	98	290	F01121-XL
M 36-38	94	252	F01121-M	XXL 45-47	115	315	F01121-XXL

## ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
XS 30-32	85	220	F01120-XS	L 39-41	90	272	F01120-L
S 33-35	86	238	F01120-S	XL 42-44	97	293	F01120-XL
M 36-38	90	256	F01120-M	XXL 45-47	112	311	F01120-XXL





## PRODUCT INFORMATION

### INDICATIONS

STENDAL is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or corrective osteotomies at the forefoot area (e.g. hallux valgus, hammer or claw toes) or other injuries who need a shoe with an open toe area, a tall shaft and forefoot support.

### SHOE TYPE

The forefoot relief shoe STENDAL is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN


STENDAL has a 30mm thick, rigid 10° wedge sole (50:50) with forefoot support which changes the foot's heel to toe movement and relieves the forefoot. The rolling off line can be shifted by grinding the sole.

### INSOLES

The shoe is delivered with a simple, 10mm thick insole made of EVA material.

### SPECIAL FEATURES

STENDAL provides an open toe area with a strap. If necessary, a protective forefoot cap can be attached there to keep the foot dry and warm. With the practical hook and loop fastener in the heel area, the shaft width can be adapted.

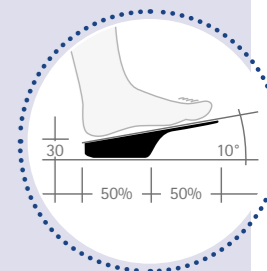

## ORDER INFORMATION

	Shoe size		Inner shoe dimensions		Art. no. piece		Shoe size		Inner shoe dimensions		Art. no. piece
			a	c					a	c	
XS	31-33		88	221	F01160-XS	L	40-42	99	282	F01160-L	
S	34-36		92	241	F01160-S	XL	43-45	107	305	F01160-XL	
M	37-39		97	261	F01160-M	XXL	46-48	118	324	F01160-XXL	





## PRODUCT INFORMATION

### INDICATIONS

LÜNEBURG is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or corrective osteotomies at the forefoot area (e.g. hallux valgus, hammer or claw toes) or other injuries who need a shoe with an open toe area, a tall shaft, a thick sole and forefoot support.

### SHOE TYPE

The forefoot relief shoe LÜNEBURG is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN


LÜNEBURG has a 40mm thick, rigid 10° wedge sole (50:50) with forefoot support which changes the foot's heel to toe movement and relieves the forefoot.

### INSOLES

Inside the shoe you will find an insole made of EVA material (F00068-XX) which has an antimicrobial coating.

### SPECIAL FEATURES

LÜNEBURG provides an open toe area with a strap. If necessary, a protective forefoot cap can be attached there to keep the foot dry and warm. With the practical hook and loop fastener in the heel area, the shaft width can be adapted.

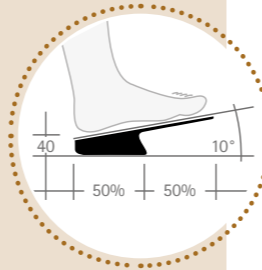

## ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
S 31-33	84	219	F01201-S	XL 41-44	102	288	F01201-XL
M 34-37	98	249	F01201-M	XXL 45-49	117	326	F01201-XXL
L 38-40	99	266	F01201-L				





## PRODUCT INFORMATION

### INDICATIONS

BERLIN is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or (partial) forefoot amputations or operations after other injuries at the forefoot area who need a shoe with shortened, thick sole and a tall shaft without forefoot support.

### SHOE TYPE

The forefoot relief shoe BERLIN is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN


BERLIN has a 40mm thick, rigid 10° wedge sole (50:50) without forefoot support. Due to the shortened sole, the foot's heel to toe movement is changed and the forefoot area is completely relieved.

### INSOLES

Inside the shoe you will find an insole made of EVA material (FO0011-XX) with soft surface.

### SPECIAL FEATURES

BERLIN provides a shortened wedge sole and thus ensures complete pressure relief of the forefoot area. If necessary, a protective forefoot cap can be attached to the insole to keep the foot dry and warm. Furthermore, BERLIN can be equipped subsequently with a protective stirrup (FO1000-Z5,-Z6,-Z7,-Z8).

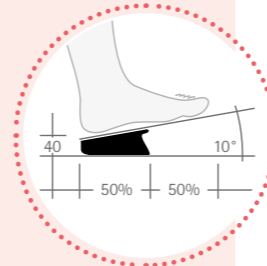

## BERLIN

Shoe size	Inner shoe dimension c	Art. no. piece	Shoe size	Inner shoe dimension c	Art. no. piece
S 36-38	164	FO1002-S	L 43-46	191	FO1002-L
M 39-42	172	FO1002-M	XL 47-51	195	FO1002-XL

## PRODUCT INFORMATION

### INDICATIONS

BERLIN with protective stirrup is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or (partial) forefoot amputations or operations after other injuries at the forefoot area who need a shoe with shortened, thick sole and a tall shaft without forefoot support, but with additional protection against bumps.

### SHOE TYPE

The forefoot relief shoe BERLIN with protective stirrup is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN


BERLIN with protective stirrup has a 40mm thick, rigid 10° wedge sole (50:50) without forefoot support. Due to the shortened sole, the foot's heel to toe movement is changed and the forefoot area is completely relieved.

### INSOLES

Inside the shoe you will find an insole made of EVA material (FO0011-XX) with soft surface.

### SPECIAL FEATURES

BERLIN with protective stirrup provides additional protection against injuries (e.g. bumps). The protective stirrup is adjustable in length as well thermoformable and can therefore be individually adapted. If necessary, a protective forefoot cap can be attached to the insole to keep the foot dry and warm.

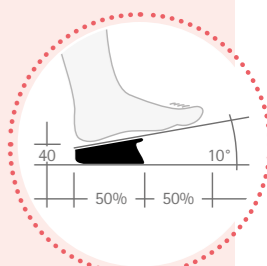

## BERLIN with protective stirrup

Shoe size	Inner shoe dimension c	Art. no. piece	Shoe size	Inner shoe dimension c	Art. no. piece
S 36-38	164	FO1012-S	L 43-46	191	FO1012-L
M 39-42	172	FO1012-M	XL 47-51	195	FO1012-XL





## PRODUCT INFORMATION

### INDICATIONS

BORDEAUX is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or forefoot amputations or operations after other injuries at the forefoot area who need a shoe with shortened, thick sole and without forefoot support.

### SHOE TYPE

The forefoot relief shoe BORDEAUX is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN


BORDEAUX has a 40mm thick, rigid 10° wedge sole (50:50) without forefoot support. Due to the shortened sole, the foot's heel to toe movement is changed and the forefoot area is completely relieved.

### INSOLES

The shoe is delivered with a simple, 10mm thick insole made of EVA material.

### SPECIAL FEATURES

BORDEAUX provides a shortened wedge sole and thus ensures complete pressure relief of the forefoot area. If necessary, a protective forefoot cap can be attached to the insole to keep the foot dry and warm. Furthermore, BORDEAUX can be equipped subsequently with a protective stirrup (FO1000-Z9, -Z10, -Z11, -Z12).

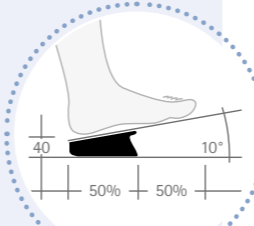

## BORDEAUX

Shoe size	Inner shoe dimension c	Art. no. piece	Shoe size	Inner shoe dimension c	Art. no. piece
S 35-37	159	FO1001-S	L 41-43	180	FO1001-L
M 38-40	170	FO1001-M	XL 44-46	188	FO1001-XL



## PRODUCT INFORMATION

### INDICATIONS

BORDEAUX with protective stirrup is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or forefoot amputations or operations after other injuries at the forefoot area who need a shoe with shortened, thick sole and without forefoot support, but with additional protection against bumps.

### SHOE TYPE

The forefoot relief shoe BORDEAUX with protective stirrup is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN


BORDEAUX with protective stirrup has a 40mm thick, rigid 10° wedge sole (50:50) without forefoot support. Due to the shortened sole, the foot's heel to toe movement is changed and the forefoot area is completely relieved.

### INSOLES

The shoe is delivered with a simple, 10mm thick insole made of EVA material.

### SPECIAL FEATURES

BORDEAUX with protective stirrup provides additional protection against injuries (e.g. bumps). The protective stirrup is adjustable in length as well thermoformable and can therefore be individually adapted. If necessary, a protective forefoot cap can be attached to the insole to keep the foot dry and warm.

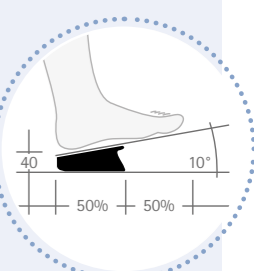

## BORDEAUX with protective stirrup

Shoe size	Inner shoe dimension c	Art. no. piece	Shoe size	Inner shoe dimension c	Art. no. piece
S 35-37	159	FO1011-S	L 41-43	180	FO1011-L
M 38-40	170	FO1011-M	XL 44-46	188	FO1011-XL





## PRODUCT INFORMATION

### INDICATIONS

KAISERSLAUTERN is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or forefoot amputations or operations after other injuries at the forefoot area who need a shoe with extremely shortened sole, tall shaft and without forefoot support.

### SHOE TYPE

The forefoot relief shoe KAISERSLAUTERN is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN


KAISERSLAUTERN has a 30mm thick, rigid 10° wedge sole (30:70) without forefoot support. Due to the extremely shortened sole, the foot's heel to toe movement is changed and the forefoot area is completely relieved.

### INSOLES

Inside the shoe you will find an insole made of EVA material (F00016-XX) with soft surface.

### SPECIAL FEATURES

KAISERSLAUTERN provides an extremely shortened wedge sole and thus ensures complete pressure relief of the forefoot area. Furthermore, KAISERSLAUTERN can be equipped subsequently with a protective stirrup (F01000-Z0,-Z1).

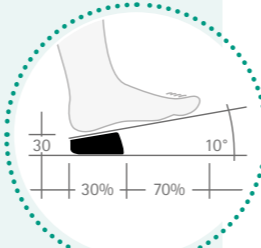

## KAISERSLAUTERN

Shoe size	Inner shoe dimension c	Art. no. piece
S 35-42	122	F01030-S
L 43-46	140	F01030-L



## PRODUCT INFORMATION

### INDICATIONS

KAISERSLAUTERN with protective stirrup is particularly suited for patients with wounds at the forefoot area on the sole of the foot (e.g. caused by traumatic injuries) or forefoot amputations or operations after other injuries at the forefoot area who need a shoe with extremely shortened sole, tall shaft and without forefoot support, but with additional protection against bumps.

### SHOE TYPE

The forefoot relief shoe KAISERSLAUTERN with protective stirrup is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN


KAISERSLAUTERN with protective stirrup has a 30mm thick, rigid 10° wedge sole (30:70) without forefoot support. Due to the extremely shortened sole, the foot's heel to toe movement is changed and the forefoot area is completely relieved.

### INSOLES

Inside the shoe you will find an insole made of EVA material (F00016-XX) with soft surface.

### SPECIAL FEATURES

KAISERSLAUTERN with protective stirrup provides additional protection against injuries (e.g. bumps). The protective stirrup is adjustable in length as well thermoformable and can therefore be individually adapted.

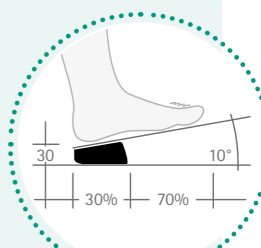

## KAISERSLAUTERN with protective stirrup

Shoe size	Inner shoe dimension c	Art. no. piece
S 35-42	122	F01031-S
L 43-46	140	F01031-L





## PRODUCT INFORMATION

### INDICATIONS

MÜNCHEN is particularly suited for patients with wounds at the heel area on the sole of the foot (e.g. caused by traumatic injuries) or operations after other injuries at the heel area (e.g. heel spur) who need a shoe with shortened sole, tall shaft and without heel support.

### SHOE TYPE

The heel relief shoe MÜNCHEN is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN


MÜNCHEN has a 30mm thick, in the forefoot area partially flexible 5° wedge sole (20:80) without heel support. Due to the dorsally shortened sole, the foot's heel to toe movement is changed and the heel area is completely relieved.

### INSOLES

Inside the shoe you will find an insole made of EVA material (F00020-XX) with soft surface.

### SPECIAL FEATURES

MÜNCHEN provides an open forefoot area with a low framing brim and an open heel area. If necessary, a protective forefoot cap and/or a protective heel cap can be attached.

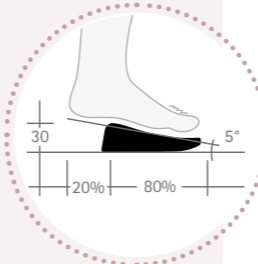

## PRODUCT INFORMATION

### INDICATIONS

MARSEILLE is particularly suited for patients with wounds at the heel area on the sole of the foot (e.g. caused by traumatic injuries) or operations after other injuries at the heel area (e.g. heel spur) who need a shoe with shortened sole, tall shaft and without heel support.

### SHOE TYPE

The heel relief shoe MARSEILLE is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN


MARSEILLE has a 30mm thick, in the forefoot area partially flexible 5° wedge sole (20:80) without heel support. Due to the dorsally shortened sole, the foot's heel to toe movement is changed and the heel area is completely relieved.

### INSOLES

The shoe is delivered with a simple insole made of EVA material.

### SPECIAL FEATURES

MARSEILLE provides an open forefoot area with a low framing brim and an open heel area. If necessary, a protective forefoot cap and/or a protective heel cap can be attached.

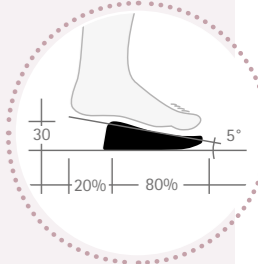

## ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
S 35-37	90	249	F02121-S	L 41-42	103	287	F02121-L
M 38-40	100	257	F02121-M	XL 43-46	104	297	F02121-XL

## ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
S 35-37	90	237	F02120-S	L 41-42	95	268	F02120-L
M 38-40	93	255	F02120-M	XL 43-46	102	288	F02120-XL





## PRODUCT INFORMATION

### INDICATIONS

WUPPERTAL is particularly suited for patients with wounds at the heel area on the sole of the foot (e.g. caused by traumatic injuries) or operations after other injuries at the heel area (e.g. heel spur) who need a shoe with shortened sole, tall shaft and without heel support.

### SHOE TYPE

The heel relief shoe WUPPERTAL is symmetric and can be worn on the left as well as on the right.

### SOLE DESIGN


WUPPERTAL has a 30mm thick, in the forefoot area partially flexible 5° wedge sole (20:80) without heel support. Due to the dorsally shortened sole, the foot's heel to toe movement is changed and the heel area is completely relieved.

### INSOLES

The shoe is delivered with a simple insole made of EVA material.

### SPECIAL FEATURES

WUPPERTAL provides an open toe and heel area and can be adapted to different needs by using the unusual hook and loop fasteners

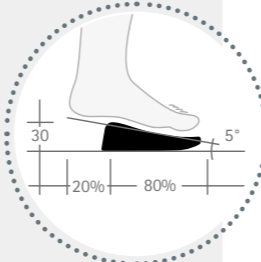

## ORDER INFORMATION

Shoe size	Inner shoe dimensions		Art. no. piece	Shoe size	Inner shoe dimensions		Art. no. piece
	a	c			a	c	
S 35-37	85	214	FO2000-S	L 41-42	94	251	FO2000-L
M 38-40	93	235	FO2000-M	XL 43-46	98	265	FO2000-XL





# INNER SHOE DIMENSIONS

## Dear Customer,

We measured our therapeutic shoes according to the Mondopoint system. Afterwards, the shoe sizes were recalculated by means of the Paris Point.

### Mondopoint System

The Mondopoint system is an international standardised system to indicate shoe sizes. In contrast to other systems, not only the inner shoe length but also the inner shoe width is measured.

### Paris Point


To determine the shoe sizes we used the Paris Point which is calculated as follows:

$$\text{shoe size} = \text{inner shoe length in cm} \times 1.5$$

Length and width are indicated in mm.

### ATTENTION!

In this catalogue, we state outdoor shoe sizes. These shoe sizes do not cover additional space for bandages or the like.

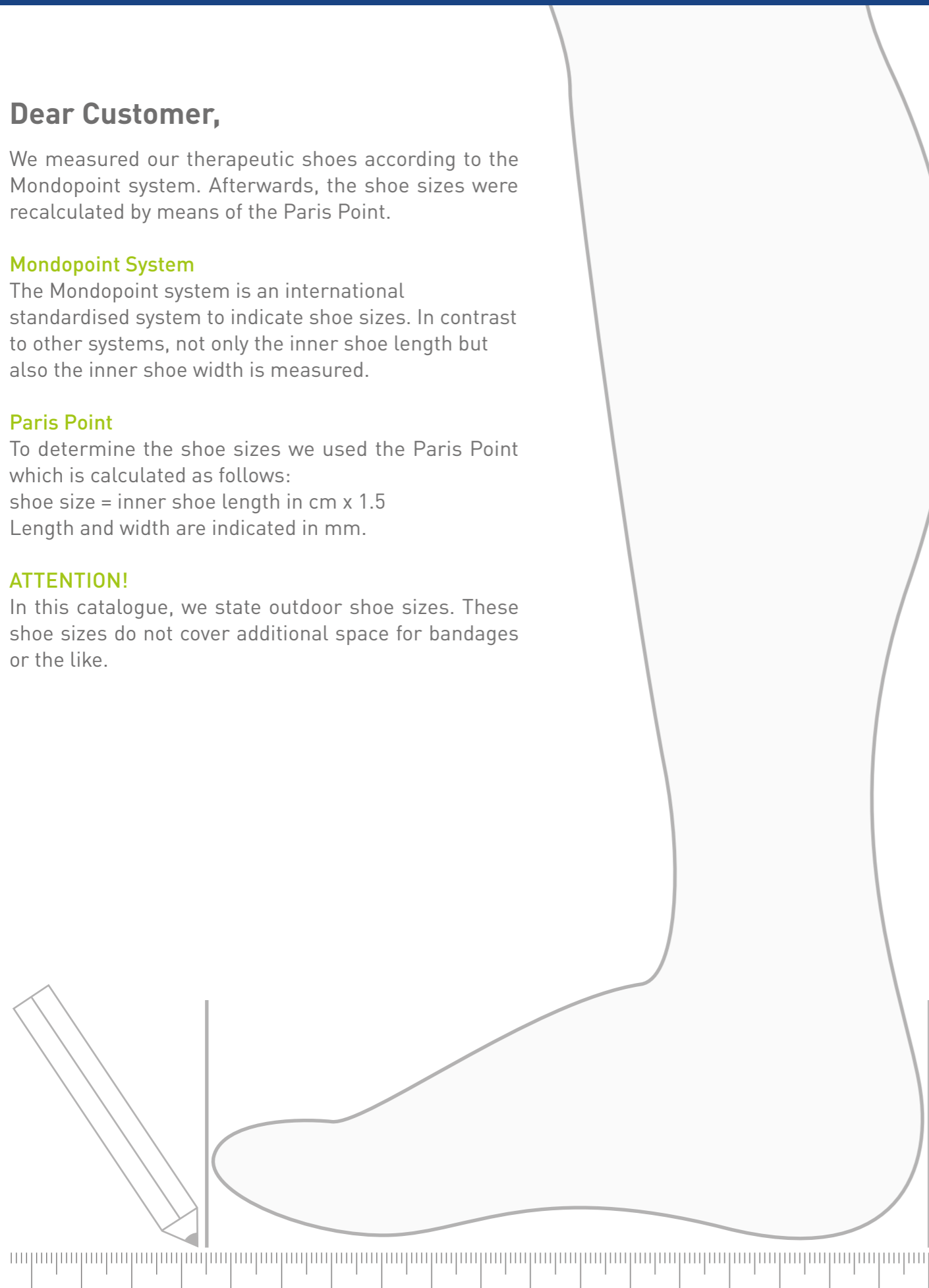

## ASYMMETRIC AND SYMMETRIC SHOES



### Inner shoe length

The inner shoe length (c) was measured from the middle of the heel area to the middle of the great toe with a standard measurement device.

### Inner shoe widths (ball and heel area)

The inner shoe widths were measured at the insole.

We measured from the toe-cap and drew a crossline at 1/3 of the insole for the inner shoe width in ball area. At this right-angled straight line a 15° angle was measured. The resulting line was where we measured the inner shoe width in ball area (a). The inner shoe width in heel area (b) was measured at the level of the middle of the heel.

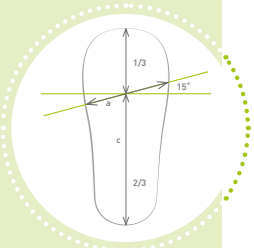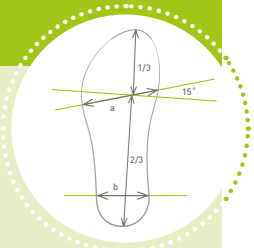

## INNER SHOE DIMENSIONS STANDARD (F00033 and F00053)

Shoe size	Inner shoe dimensions [mm]			Shoe size	Inner shoe dimensions [mm]		
	a	b	c		a	b	c
35	93	66	231	43	106	75	288
36	94	67	237	44	109	76	295
37	95	69	245	45	109	79	299
38	98	69	250	46	110	79	305
39	99	71	258	47	111	81	313
40	101	73	266	48	113	81	318
41	104	74	271	49	115	81	325
42	105	75	277	50	115	81	332

AUGSBURG STANDARD, ERFURT STANDARD, FLENSBURG STANDARD

## INNER SHOE DIMENSIONS WIDE (F00034 and F00054)

Shoe size	Inner shoe dimensions [mm]			Shoe size	Inner shoe dimensions [mm]		
	a	b	c		a	b	c
35	102	67	233	43	114	76	282
36	103	69	240	44	116	79	287
37	104	71	247	45	118	80	295
38	107	73	249	46	121	81	302
39	109	73	255	47	123	82	308
40	111	74	263	48	126	83	319
41	111	75	269	49	126	83	325
42	112	76	277	50	127	87	333

AUGSBURG WIDE, ERFURT WIDE, FLENSBURG WIDE



# SPARE AND ACCESSORY PARTS



The top insole is antimicrobial. The insoles are made of EVA material and provide a different shore hardness: grey = 46°, white = 23°, antimicrobial = 28°

INSOLE SET STANDARD			
Shoe size	Art. no. set	Shoe size	Art. no. set
35	F00053-35	43	F00053-43
36	F00053-36	44	F00053-44
37	F00053-37	45	F00053-45
38	F00053-38	46	F00053-46
39	F00053-39	47	F00053-47
40	F00053-40	48	F00053-48
41	F00053-41	49	F00053-49
42	F00053-42	50	F00053-50

AUGSBURG STANDARD, ERFURT STANDARD, FLENSBURG STANDARD

INSOLE SET WIDE			
Shoe size	Art. no. set	Shoe size	Art. no. set
35	F00054-35	43	F00054-43
36	F00054-36	44	F00054-44
37	F00054-37	45	F00054-45
38	F00054-38	46	F00054-46
39	F00054-39	47	F00054-47
40	F00054-40	48	F00054-48
41	F00054-41	49	F00054-49
42	F00054-42	50	F00054-50

AUGSBURG WIDE, ERFURT WIDE, FLENSBURG WIDE



The insoles are made of EVA material. The cover made of PA material is antimicrobial.



The insoles are made of EVA material and provide a different shore hardness: black = 40°, grey = 25°, white = 50°

INSOLE SET STANDARD			
Shoe size	Art. no. set	Shoe size	Art. no. set
35	F00033-35	43	F00033-43
36	F00033-36	44	F00033-44
37	F00033-37	45	F00033-45
38	F00033-38	46	F00033-46
39	F00033-39	47	F00033-47
40	F00033-40	48	F00033-48
41	F00033-41	49	F00033-49
42	F00033-42	50	F00033-50

AUGSBURG STANDARD, ERFURT STANDARD, FLENSBURG STANDARD

INSOLE SET WIDE			
Shoe size	Art. no. set	Shoe size	Art. no. set
35	F00034-35	43	F00034-43
36	F00034-36	44	F00034-44
37	F00034-37	45	F00034-45
38	F00034-38	46	F00034-46
39	F00034-39	47	F00034-47
40	F00034-40	48	F00034-48
41	F00034-41	49	F00034-49
42	F00034-42	50	F00034-50

AUGSBURG WIDE, ERFURT WIDE, FLENSBURG WIDE

ANTIMICROBIAL INSOLE	
Size	Art. no.
XS	F00060-XS
S	F00060-S
M	F00060-M
L	F00060-L
XL	F00060-XL
XXL	F00060-XXL

AMSTERDAM, FRANKFURT, FREIBURG, HANNOVER, KOPENHAGEN, LONDON, MAILAND, MAINZ, REGENSBURG

ANTIMICROBIAL INSOLE	
Size	Art. no.
XS	F00063-XS
S	F00063-S
M	F00063-M
L	F00063-L
XL	F00063-XL
XXL	F00063-XXL

STUTT GART, MADRID

ANTIMICROBIAL INSOLE	
Size	Art. no.
S	F00068-S
M	F00068-M
L	F00068-L
XL	F00068-XL
XXL	F00068-XXL

STENDAL, LÜNEBURG

ANTIMICROBIAL INSOLE	
Size	Art. no.
XS	F00062-XS
S	F00062-S
M	F00062-M
L	F00062-L
XL	F00062-XL
XXL	F00062-XXL

HAMBURG, HELSINKI, KÖLN, LÜBECK, OSLO, STOCKHOLM

ANTIMICROBIAL INSOLE	
Size	Art. no.
XS	F00064-XS
S	F00064-S
M	F00064-M
L	F00064-L
XL	F00064-XL
XXL	F00064-XXL

BREMEN, TOKIO

ANTIMICROBIAL INSOLE	
Size	Art. no.
XS	F00069-XS
S	F00069-S
M	F00069-M
L	F00069-L
XL	F00069-XL
XXL	F00069-XXL

ROSTOCK





The insoles are made of EVA material and a cushioning made of PA foam. The soft cover consists of woven synthetic fibre.

## INSOLES

Size	Art. no.
S	F00011-S
M	F00011-M
L	F00011-L
XL	F00011-XL

BERLIN, BORDEAUX

## INSOLES

Size	Art. no.
S	F00016-S
L	F00016-L

KAISERSLAUTERN

## INSOLES

Size	Art. no.
S	F00020-S
M	F00020-M
L	F00020-L
XL	F00020-XL

MÜNCHEN, MARSEILLE



## PROTECTIVE FOREFOOT CAP (ATTACHABLE INSIDE)

Size	Art. no.
XS	F09621-XS
S	F09621-S
M	F09621-M
L	F09621-L
XL	F09621-XL
XXL	F09621-XXL

AMSTERDAM, KOPENHAGEN, LONDON, MAILAND, STENDAL

The protective forefoot cap can easily be attached and keeps the foot dry and warm.



## PROTECTIVE FOREFOOT CAP

Size	Art. no.
XS	F09121-XS
S	F09121-S
M	F09121-M
L	F09121-L
XL	F09121-XL
XXL	F09121-XXL

STOCKHOLM, OSLO, MARSEILLE, HELSINKI

The protective forefoot cap can easily be attached and keeps the foot dry and warm.



## PROTECTIVE FOREFOOT CAP (ATTACHABLE INSIDE)

Size	Art. no.
XS	F09100-XS
S	F09100-S
M	F09100-M
L	F09100-L
XL	F09100-XL
XXL	F09100-XXL

KÖLN, LÜBECK, HAMBURG, MÜNCHEN, ROSTOCK

The protective forefoot cap can easily be attached inside the shoe and keeps the foot dry and warm.



## PROTECTIVE FOREFOOT CAP

Size	Art. no.
S	F09401-S
M	F09401-M
L	F09401-L
XL	F09401-XL

The protective forefoot cap can easily be attached to the BORDEAUX shoe and keeps the foot dry and warm.



## PROTECTIVE FOREFOOT CAP (ATTACHABLE OUTSIDE)

Size	Art. no.
XS	F09110-XS
S	F09110-S
M	F09110-M
L	F09110-L
XL	F09110-XL
XXL	F09110-XXL

KÖLN, LÜBECK, HAMBURG, MÜNCHEN, ROSTOCK

The protective forefoot cap can easily be attached outside the shoe and keeps the foot dry and warm.





## PROTECTIVE FOREFOOT CAP

Size	Art. no.
S	F09400-S
M	F09400-M
L	F09400-L
XL	F09400-XL

The protective forefoot cap can easily be attached to the BERLIN shoe and keeps the foot dry and warm.



## PROTECTIVE HEEL CAP

Size	Art. no.
S	F09301-S
M	F09301-M
L	F09301-L
XL	F09301-XL

The protective heel cap can easily be attached to the MÜNCHEN shoe and keeps the foot dry and warm.



## PROTECTIVE FOREFOOT CAP

Shoe size	Art. no.
35-36	F09580-XXS
37-38	F09580-XS
39-40	F09580-S
41-42	F09580-M
43-44	F09580-L
45-46	F09580-XL
47-48	F09580-XXL
49-50	F09580-XXXL

The protective forefoot cap can easily be attached to the AUGSBURG STANDARD and WIDE shoe and keeps the foot dry and warm.



## PROTECTIVE HEEL CAP

Size	Art. no.
S	F09320-S
M	F09320-M
L	F09320-L
XL	F09320-XL

The protective heel cap can easily be attached to the MARSEILLE shoe and keeps the foot dry and warm.



## PROTECTIVE FOREFOOT CAP

Size	Art. no.
XS	F09620-XS
S	F09620-S
M	F09620-M
L	F09620-L
XL	F09620-XL
XXL	F09620-XXL

HANNOVER, FREIBURG, LÜNEBURG, FRANKFURT, MAINZ

The protective forefoot cap can easily be attached outside the shoe and keeps the foot dry and warm.



## OUTSOLE MATERIAL, RUBBER

Art. no.	Unit	Dimensions [mm]
F00092-S	Sheet	220 x 110 x 1.5
F00092-M	Sheet	295 x 145 x 1.5
F00092-L	Sheet	357 x 158 x 1.5





HOOK AND LOOP FASTENER

Size	Art. no.	Measures
XS-XXL	F00002-S	50 x 200mm



HOOK AND LOOP FASTENER

Size	Art. no.	Measures
XXS-XXL	F00001-L	25 x 650mm



PROTECTIVE STIRRUP  
KAISERSLAUTERN

Size	Art. no.
S	F01000-Z0
L	F01000-Z1

including assembly set



PROTECTIVE STIRRUP BERLIN

Size	Art. no.
S	F01000-Z5
M	F01000-Z6
L	F01000-Z7
XL	F01000-Z8

including assembly set

PROTECTIVE STIRRUP BORDEAUX

Size	Art. no.
S	F01000-Z9
M	F01000-Z10
L	F01000-Z11
XL	F01000-Z12

including assembly set

THERAPEUTIC SHOES  
DIABETES



You will find our therapeutic shoe catalogue  
DIABETES on our website in the download area:  
[www.fior-gentz.com/download](http://www.fior-gentz.com/download)

THERAPEUTIC SHOES  
HOMECARE



You will find our therapeutic shoe catalogue  
HOMECARE on our website in the download area:  
[www.fior-gentz.com/download](http://www.fior-gentz.com/download)

The information in this catalogue is valid at the date of printing. The contained product information serves as a guideline. Subject to technical modifications. All rights, particularly the distribution, copy and translation of this manual or any part of it, in paper or as electronic document, must be authorised in writing by FIOR & GENTZ Gesellschaft für Entwicklung und Vertrieb von orthopädiotechnischen Systemen mbH. Reprints, copies and any other electronic reproduction, even partial, must be authorised in writing by FIOR & GENTZ Gesellschaft für Entwicklung und Vertrieb von orthopädiotechnischen Systemen mbH.





KT2700-GB-05/2016

**FIOR & GENTZ**

Gesellschaft für Entwicklung und Vertrieb  
von orthopädietechnischen Systemen mbH

Dorette-von-Stern-Straße 5  
21337 Lüneburg (Germany)

+49 4131 24445-0  
+49 4131 24445-57

info@fior-gentz.de  
www.fior-gentz.com

**FIOR & GENTZ**  
ORTHOPÄDIETECHNIK MIT SYSTEM