


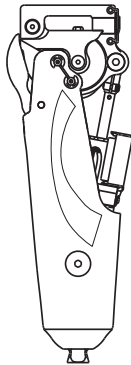

User Instructions

Fine tuning the Smart IP is recommended when a new cosmesis is fitted, for significant changes in footwear weight and for a major change in activity.



Please make yourself familiar with the battery failure advice given on the back of these instructions.

ENDOLITE

Battery failure

When the batteries begin to fail you will receive a warning in the form of ten short beeps from the limb. The limb will still be usable, and you should seek to replace the batteries at the next convenient opportunity.

The limb will still be usable, and you should seek to replace the batteries at the next convenient opportunity.

Upon complete battery failure, the limb will emit one long beep. The limb is still safe to use, as a medium setting will have been selected, though speed changes will not now occur. Seek attention from your practitioner to replace the batteries at the earliest opportunity. Original program settings will not be lost, normal limb function will be resumed upon installation of new batteries.



Safety Notes:-

1. Any changes in performance should be reported to the practitioner.
2. If the device becomes immersed in liquid of any kind or exposed to severe damp, check for function and if necessary report to practitioner immediately.
3. Not to expose the batteries to a naked flame.
4. Dis-assembly and repair of Smart IP must only be carried out by authorised personnel.
5. Avoid strong magnetic fields, sources of electrical interference, atmospheres containing powders and liquids and extremes of temperature.
6. Contact Blatchford Products Limited for advice if limb is to be fitted to an amputee using a pacemaker, hearing aid or any other electronic device.

In normal service an Endolite limb should be inspected annually to check for correct function, lubrication and security of assembly.



ENDOLITE


Blatchford Products Ltd

Lister Road, Basingstoke, Hampshire, RG22 4AH, United Kingdom
Telephone: +44 (0)1256 316600 Facsimile: +44 (0)1256 479705
Email: sales@blatchford.co.uk www.blatchford.co.uk
Blatchford Products Limited and ENDOLITE are companies and trademarks of Chas. A. Blatchford and Sons Ltd.

938249/01-1006

Fine Tuning

START

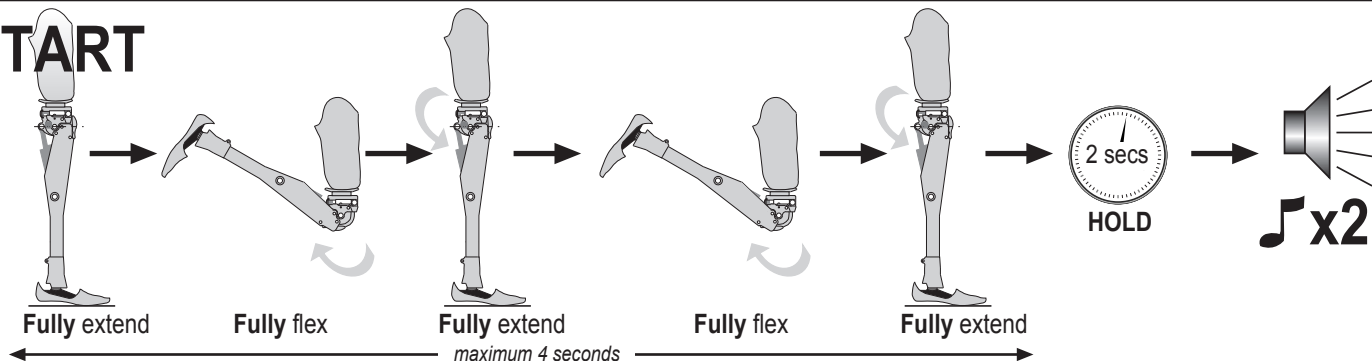

Programming



— vary speed gradually for approx. 200 steps —

refrain from sitting during programming

