

endolite

Elite Blade

EB24L1 - EB30R9

EB24L1D - EB30R9D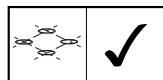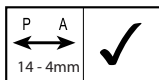

endolite

Available with separately supplied alignment devices

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1 Description and Purpose

Application:

These instructions are for use by the Practitioner.

The elite blade is to be used exclusively as part of a lower limb prosthesis.

A lightweight, high energy return foot, ideal for high impact use from the casual jogger to the serious runner. The flexible extended pylon reduces shock load transmission making it suitable for work, leisure and sports activities. Independent heel and toe springs provide axial deflection. The split toe provides good ground compliance.

This device is recommended for users that have the potential to achieve Activity Level 3 or 4. Of course there are exceptions and in our recommendation we want to allow for unique, individual circumstances and any such decision should be made with sound and thorough justification.

Appropriate footwear must be worn to avoid the risk of slipping in wet environments

Contra-indications:

This device may not be suitable for Activity Level 1 individuals or for competitive sports events, as these types of users will be better served by a specially designed prosthesis optimized for their needs.

Intended for single user.

Ensure that the user has understood all instructions for use, drawing particular attention to the section regarding Maintenance.

Spring Set Selection

Impact	Activity	User Weight										Foot spring set
		44-52 (100-115)	53-59 (116-130)	60-68 (131-150)	69-77 (151-170)	78-88 (171-195)	89-100 (196-220)	101-116 (221-255)	117-130 (256-285)	131-147 (286-325)	148-166 (326-365)	
Low	3	1	1	2	3	4	5	6	7	8	9	
Mod	4	1	2	3	4	5	6	7	8	9		
High	4	2	3	4	5	6	7	8	9			

Low Daily walking and occasional sports such as golf and hiking

Moderate Aggressive walking, frequent or daily sports such as jogging

High Daily activities such as distance running, climbing, lifting and carrying heavy objects for vocational purposes

Important: For higher impact users, do not exceed the weight limit for individual springs.

Activity Level 3

Has the ability or potential for ambulation with variable cadence. Typical of the community ambulator who has the ability to traverse most environmental barriers and may have vocational, therapeutic, or exercise activity that demands prosthetic utilization beyond simple locomotion.

Note:

If in doubt choosing between two categories, choose the higher rate spring set.

Foot Spring set recommendations shown are for transtibial users.

For transfemoral users we suggest selecting a spring set one category lower, refer to fitting advice Section 9 to ensure satisfactory function and range of movement

Order Example:

EB

25L

5

e.g. EB25L5

Size Spring Set
Side Category

Available from size 24 to size 30:

EB24L1 to EB30R9


EB24L1D to EB30R9D

(add 'D' for a dark tone foot shell)

2 Construction

Principal parts:

- Carrier (Aluminium)
- Heel & Toe Springs (e-Carbon)
- Spring attachment screws (Titanium)
- Glide Sock (UHM PE)
- Foot shell (PU)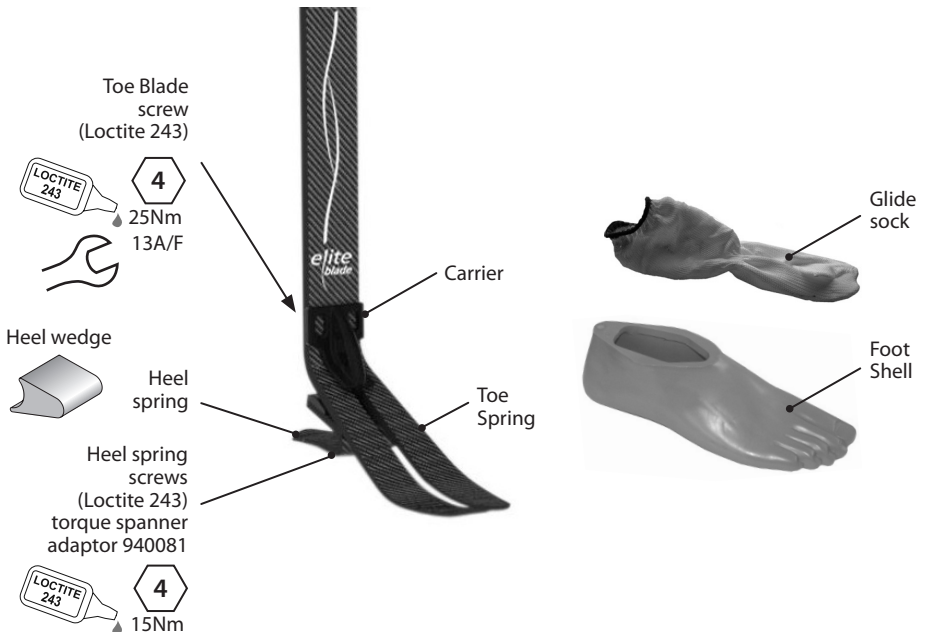

3 Function

The elite blade comprises an e-carbon toe spring pylon and independent heel spring. The pylon can be cut to length to suit individual users.

Heel and toe springs are attached to the carrier assy using titanium screws. The foot is wrapped in a UHM PE sock which is in turn surrounded by a PU foot shell.

4 Maintenance

Maintenance must be carried out by competent personnel.

It is recommended that the following maintenance is carried out annually:

- Remove the foot shell and glide sock, check for damage or wear and replace if necessary.
- Check all screws for tightness, clean and reassemble as necessary.
- Visually check heel spring and toe blade for signs of delamination or wear and replace if necessary. Some minor surface damage may occur after a period of use, this does not affect the function or strength of the foot.

The wearer must be handed the user information card supplied, and be advised of the following:

- Any changes in performance of this device must be reported to the practitioner e.g. reduced energy return or unusual noises.
- The practitioner must also be informed of any changes in body weight and/or activity level.

If this product is used for extreme activity, the maintenance level and interval should be reviewed and if required advice and technical support sought to plan a new maintenance schedule dependent upon the frequency and nature of the activity. This should be determined by a local risk assessment carried out by a suitably qualified individual.

Cleaning:

Use a damp cloth and mild soap to clean outside surfaces, do not use aggressive cleansers.

5 Limitations on use:

Intended life:

A local risk assessment should be carried out based upon activity and usage.

Lifting loads:

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

Environment:

The product is water proof to a maximum depth of 1 metre.

Thoroughly rinse with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts.


Thoroughly rinse with fresh water after use in salt or chlorinated water.

Foot products must be adequately finished to prevent water ingress into the foot shell where possible. If water enters the foot shell, the limb should be inverted and dried before further use.

Exclusively for use between -15°C and 50°C (5°F to 122°F).

It is recommended that only Endolite products be used in conjunction with the elite blade.

6 Bench Alignment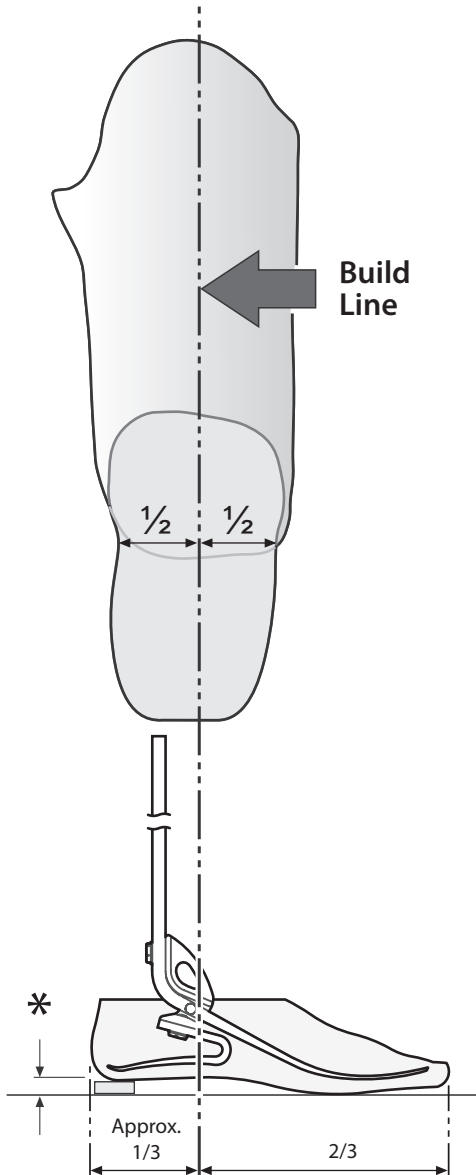

Static alignment

Set Up Length

Note: Shin blade should be vertical when used with a 10mm heel height shoe. Lower heel height shoes may require the use of a packing piece below the heel.

Cut shin to approximate length, allowing 10-15mm for final trimming. Note trial length will typically include 5mm to allow for axial compression of heel and toe.

Build Line

This should fall $\frac{1}{3}$ of the foot length from the heel.

Dynamic alignment

Coronal Plane

Ensure that M-L thrust is minimal by adjusting relative positions of socket and foot.

Sagittal Plane

Check for smooth transition from heel strike to toe off. Ensure also that when standing the heel and toe are evenly loaded and that both are touching the floor.

Trans-femoral Alignment

Align trans-femoral devices according to fitting instructions supplied with the knee, keeping the build line relative to the elite blade as shown.

***** Allow for users own footwear


7 Cutting Shin to length

Note: the following instructions assume the use of the male, sliding, rotating pyramid supplied with the elite blade. If an alternative adaptor is required, this should be taken into account throughout the build process.

Use appropriate health and safety equipment at all times including extraction facilities

1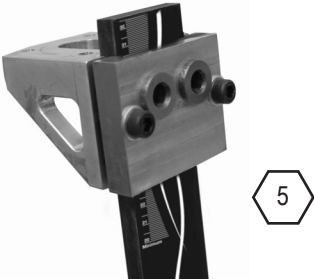

5

Cut the blade 10-15mm longer than required and assemble fitting clamp (941255).

2

4
15Nm

Fit socket to fitting clamp top housing and tighten pyramid screws (15Nm). Refer to alignment page for initial set up.

3

5
15Nm


Tighten fitting clamp bolts to 15Nm (check each screw at least twice).

4

Stand user, carry out dynamic alignment and walking trials. Adjust height to achieve definitive length with weight evenly distributed.

5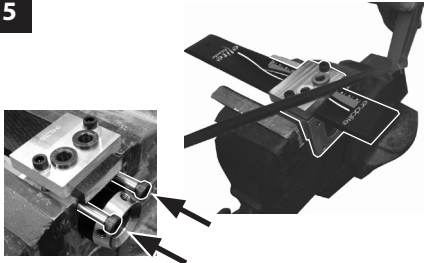

5

Remove socket, place fitting clamp in the vice and cut blade to length. Fit shoulder screws to prevent damage to components


6

Drill two 8.1 diameter holes through drill bushes.

7 Cutting Shin to length *(continued)*

7

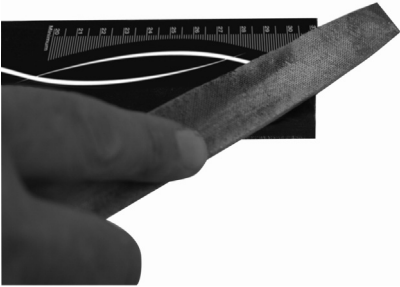Deburr end of blade.

8

Slide pyramid into top housing (or fit preferred alignment option).

9

Loctite (243) & tighten (25Nm) top housing bolts. Attach and align socket.



10

If a foam cosmesis is to be fitted, roughen the top surface of the foot shell to provide a bonding surface.

Bond foam cosmesis to upper surface of the foot shell using Thixofix adhesive (926204) or equivalent and shape to suit.

8 Foot shell removal & blade/spring replacement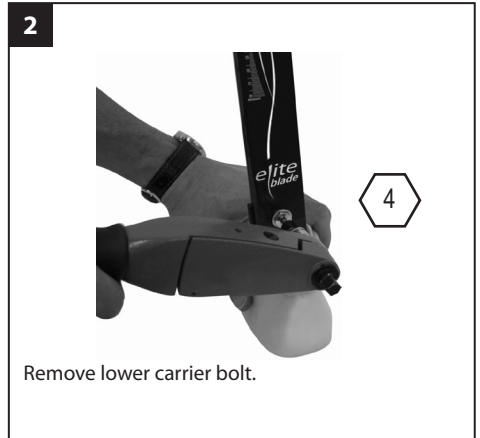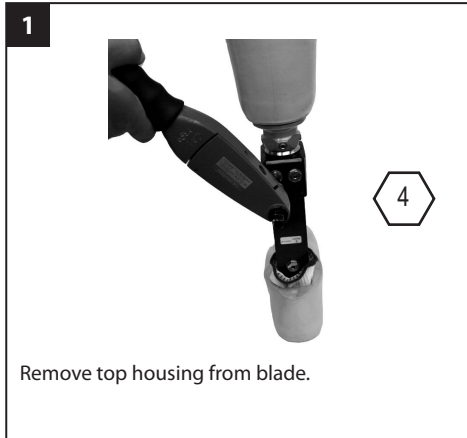

8 Foot shell removal & blade/spring replacement (cont.)

7

Attach blade to heel carrier. Loctite (243) & tighten bolt (25Nm).

8


Fit glide sock and lubricate footshell with 928017.

9


Fit glide sock and lubricate footshell with 928017. Fit assembly into footshell, engage toe of blade into footshell.

10

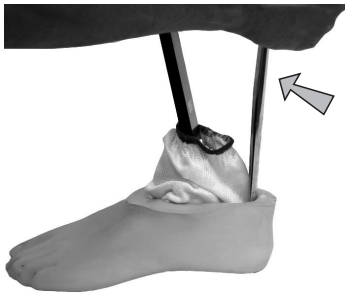Using a lever, ease assembly into shell.

11

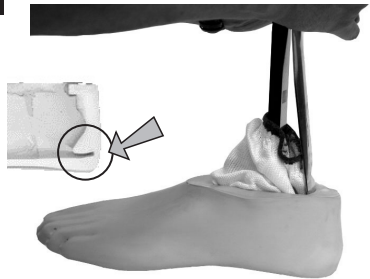Ensure heel spring is engaged into slot.

12

Cut blade using method shown previously before attaching top housing. Loctite (243) & tighten bolts (25Nm). Re-apply cosmesis if applicable.

9 Fitting Advice

Elite blade springs are supplied as matched sets i.e. the blade/toe and heel springs are designed to work together to give smooth progression for most users.

Heel Wedge

A heel wedge is supplied with the foot. Fitting the wedge will have the effect of stiffening the heel spring. These can be taped in place for trial. For permanent fitting, wedges should be adhered in place by application of Loctite 424 (926104) between the lower contacting surface of the heel and the wedge.

	<i>Symptoms</i>	<i>Remedy</i>
Heel too soft	<ul style="list-style-type: none">• Sinking at heel strike• Difficulty climbing over the toe (toe feels too hard)	<ol style="list-style-type: none">1 Move socket anteriorly in relation to the foot (excess movement may result in drop off)2 Add heel wedge
Heel too hard	<ul style="list-style-type: none">• Rapid transition from heel strike through stance phase• Difficulty in controlling heel action, foot jars into mid-stance• Foot feels too rigid	<ol style="list-style-type: none">1 Remove heel wedge (if fitted)2 Move socket posteriorly in relation to foot
Toe/ Blade too soft	<ul style="list-style-type: none">• Rapid progression through mid-stance.• 'Drop off' at higher activity levels	<ol style="list-style-type: none">1 Move socket posteriorly in relation to foot2 Plantarflex foot slightly - note: some re-alignment may be required


Please contact your supplier if it is not possible to achieve a smooth gait after following the advice above.

10 Technical Data

Operating and Storage Temperature Range:	-15°C to 50°C (5°F to 122°F)
Component Weight [size 26]:	580g (1lb 5ozs)
Recommended Activity Level:	3 - 4
Maximum User Weight:	166kg (365lb)
Proximal Alignment attachment:	Male Pyramid (Endolite)

Build Height:	200-365mm
[See diagram below]	
Heel Height	10mm

Fitting length

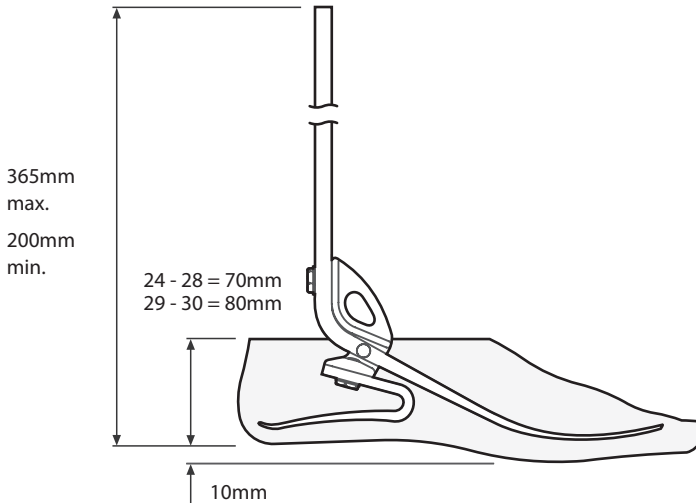11 Ordering Information

Spring Kits			
Rate	Foot sizes		
	24-26	27-28	29-30
Set 1	539610	539619	special order
Set 2	539611	539620	special order
Set 3	539612	539621	539630
Set 4	539613	539622	539631
Set 5	539614	539623	539632
Set 6	539615	539624	539633
Set 7	539616	539625	539634
Set 8	539617	539626	539635
Set 9	539618	539627	539636

Footshell

For dark add suffix D

Left	
24L	539005
25L	539007
26L	539009
27L	539011
28L	539013
29L	539015
30L	539017

Right	
24R	539006
25R	539008
26R	539010
27R	539012
28R	539014
29R	539016
30R	539018

Glide Sock

One size fits all Part No. 532811

Fitting Clamp Part No. 941255

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty

The elite blade is warranted for 36months - foot shell 12 months - glide sock 3 months. from original date of purchase unless otherwise stated.

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Endolite Deutschland GmbH
endolite India Ltd