endolite

get busy living

Elite Blade/Javelin Foot Shell Cover

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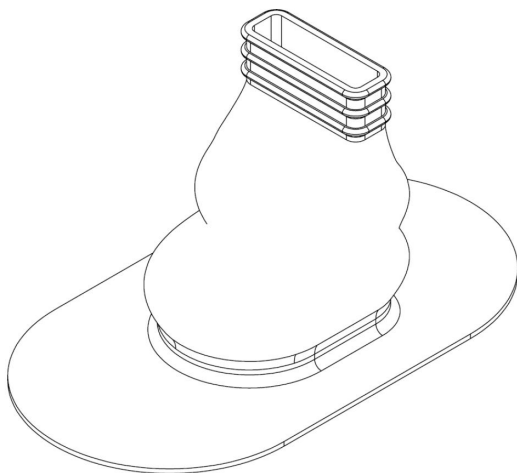
# 1 Description and purpose

## Application:

These instructions are for use by the Clinician/Practitioner.

The cover is designed for use with the Elite Blade and Javelin products.

If fitted according to the following instructions, the cover provides protection from the ingress of debris into the foot shell.

If water, or any other liquid enters the foot, it may be necessary to create a gap between the cover and the Blade at its top edge before inverting the foot and allowing it to dry. In the case of use with the Javelin, the interface plate may be removed.

Intended for single user.

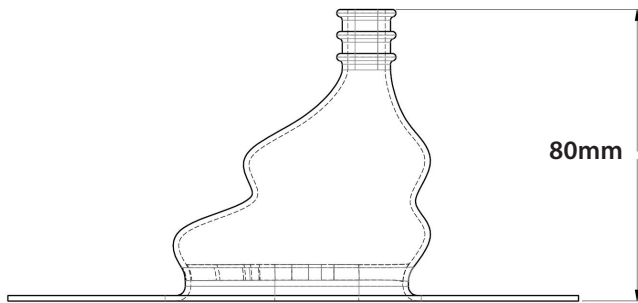
# 2 Maintenance

## Cleaning:

Use a damp cloth and mild soap to clean outside surfaces, DO NOT use aggressive cleansers.

# 3 Technical Data

Material	Neoprene
Component Weight (prior to trimming)	47g (1.5ozs)
Operating and Storage Temperature Range	-15°C to 50°C (5°F to 122°F)

Fitting Length

## 4 Assembly Instructions

For use if cover is to be adhered to the foot shell (Elite Blade) or interface plate (Javelin).

- 1** Roughen the top surface of the foot shell or interface plate (with it fitted to the foot shell).

- 2** Fit Blade into foot shell and apply Loctite 424 (926104) between the underside of the cover and top surface of the footshell or interface plate.

- 3** After pressing the two surfaces together until the adhesive has cured, trim excess material following the foot shell or interface plate edge.

- 4** Grind edges flush with foot shell or interface plate to finish.

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Endolite Deutschland GmbH
### **endolite India Ltd**