

mini
blade
XT



WATERPROOF

Keep up the pace
with the Mini BladeXT!



endolite



“ I can stand still, I can walk, I can do trampolining. I love the bright colour. My Mini BladeXT's are the whole package! ” Izzy

Join in the Fun!

For youngsters, life never stops. An active lifestyle needs to be fully equipped to meet the needs of the most demanding user, whether it's going to school, spending time with friends, running, cycling or taking part in other sports.

Designed to offer a complete solution, the Mini BladeXT offers outstanding stability, ground compliance and improved comfort through the split toe and traction sole, while its C-shaped toe spring is primed for optimal energy response.

Available in dazzling pink or electric blue, the Mini BladeXT is the ideal choice for children and may be the only foot they need!

mini
blade
XT





Optimal energy response

The curved toe design works like a loaded spring to assist forward progression. From initial ground strike, the C-shaped curve compresses and stores energy, ready to release upon the next step.

Protection and stability

With a unique toe and heel design, users feel balanced and secure when standing, running or making a change of direction. This also ensures shock absorption for comfort and protection on every step.

Safety and compliance

An integral traction sole helps to provide safety and surefootedness for all activities. The split toe improves ground compliance, provides flexibility and enhances comfort on uneven ground.

Bright, colourful finish

Available in dazzling pink and electric blue, the Mini BladeXT is attractive and fun for younger users.

Waterproof

Been somewhere muddy? Simply rinse the Mini BladeXT with fresh water after use in abrasive environments and after exposure to salt or chlorinated water.

Lightweight and strong

The lightweight construction won't slow you down. Made from e-carbon, the Mini BladeXT is strong and tough, yet flexible and responsive.



“ I feel more confident. I can run faster and do more sports with my brothers. ” Cody



Natural motion

As the wearer transfers their weight onto the Mini BladeXT, the heel spring compresses, absorbing shock for greater comfort and stability during early stance. This ensures a smooth roll-over phase and seamless loading of the e-carbon toe spring. In late stance, the stored energy is released in a forward direction to spring into the next step. The net result is a more balanced, symmetrical and energy efficient gait.

Features

- Twin toe and heel springs
- Split toe design
- Traction sole
- Heel stiffness wedge
- Waterproof
- Field replaceable heel spring kit
- Male mini pyramid with 30mm shift and rotation



Fitting

Available in soft, medium and firm spring ratings, the Mini BladeXT is suitable for users weighing up to 60kg. The A/P slide pyramid connection allows for customised fitting and adjustment. A heel wedge can be added if extra heel stiffness is required. Available with 25mm or 30mm female mini pyramid adapters, the Mini BladeXT can be tailored to the chosen system componentry.

Specification

User Weight:	20-60kg
Amputation Level:	All
Activity Level:	3-4
Spring Range:	1-3
Component Weight:	450g (including male mini pyramid)
Build Height:	160mm
Colour Options:	Dazzling Pink (P) or Electric Blue (B)
Warranty:	12 months (excludes paintwork and sole cover)

Order Example

MBXTA B 3 SM Foot example: Mini BladeXT, Colour Blue, Spring Set 3

Activity	User Weight			kg
	20-35	36-50	51-60	
	44-77	78-110	111-132	lbs
4	1	2	3	Toe and Heel Spring Set
	Soft	Medium	Firm	



Adapters

Male Mini Pyramid (supplied with foot): 189627

Female Mini Pyramid 25mm
Tube Clamp Adapter: 189526




Female Mini Pyramid 30mm
Tube Clamp Adapter: 189727



Replacement Heel Spring Kits

- 1 (soft) – 539084SS
- 2 (medium) – 539084MS
- 3 (firm) – 539084FS

Build Height

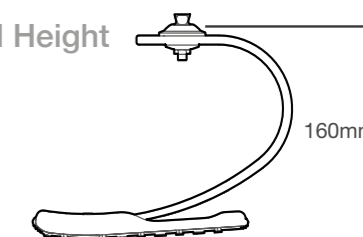

And for the big kids...

The fun doesn't stop once you've outgrown the Mini BladeXT. The adult BladeXT is the next step on the journey, available in 9 spring ratings and suitable for users up to 166kg.



Follow Blatchford:

 @BlatchfordGrp

 /BlatchfordGrp

blatchford.co.uk

0114 263 7900

sales@blatchford.co.uk



Endolite is part of the Blatchford Group. Registered address, Chas A Blatchford & Sons Ltd, Unit D, Antura, Kingsland Business Park, Basingstoke, RG24 8PZ, United Kingdom. t: +44 (0) 1256 316600 f: +44 (0) 1256 316710



An annual visual inspection is recommended. Check for visual defects that may affect proper function. Maintenance must be carried out by competent personnel.