EN	User Guide	1
FR	Guide de l'Utilisateur	10
DE	Bedienungsanleitung für Anwender	19
IT	Guida per il Paziente	28
ES	Guia de Usuario	37
NO	Brukerhåndbok	46
RU	Руководство пользователя	55
TR	Kullanım Kılavuzu	64

ORION³

endolite

- 1 Introduction
 - Safety Information
- 2 Key Points
- 3 Transport and Storage
- 4 Battery Charging
- 5 User Modes


Battery Safety

The Orion3 contains Li-ion type rechargeable batteries. Please read and comply with the safety information see Section 4. Failure to do so may cause the battery to become hot, explode, or ignite and cause serious injury.

IMPORTANT: Battery Charging

To charge the Orion3 ensure that the On/Off Switch is in the ON position. When the Orion3 is charging the Battery Charger LED will be flashing green.

The battery should only be charged within the temperature range 10°C to 40°C (50°F to 104°F).

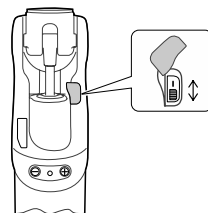Battery On/Off Switch

Low Temperature Operation

IMPORTANT:

If the Orion3 is subjected to temperatures lower than -10°C (14°F) it must be returned to be inspected for possible damage to the battery pack.

Following extended periods of inactivity at temperatures below 0°C (32°F) the knee may swing less freely. In such an eventuality, it is recommended to exercise caution while descending stairs and avoid descending stairs leg-over-leg using the knee yield until you feel the swing returns to normal.

Package contents:

- 1 Knee prosthesis
- 2 User Guide
- 3 Cable Bag and accessories
 - i) Battery Charger
 - ii) Battery Charger AC power socket outlet adapter(s)

1 Introduction

The Orion3 is a microprocessor controlled knee prosthesis that provides swing and stance control and adapts to variable cadence. It has a programmable stance yield setting for secure stance control on stairs, ramps and during active gait with a cadence range from slow walking to fast walking. The knee also provides an additional standing support mode for enhanced stability. The set-up procedure is through an easy to use PC interface via a Bluetooth® connection to the on-board module.

Application:

The Orion3 is to be used exclusively as part of a lower limb prosthesis.

Intended for single user.

Please read all these instructions for use, paying particular attention to the sections regarding safety and maintenance.

Features:

- Microprocessor controlled stance and swing
- Standing support mode that controls knee flexion for standing
- Software based set-up via a built-in Bluetooth® link
- User modes: cycling and flexion lock mode
- Up to 3 days of normal use from the internal rechargeable Li-ion battery pack

In addition the Orion3 offers enhanced stability:

- Dynamic stair descent with easy initiation of knee flexion with customisable *yield rate* (resistance level increases with knee flexion angle)
- Supportive resistance to flexion as soon as the knee stops flexing
- Increased supportive resistance to flexion to prevent 'stumbles' associated with instability at heel strike
- Supportive yield for sitting down

Safety Information

The caution symbol highlights safety information which must be followed carefully.

Any changes in the performance of the knee e.g. restricted movement, non-smooth motion or unusual noises should be immediately reported to your service provider.

Always use a hand rail when descending stairs and at any other time if available.

Any excessive changes in heel height after programming will adversely affect knee function and should be immediately reported to your service provider for reprogramming and calibration.

After continuous use the knee casing may become hot to the touch.

Avoid strong magnetic fields, sources of electrical interference, atmospheres containing liquids and/or powders.

Do not place near any heat source. Do not leave in direct sunshine or inside a car in hot weather.

The Orion3 is not intended for use when immersed in water or as a shower prosthesis. If the knee comes into contact with water turn it off and wipe dry immediately.

Ensure that no water enters the charger plug socket and that the socket cover is in place at all times.

Never plug the charger into the Orion3 whilst still wearing the knee.

Be aware of finger trap hazard at all times.

The Orion3 is not suitable for extreme sports, running or cycle racing, ice and snow sports, extreme slopes and steps. Any such activities undertaken are done so completely at the users' own risk. Recreational cycling is acceptable.

Contact your service provider for advice if the Orion3 is to be used with a pacemaker or any other electronic medical device.

Assembly, programming and repair of the Orion3 must only be carried out by a suitably qualified clinician, certified by Endolite.

The user must not adjust or tamper with the setup of the Orion3.

The user should be advised to contact their practitioner if their condition changes.

The Orion3 detects standing and walking at various speeds. Non-identified movements may cause unexpected behaviour.

Only charge the batteries in the Orion3 using the charger supplied with the product. Do not use this charger for charging any other device.

Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.

2 Key points

The Orion3 **must not** be used with large variations in heel height.
Do not attempt stairs descent leg-over-leg on a Orion3 knee unless you have been trained for stairs descent.

Any changes in body weight and/or activity level of the user must also be reported to the practitioner.

High flexion resistance settings may induce some noise during stairs descent, this will not affect function.

Any changes in performance of this device must be reported to the practitioner.

Changes in performance may include:

- Increase or decrease in knee stiffness
- Instability
- Reduced knee support (free movement)
- Any unusual noise
- Any unrecognized beeps or warning lights

Cleaning:

Use a damp cloth and mild soap to clean outside surfaces, DO NOT use aggressive cleansers. Ensure water/liquids do not enter the charging socket. Thoroughly dry before use.

Environment:

Avoid exposing the Orion3 to corrosive elements such as water, acids and other liquids. Also avoid abrasive environments such as those containing sand for example as these may promote premature wear.

Suitable for outdoor use

Exclusively for use between -10°C and 50°C (14°F to 122°F).

3 Transport and storage

- 1 Ensure that the Orion3 is in a straight (fully extended) position when not in use.
- 2 If the limb is to be stored upright then it must be secured to prevent it falling over or flexing.
- 3 If the Orion3 is shipped either on its own or built into a limb it should be maintained in an extended position and packaged in a suitable box to allow this.
- 4 If a unit is to be returned, contact Endolite prior to sending if there is any sign of physical damage to the battery pack.
- 5 For transport purposes the unit must be switched off, and packed securely so that a) it cannot be inadvertently switched on and become operative while in transit; b) the unit is thoroughly protected from damage in transit.

4 Battery Charging


The Orion3 contains Li-ion type rechargeable batteries. Please read and comply with the safety information. Failure to do so may cause the battery to become hot, explode, or ignite and cause serious injury.

Please advise the user of the charging information in this section. The batteries are not field replaceable and will be replaced as necessary when serviced.

If the Orion3 is subjected to temperatures lower than -10°C (14°F) it must be returned to be inspected for possible damage to the battery pack.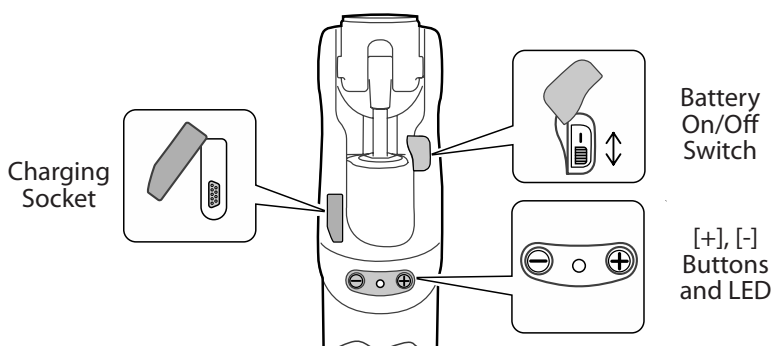

1. The battery should only be charged in temperatures between 10°C to 40°C (50°F to 104°F).

2. Only charge the batteries in the Orion3 using the charger supplied with the product.

Do not use this charger for charging any other device.

3. The battery charger can be used with an electrical outlet that is rated between 100V and 240V AC, 50/60Hz.

4. Never plug the charger into the Orion3 while still wearing the knee.

5. During charging no power is delivered to the Orion3 control system.

6. The knee module may become warm during charging but should never be too hot to touch. There should be no unusual smell, if there is immediately switch **OFF** the battery, unplug the charger from the electrical supply and contact your service provider.

7. Do not refit the knee until the charger has been disconnected.


8. Always recharge the Orion3 before use if it has not been used for an extended period of time.

Battery Charging (continued)

We recommend that the Orion3 knee should be fully charged every night. It should take up to 8 hours to charge fully. We also recommend battery charging whenever convenient. Beware; misalignment while plugging in may cause damage to the connector.

Step by Step Guide:

1. Lay the knee on a flat surface.
2. Switch the Battery On/Off switch to ON.
3. Connect the charger plug to the charging socket on the posterior part of the knee module.
4. Check the Battery Charger LED is flashing green to confirm the battery is charging.
5. Refer to table below to ensure the knee is charging.
6. Disconnect from the charger when charged sufficiently.
7. Wait for 30 seconds while observing the Orion3 LED changing color as indicated:
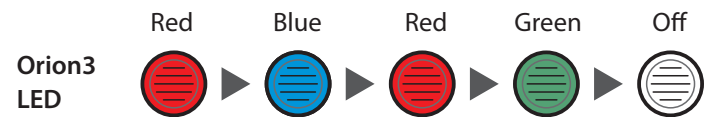


8. After a successful reset sequence 2 short beeps will be heard.
9. If the reset sequence is unsuccessful or the LED remains Red switch the battery Off then On and repeat from step 7.

Battery Charger LED	Charging state
Yellow	Standby
Green > fast flashing	Fast charging
Green > slow flashing	Close to full charge
Green	Maintain (trickle charge)
Yellow > fast flashing	Error (Ensure Battery Switch is ON)

NOTE:


If the battery charger LED is yellow and the knee is making a clicking noise check that the Battery switch is in the ON position.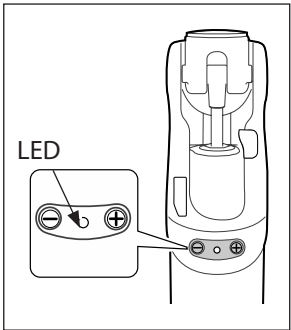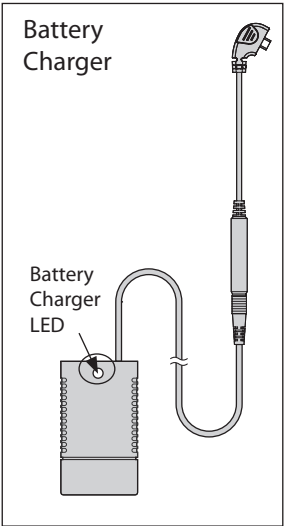

The user can enable/disable the audible reset confirmation by pressing and holding the [+] and [-] buttons simultaneously for 2 seconds. The change will be confirmed by 2 short beeps.

Battery Status:

To check the battery condition press and *hold* the [-] button on the knee, one short beep will be heard and the LED will indicate the battery condition.

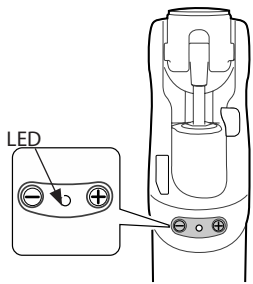Battery Charge Indicator			
LED		Beeps	Battery Condition
Green	3 flashes	--	fully charged
Green	2 flashes	--	good
Green	1 flash	--	ok
Yellow	slow flash	8 x Short beeps	low charge
Red	slow flash	5 x Long beeps	no charge

When fully charged the Orion3 battery is intended to provide 3 days of normal use (conditional on the type of use). We recommend that the knee be fully charged every night. Switching off the Orion3 when not in use will also prolong the period of use (Note: The Orion3 will take 30 seconds to reset after it is switched back ON refer to Battery Charging for the reset sequence).

5 User Modes

The Orion3 has 2 User Modes. Please note that these modes are only available if they have been enabled by your practitioner.

Warning:

Care must be taken to only use those modes as instructed by your practitioner.

1 Flexion Lock

Sets the limit of flexion to a predetermined angle.

The maximum lock angle is 45 degrees from fully extended.

2 Cycling (Free swing)

Releases stance resistance, the Orion3 knee will then swing freely.

Warning:

In Cycling Mode there is no stance control active, this mode must be de-activated before resuming walking to avoid falling and possible injury.

Flexion Lock Mode

User activation of Flexion Lock Mode with $\ominus$ button on the Orion3 knee

To activate Flexion Lock:

Fully extend the knee.

Press the $\ominus$ button 3 times, a short beep confirms each button press, 3 short beeps confirm that Flexion Lock is ON.

To de-activate Flexion Lock:

Press and hold $\ominus$ you will hear 2 long beeps.

Cycling Mode

Warning:

In cycling (free swing) mode there is NO stance resistance.

User activation of Cycling Mode with $\ominus$ button on the Orion3 knee

To activate Cycling Mode:

Press the $\ominus$ button 4 times, a short beep confirms each button press, 4 short beeps confirm that Cycling Mode is activated.

To de-activate Cycling Mode:

Press and hold $\ominus$ you will hear 2 long beeps.

The user should be aware that changes or modifications not expressly approved could void the operating licences and exemptions.

This device complies with part 15 of the FCC Rules.

Operation is subject to the following three conditions:-

- 1) This device may not cause harmful interference.
- 2) This device must accept any interference received, Including interference that may cause undesired operation.
- 3) Contains pre-approved Bluetooth module:

FCC ID: PVH0946 IC: 5325A-0946

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty

The warranty for the Orion3 knee is 36 months.

A scheduled service is required every 20 months.

See Endolite website for current full warranty statement.

Extended additional warranty for the Orion3 is available on request contact service provider.

The user should be aware that changes or modifications not expressly approved will void the warranty.

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