

# OP-FULLCF-LP

## Offset Plate

### Product Information



### Description:

The OP-FULLCF-LP is constructed of 1/4" (6.35mm) carbon fiber, is rated to 265 lbs normal gait activity, weighs only 1.69 ounces (48g), has a hole for plunger pin protrusion, and can interface between most socket and lower extremity endoskeletal systems that utilize a standard European 4-hole pattern. The OP-FULLCF-LP is shipped with (2) each M6X10, M6X25 and M6X30 bolts.

### Instructions:

Determine desired orientation of OP-FULLCF-LP on distal end of socket (A/P or M/L). USE LOCTITE 242 ON EACH BOLT. Connecting component bolt to OP-FULLCF-LP by threading M6X10 bolts through the connecting component into threaded holes on the OP-FULLCF-LP. Position lock or attachment plate inside of the socket. Position OP-FULLCF-LP on distal end of socket and thread the M6X30 bolts through the connector and OP-FULLCF-LP and into the lock threads. Thread M6X25 bolts through OP-FULLCF-LP and into remaining lock threads. Tighten and torque all bolts to 12Nm.

### Specifications:

<b>Added Height:</b>	<b>6.35mm (1/4")</b>
<b>Weight:</b>	<b>48g (1.69 oz)</b>
<b>Load Rating:</b>	
Moderate Activity	265 lbs. (120 Kg)
Low Activity	325 lbs. (147 Kg)
<b>Suggested L-Code</b>	<b>L5910 - BK / L5920 - AK</b>

**Warranty Information:** PDI warrants its components to purchasers of the prosthetic components only. No warranties are made to consumers. All components are warranted for 30 days from original purchase date. This warranty applies only to manufacturing defects and does not cover: (1) any damage due to misuse, altering of the component, improper service or accidents; (2) any secondary costs that might be incurred in the exchange of components; (3) wear, normal, or due to incorrect selection of components. During the warranty period, defective parts will be replaced or repaired at no charge. Missing bolts will be replaced at retail prices. This warranty is in lieu of all other warranties, expressed or implied, including any implied warranty of merchantability or fitness for particular purpose. The remedies provided above are exclusive; in no event shall Prosthetic Design, Inc. be liable for any special, indirect, incidental, or consequential damages.

**Returns:** A restocking fee totaling 2% of retail price will be charged to customer should the customer wish to return any PDI product for credit. The part must be unused, and in its original, unopened packaging. In addition, the return must be initiated within 30 days of its purchase in order to qualify for the credit. If returned after 30 days of purchase date a 15% restocking fee will be charged.



Prosthetic Design, Inc., 700 Harco Drive, Clayton, OH 45315 USA [www.prostheticdesign.com](http://www.prostheticdesign.com)  
Telephone 800-459-0177 International Telephone 937-836-1464 Fax 937-832-5361

