



### SPECIFICATIONS

Part No.	Alloy	Adapter Style	Product Height	Product Weight	Max Patient Weight	Torque
3T54	Titanium	Receiver/Receiver	34 mm	114 gr.	136 Kg. / 300 lbs.	15 Nm
3T55	Titanium	Receiver/Receiver	42 mm	129 gr.	136 Kg. / 300 lbs.	15 Nm
3T56	Titanium	Receiver/Receiver	53 mm	143 gr.	136 Kg. / 300 lbs.	15 Nm
3T57	Titanium	Receiver/Receiver	66 mm	156 gr.	136 Kg. / 300 lbs.	15 Nm
3T61	Titanium	Pyramid/Receiver	42 mm	117 gr.	136 Kg. / 300 lbs.	15 Nm
3T63	Titanium	Pyramid/Receiver	34 mm	104 gr.	136 Kg. / 300 lbs.	N/A
3A54	Aluminum	Receiver/Receiver	34 mm	119 gr.	100 Kg. / 220 lbs.	15 Nm
3A55	Aluminum	Receiver/Receiver	42 mm	131 gr.	100 Kg. / 220 lbs.	15 Nm
3A56	Aluminum	Receiver/Receiver	53 mm	142 gr.	100 Kg. / 220 lbs.	15 Nm
3A57	Aluminum	Receiver/Receiver	66 mm	148 gr.	100 Kg. / 220 lbs.	15 Nm
3A61	Aluminum	Pyramid/Receiver	42 mm	130 gr.	100 Kg. / 220 lbs.	15 Nm
3A63	Aluminum	Pyramid/Pyramid	34 mm	95 gr.	100 Kg. / 220 lbs.	N/A

### USAGE

This product is designed to be used for lower limb external prosthetic length and alignment corrections. With set screws loosened, rotate to desired position and tighten screws. Before fitting this product onto the patient, all parts must be joined properly and all set screws must be secured enough for the patient to walk. In order to make final adjustments, there should be no loose screws.

### ATTENTION

After the final adjustments, apply single drop of removable Loctite® 242 to each of the set/cap screws and then torque the product. Improper installation or uneven, insufficient or excessive torque may cause bolt or adapter failure. Always use quality metric tools for installation.

### WARRANTY

All MEDEX Modular Adapters are unconditionally guaranteed for 2 years. Cutting, drilling, modifying or using the component other than as described in this manual will void the warranty.