



SPECIFICATIONS

Part No.	Alloy	Adapter Style	Product Weight	Max Patient Weight	Slide Screw Torque	Set Screw Torque
3T80	Titanium	Receiver/Receiver	157 gr.	136 Kg. / 300 lbs.	10 Nm	15 Nm
3T81	Titanium	Receiver/Pyramid	121 gr.	136 Kg. / 300 lbs.	10 Nm	15 Nm
3A80	Aluminum	Receiver/Receiver	157 gr.	100 Kg. / 220 lbs.	10 Nm	15 Nm
3A81	Aluminum	Receiver/Pyramid	126 gr.	100 Kg. / 220 lbs.	10 Nm	15 Nm

USAGE

This product is designed to be used for lower limb external prosthetic length and alignment corrections. Rotate and slide to desired placement, then tighten set screws. Before fitting this product onto the patient, all parts must be joined properly and all set screws must be secured enough for the patient to walk. In order to make final adjustments, there should be no loose screws.

ATTENTION

After the final adjustments, apply single drop of removable Loctite® 242 Blue® to each of the set/cap screws and then torque the product. Improper installation or uneven, insufficient or excessive torque may cause bolt or adapter failure. Always use quality metric tools for installation.

WARRANTY

All MEDEX Modular Adapters are unconditionally guaranteed for 2 years. Cutting, drilling, modifying or using the component other than as described in this manual will void the warranty.