
# Child's 4-Bar Knee

239133

EN	Instructions for Use	1
FR	Instructions D'Utilisation	7
DE	Gebrauchsanweisung	13
IT	Istruzioni per L'Uso	19
ES	Instrucciones de Uso	25
NO	Bruksanvisning	31
RU	Инструкция протезиста	37
TR	Kullanım Talimatları	43

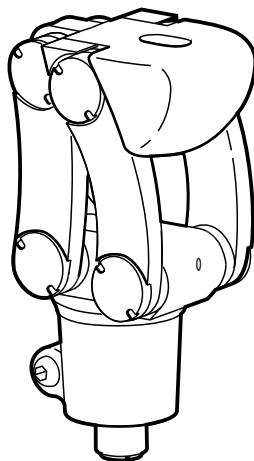endolite

# 1 Description and Purpose

EN

## Application:

These instructions are for use by the practitioner.

The Child's 4 bar knee is to be used exclusively as part of a lower limb prosthesis.

A low profile knee for children with linkage design for excellent range of motion providing increased ground clearance during swing phase.

- Adjustable spring extension assist
- Stable geometry on heel strike
- Lightweight and robust knee
- A range of proximal adapters

This device is suitable for walking and general use

This device is recommended for users that have Activity Level 1-4.

## Contra-indications:

This device is not suitable for use with foot sizes larger than 21 cm.

Intended for single user.

Ensure that the user has understood all instructions for use, drawing particular attention to the Safety Information and section regarding maintenance.


Be aware of finger trap hazard at all times

# 2 Construction

## Principal parts:

- Knee Assembly (Aluminium, Stainless Steel, Polymer, Rubber)
- Spring (Stainless Steel)
- Adjuster (Nylon, Stainless Steel)
- Tube Clamp Screw (Steel)
- Lock Washer (Steel)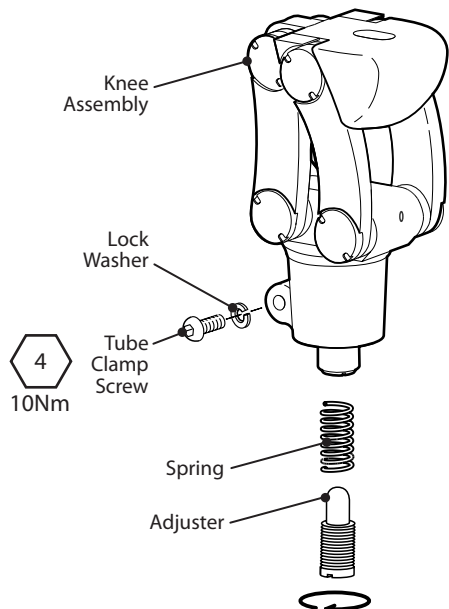

---

### 3 Function

The Endolite Child's 4-bar knee is a durable and lightweight paediatric knee designed for active children. The knee has a polycentric design for stability during stance, comprising pivots connecting a chassis to a main body via 4 link arms. The geometry of the knee provides efficient progression into swing phase at the required point in the gait cycle, increased toe clearance during the swing phase when compared to a single axis knee and short protrusion when sitting.

There is a choice of three extension assist springs with adjustable pre-load to provide swing control. Main bearings in the linkage are both replaceable and adjustable to prolong service life.

---

### 4 Maintenance

Maintenance must be carried out by competent personnel.

It is recommended that the following maintenance is carried out annually:

- Check the Alignment Bolt and Tube Clamp Screw for tightness, tighten to the correct torque setting if required
- Check for defects, including corrosion that could affect proper function

To adjust any sideplay in the 4 bar links see Section 7.

Advise the user to report any changes in the performance of this device to the practitioner, changes may include:

- Loss of alignment
- Unusual noises
- Stiffness
- Looseness/play

The practitioner must also be informed of any changes in body weight and/or activity level.

#### Cleaning

Use a damp cloth and mild soap to clean the outside surfaces.

DO NOT use aggressive cleansers. Thoroughly dry before use.

---

### 5 Limitations on Use:

#### Intended Life:

A local risk assessment should be carried out based upon activity and usage.

#### Lifting Loads:

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

#### Environment:

Avoid exposing the product to corrosive elements such as water, acids and other liquids. Also avoid abrasive environments such as those containing sand for example as these may promote premature wear.


#### Suitable for outdoor use

Exclusively for use between -15°C to 50°C (5°F to 122°F).

It is recommended that only Endolite products be used in conjunction with the product.

## 6 Alignment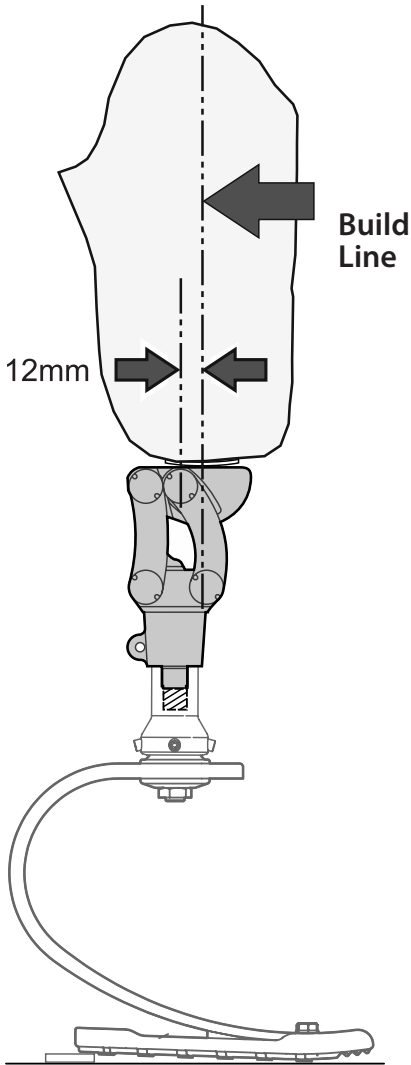

### Static Alignment


#### Build Line

Construct the socket to fully accommodate alignment of residuum so that the Build Line passes through the attachment point. The Build Line should fall in the middle of the slot on top of the Child's Knee, 12mm anterior of upper front pivot. Before walking stand the wearer in suitable walking rails to confirm alignment before progressing to dynamic alignment.

### Dynamic Alignment

#### Adjust the Extension Assist

The adjuster can be used to reduce the preload on the spring to suit the user's requirement. Undo the adjuster to reduce the assistance up to a maximum of 8mm.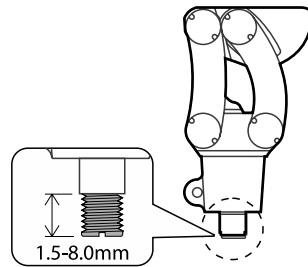

#### To change the spring:

Fully unscrew the adjuster and remove the spring. Insert the appropriate spring and screw the adjuster back in with 1.5mm showing. The adjuster can then be used to adjust the pre-load on the spring to suit the user's individual requirements.




<i>Spring</i>	<i>Spring Force</i>	<i>Extension Assist</i>
Silver	Standard (as fitted)	Medium assistance
Gold	Weaker spring force	Reduced assistance
Black	Stronger spring force	More assistance

# 7 Service

## Sideplay adjustment in the Front and Rear Links

After a period of use it may be necessary to adjust the bearings to reduce side play in the links.


- 1 Loosen the 4 Locking Set Screws **A** to release the Pivot Pins.
- 2 Loosen the 4 Adjuster Cap internal Locking Set Screws **B** inside the Pivot Pins.
- 3 To reduce the play in the Links:  
Using a Peg Spanner (Part No. 940054) in the Adjuster Cap to prevent rotation, use a second Peg Spanner to carefully turn the Pivot Pin clockwise to eliminate unnecessary side play in the links. Check that the links move freely.
- 4 Re-tighten all locking screws, using Loctite 222 on Locking Set Screws **A**.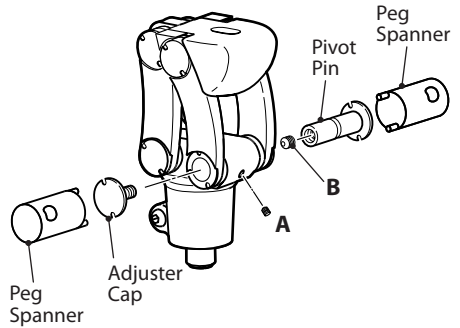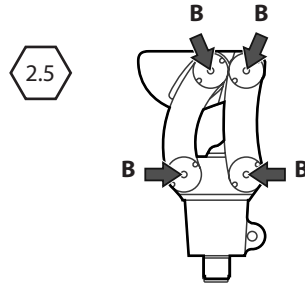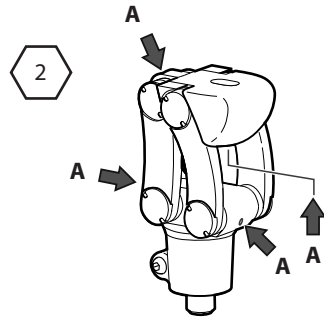

## 8 Technical Data

Operating and Storage Temperature Range:	-15°C to 50°C (5°F to 122°F)
Component Weight:	335g (12oz)
Activity Level:	1-4
Maximum User Weight:	60kg (130lb)
Distal Attachment:	Ø 25mm Full 360° rotation
Proximal Attachment:	Endolite Single Bolt with 7mm A-P shift
Build Height:	105mm
[See diagram below]	

### Fitting Length

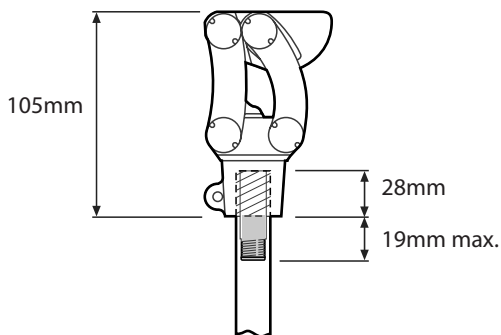---

## 9 Ordering Information

Peg Spanner	Part No. 940054
Bearing Kit	Part No. 339006
Bearing and Links Kit	Part No. 339010
Spring, Silver	Part No. 925014
Spring, Gold	Part No. 925015
Spring, Black	Part No. 925016
Tube Clamp Screw	Part No. 910215

---

### **Liability**

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

### **CE Conformity**

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

### **Warranty**

The Child's 4-bar knee is warranted for 24 months.

See Endolite website for the current full warranty statement.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licences and exemptions.

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Endolite India Ltd
Endolite Deutschland GmbH,