# Esprit

ESP22L1S-ESP30R8S  
ESP22L1SD-ESP30R8SD

---

|    |                            |    |
|----|----------------------------|----|
| EN | Instructions for Use       | 1  |
| FR | Instructions D'Utilisation | 11 |
| DE | Gebrauchsanweisung         | 21 |
| IT | Istruzioni per L'Uso       | 32 |
| ES | Instrucciones de Uso       | 42 |

endolite

# 1 Description and Purpose

## Application

These instructions are for the practitioner.

The term *device* is used throughout these instructions for use to refer to Esprit.

This device is to be used exclusively as part of a lower limb prosthesis.

A moderate-energy-return foot. Independent heel and toe spring provides some axial deflection. The split toe provides good ground compliance.

This device is recommended for users that have the potential to achieve Activity Level 3.

Of course there are exceptions and in our recommendation we want to allow for unique, individual circumstances. There may also be a number of users in the Activity Levels 2 and 4\* who would benefit from this device, but this decision should be made with sound and thorough justification.

\*Maximum user weight 100 kg and always use one higher spring rate category than shown in the Spring set selection table.

To minimise the risk of slipping and tripping, appropriate footwear that fits securely onto the footshell must be used at all times.

## Contra-indications

This device might not be suitable for Activity Level 1 individuals or for competitive sports events, as these types of users will be better served by a specially designed prosthesis optimized for their need.

Intended for a single user.

Ensure that the user has understood all instructions for use, drawing particular attention to the section regarding maintenance.

## Spring set selection

|          | User weight |           |           |           |           |           |           |           |                 |
|----------|-------------|-----------|-----------|-----------|-----------|-----------|-----------|-----------|-----------------|
| Activity | 44–52       | 53–59     | 60–68     | 69–77     | 78–88     | 89–100    | 101–116   | 117–125   | kg              |
|          | (100–115)   | (116–130) | (131–150) | (151–170) | (171–195) | (196–220) | (221–255) | (256–275) | (lb)            |
| 3        | 1           | 2         | 3         | 4         | 5         | 6         | 7         | 8         | Foot spring set |

### Activity Level 3

Has the ability or potential for ambulation with variable cadence. Typical of the community ambulator who has the ability to traverse most environmental barriers and may have vocational, therapeutic, or exercise activity that demands prosthetic utilization beyond simple locomotion.

Note:

If in doubt choosing between two categories, choose the higher rate spring set.


Foot Spring set recommendations shown are for transtibial users.

For transfemoral users we suggest selecting a spring set one category lower, refer to fitting advice Section 7 to ensure satisfactory function and range of movement

## 2 Construction

### Principal parts:

- Carrier Assembly (aluminum/St. Stl./titanium)
- Heel & Toe Springs (e-carbon)
- Spring attachment screws (titanium/St. Stl.)
- Glide Sock (UHM PE)
- Foot shell (PU)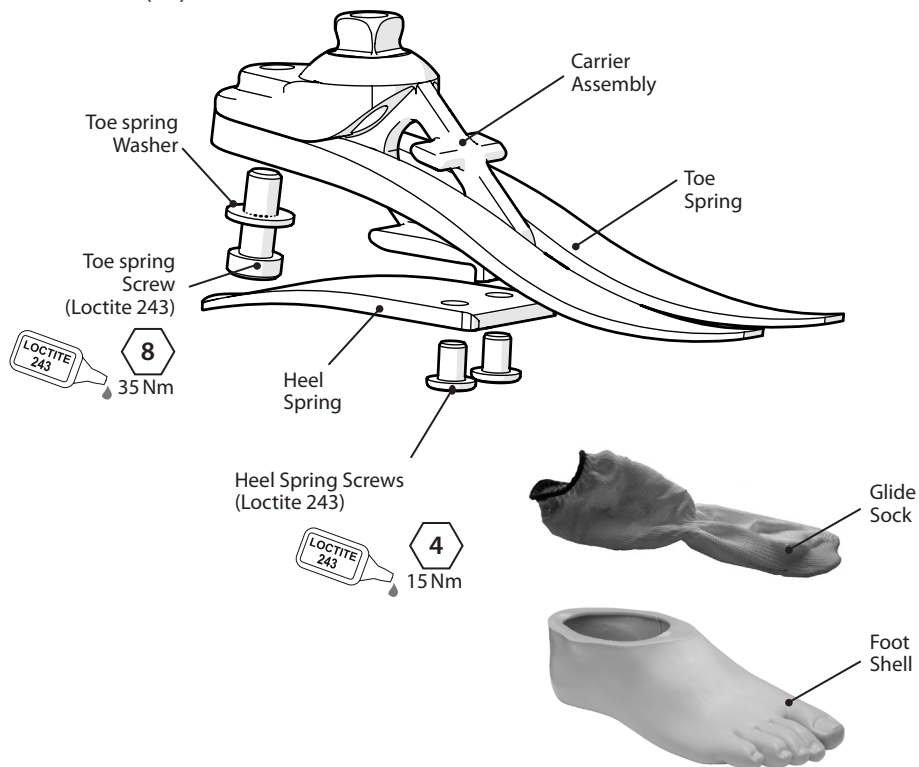

---

## 3 Function

This device comprises an e-carbon toe and independent heel spring. Heel and toe springs are attached to the carrier using titanium and stainless steel screws. The foot is wrapped in a UHM PE sock which is in turn surrounded by a PU foot shell.

---

## 4 Maintenance

Maintenance must be carried out by competent personnel.

We recommend the following annual maintenance:

- Remove the foot shell and glide sock, check for damage or wear and replace if necessary.
- Check all screws for tightness, clean and reassemble as necessary.
- Visually check the heel and toe springs for signs of delamination or wear and replace if necessary. Some surface damage may occur after a period of use, this does not affect the function or strength of the foot.

The user must be handed the user information card supplied, and be advised of the following:

- Any changes in performance of this device must be reported to the practitioner e.g. reduced energy return or unusual noises.
- The practitioner must also be informed of any changes in body weight and/or activity level.

The user should be advised that a regular visual check of the foot is recommended, signs of wear that may affect function should be reported to their service provider (e.g. significant wear or excessive discoloration from long term exposure to UV).

### Cleaning

Use a damp cloth and mild soap to clean outside surfaces, do not use aggressive cleansers.

---

## 5 Limitations on use:

### Intended life

A local risk assessment should be carried out based upon activity and usage.

### Lifting loads

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

### Environment

This device is waterproof to a maximum depth of 1 meter.

Thoroughly rinse this device with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts.

Thoroughly rinse with fresh water after use in salt or chlorinated water.

Foot products must be adequately finished to prevent water ingress into the foot shell where possible. If water enters the foot shell, the limb should be inverted and dried before further use.


Suitable for submersion

Exclusively for use between -15 °C and 50 °C (5 °F to 122 °F).

We recommend using only Endolite products with this device.

## 6 Bench Alignment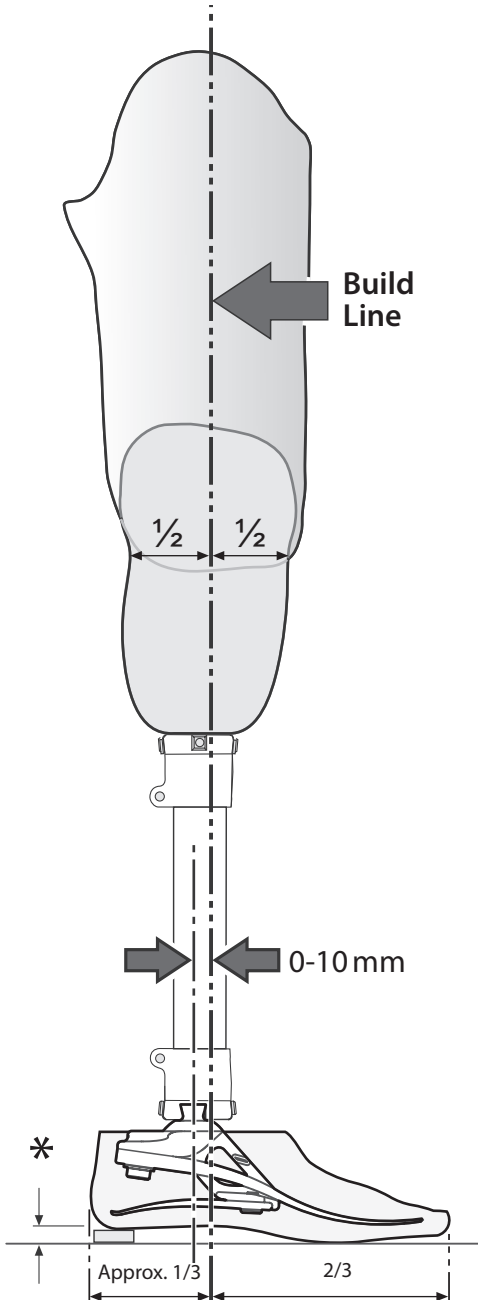

### Static alignment

#### Setup length

With flexion, adduction and abduction properly accommodated, set the limb length 5 mm longer than the sound side to allow for compression and deflection of the foot springs during gait. This should be re-assessed once the dynamic trial commences and the length adjusted accordingly.

#### Load line

This should fall between the centre line of the pyramid and 10 mm anterior (with heel height properly accommodated). The socket should be positioned accordingly.

### Dynamic alignment

#### Coronal plane

Ensure that M-L thrust is minimal by adjusting relative positions of socket and foot.

#### Sagittal plane

Check for smooth transition from heel strike to toe-off. Ensure also that when standing the heel and toe are evenly loaded and that both are touching the floor.

### Transfemoral alignment

Align transfemoral components according to fitting instructions supplied with the knee, keeping the build line relative to this device as shown.

\* Allow for users own footwear

## 7 Fitting Advice

Spring sets are supplied as matched pairs i.e. the heel and toe spring are designed to work together to give smooth progression for most users.

### Heel stiffness

Progression throughout the stance phase should be smooth; heel function is key to this process:

- Too soft a heel or load line excessively posterior will result in sinking at heel strike and difficulty in getting over the toe.
- Too hard a heel or load line excessively anterior will result in a rapid progression through mid-stance or jarring at heel strike.

|                      | Symptoms   | Remedy   |
|----------------------|--|--|
| <b>Heel too soft</b> | <ul style="list-style-type: none"><li>• Sinking at heel strike</li><li>• Difficulty climbing over the toe (toe feels too hard)</li></ul>   | <ol style="list-style-type: none"><li>1. Move socket anteriorly in relation to the foot (excess movement may result in drop-off)</li><li>2. Fit a stiffer spring set</li></ol> |
| <b>Heel too hard</b> | <ul style="list-style-type: none"><li>• Rapid transition from heel strike through stance phase</li><li>• Difficulty in controlling heel action, foot jars into mid-stance</li><li>• Foot feels too rigid</li></ul> | <ol style="list-style-type: none"><li>1. Move socket posteriorly in relation to foot</li><li>2. Fit softer spring set</li></ol>  |

Please contact your supplier if it is not possible to achieve a smooth gait after following the advice above.


# 8 Assembly Instructions

**Be aware of finger trap hazard at all times.**

**1**

Assemble toe spring onto carrier, use Loctite 243 (926012) and torque to 35 Nm.

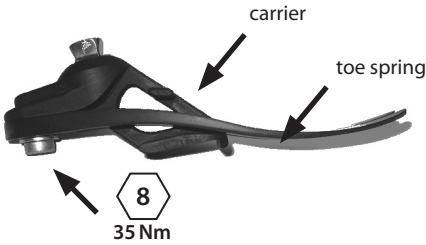**2**

Assemble heel spring onto carrier, use Loctite 243 (926012) and torque to 15 Nm.

**3**

Cover appropriate lines on carrier with permanent black marker to leave spring set number showing.

**4**

**928017**  
Lubricate toe and heel if required.  
(Foot shell is pre-lubricated)

Fit glide sock over the carrier assembly as shown.

**5**

If a foam cosmesis is to be fitted, roughen top surface of foot shell to provide ideal bonding surface.

**6**


Slide carrier/heel spring assembly into the foot shell.

## 8 Assembly Instructions *(continued)*

**7**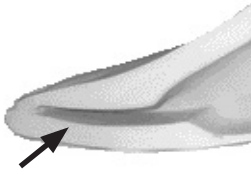

Toe spring location in foot shell


**8**

Use a suitable lever to encourage the heel spring into the location in the foot shell.

**9**

Ensure heel spring is engaged into slot.

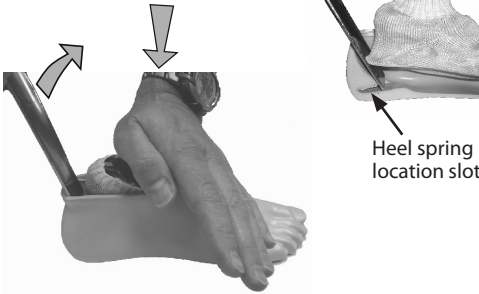Heel spring location slot

**10**

Ensure glide sock does not get trapped when assembling to female pyramid part.


## 9 Disassembly Instructions

Disassembly is the reverse of assembly.

# 10 Technical Data

|  |                                     |
|--|-------------------------------------|
| Operating and Storage Temperature Range: | -15 °C to 50 °C<br>(5 °F to 122 °F) |
| Component Weight (Size 26N):             | 530 g (1 lb 3 oz)                   |
| Activity Level:                          | 3                                   |
| Maximum User Weight:                     | 125 kg (275 lb)                     |
| Proximal Alignment Attachment:           | Male Pyramid (Endolite)             |
| Range of Adjustment:                     | ±7° angular                         |
| Build Height:                            | (sizes 22–24) 65 mm                 |
| (See diagram below)                      | (sizes 25–26) 70 mm                 |
|  | (sizes 27–30) 75 mm                 |

## Fitting length

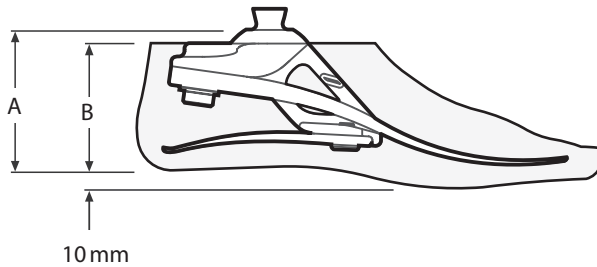| Size  | A     |
|-------|-------|
| 22–24 | 65 mm |
| 25–26 | 70 mm |
| 27–30 | 75 mm |

| Size  | B     |
|-------|-------|
| 22–26 | 65 mm |
| 27–28 | 70 mm |
| 29–30 | 75 mm |

# 11 Ordering Information

## Order example

|            |           |               |                 |                        |               |
|------------|-----------|---------------|-----------------|------------------------|---------------|
| <b>ESP</b> | <b>25</b> | <b>L</b>      | <b>N</b>        | <b>3</b>               | <b>S</b>      |
|            | Size      | Side<br>(L/R) | Width*<br>(N/W) | Spring Set<br>Category | Sandal<br>Toe |

Available from size 22 to size 30:

ESP22L1S to ESP30R8S

ESP22L1SD to ESP30R8SD

(add 'D' for a dark tone foot shell)

\*Sizes 25-27 only. For all other sizes, omit the Width field.

e.g. ESP25LN3S, ESP22R4S, ESP27RW4SD

| Spring Kits |            |            |           |                  |
|-------------|------------|------------|-----------|------------------|
| Rate        | Foot sizes |            |           |                  |
|             | Small (S)  | Medium (M) | Large (L) | Extra Large (XL) |
|             | 22-24      | 25-26      | 27-28     | 29-30            |
| Set 1       | 539801S    | 539810S    | 539819S   | 539828S          |
| Set 2       | 539802S    | 539811S    | 539820S   | 539829S          |
| Set 3       | 539803S    | 539812S    | 539821S   | 539830S          |
| Set 4       | 539804S    | 539813S    | 539822S   | 539831S          |
| Set 5       | 539805S    | 539814S    | 539823S   | 539832S          |
| Set 6       | 539806S    | 539815S    | 539824S   | 539833S          |
| Set 7       | 539807S    | 539816S    | 539825S   | 539834S          |
| Set 8       | 539808S    | 539817S    | 539826S   | 539835S          |

| Foot Shell (for dark add 'D') |          |          | Glide Sock |
|-------------------------------|----------|----------|------------|
| Size/Side                     | Narrow   | Wide     |            |
| 22L                           | 539038S  | -        | 531011     |
| 22R                           | 539039S  | -        |            |
| 23L                           | 539040S  | -        |            |
| 23R                           | 539041S  | -        |            |
| 24L                           | 539042S  | -        |            |
| 24R                           | 539043S  | -        |            |
| 25L                           | 539044SN | 539044SW |            |
| 25R                           | 539045SN | 539045SW |            |
| 26L                           | 539046SN | 539046SW | 532811     |
| 26R                           | 539047SN | 539047SW |            |
| 27L                           | 539048SN | 539048SW |            |
| 27R                           | 539049SN | 539049SW |            |
| 28L                           | -        | 539050S  |            |
| 28R                           | -        | 539051S  |            |
| 29L                           | -        | 539052S  |            |
| 29R                           | -        | 539053S  |            |
| 30L                           | -        | 539054S  |            |
| 30R                           | -        | 539055S  |            |

**Liability**

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

**CE Conformity**

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

**Warranty**

This device is warranted for 36 months - foot shell 12 months - glide sock 3 months. The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions. See Endolite website for the current full warranty statement.

**Environmental Aspects**

Where possible the components should be recycled in accordance with local waste handling regulations.

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Endolite India Ltd
Endolite Deutschland GmbH,