EN	User Guide	1
FR	Guide de l'Utilisateur	10
DE	Bedienungsanleitung für Anwender	19
IT	Guida per il Paziente	28
ES	Guia de Usuario	37
NO	Brukerhåndbok	46
RU	Руководство пользователя	55
TR	Kullanım Kılavuzu	64

**LiNX**

endolite

## Contents

- 1 Introduction
  - Safety Information
- 2 Key Points
- 3 Transport and Storage
- 4 Battery Charging


### Battery Safety

Linx contains Li-Ion type rechargeable batteries. Please read and comply with the safety information see Section 4. Failure to do so may cause the battery to become hot, explode, or ignite and cause serious injury.

#### **IMPORTANT:** Battery Charging

Linx has an On/Off Switch which must be in the ON position to ensure that the Linx battery is being charged. If Linx is charging the Battery Charger LED will be flashing green.

The battery should only be charged within the temperature range 10 °C to 40 °C (50 °F to 104 °F).

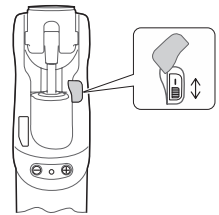Battery On/Off Switch

### Low Temperature operation

#### **IMPORTANT:**

If the limb is subjected to temperatures lower than -10°C ( 14°F) it must be returned to be inspected for possible damage to the battery pack.

Following extended periods of inactivity at temperatures below 0°C (32°F) the limb may swing less freely. In such an eventuality, it is recommended to exercise caution while descending stairs and avoid descending stairs leg-over-leg using the knee yield until you feel the swing returns to normal.

#### **Notes:**

1. The Linx firmware has been updated. To check which version of the Linx firmware is installed on your device, reset the device. If your device emits 2 short beeps 30 seconds after turning on, it is installed with version 1.0N or higher of the Linx firmware.
2. This symbol represents information that only applies to devices with version **1.0N or higher** of the Linx firmware: **1.0N+**
3. This symbol represents information that only applies to devices with version **1.0B or lower** of the Linx firmware: **1.0B**

## Package Contents:

- 1 Linx limb
- 2 User guide
- 3 Cable bag and accessories
  - 1 Battery charger
  - 2 Battery charger AC power socket outlet adapter(s)

---

# 1 Introduction

Linx is a fully integrated, microprocessor-controlled knee and foot system. The four microprocessors provide simultaneous control over both foot and knee co-ordinating their responses for a variety of terrains and situations in one complete biomimetic limb. Individual adjustment can be made for slopes and stairs and a stop and lock mode for standing. The set-up procedure is through an easy to use PC interface via a Bluetooth® connection to the on-board module.

## Application

Linx is to be used exclusively as part of a lower limb prosthesis.

Intended for single user.

Ensure that the user has understood all instructions for use, drawing particular attention to the sections regarding safety and maintenance.

## Features

- All the functions of a microprocessor control in both knee and hydraulic ankle
- Centralised control of knee and foot to give the user a better response
- One software based set-up via a built-in Bluetooth link
- Proven to reduce stress at the socket interface
- An easy access single charging and point
- Longer battery life, 3 days average use from the internal rechargeable Li-ion battery pack

The enhanced stability system offers:

- *Stop and Lock* mode, control of knee flexion and ankle for standing
- Controlled ramp descent with *Braking* effect of knee and ankle
- Provides assistance during ramp ascent
- Dynamic stair descent with an easy start of knee flexion coupled with increasing support to gently lower the user during flexion
- Supportive resistance to flexion as soon as the knee stops flexing
- Increased supportive resistance to flexion to prevent 'stumbles' associated with instability at heel strike

# Safety Information

 **The caution symbol highlights safety information which must be followed carefully.**

-  Any changes in the performance of the limb e.g. restricted movement, non-smooth motion or unusual noises should be immediately reported to your service provider.
-  Always use a hand rail when descending stairs and at any other time if available.
-  Any excessive changes in heel height after programming will adversely affect limb function and should be immediately reported to your service provider for reprogramming and calibration.
-  After continuous use the ankle and knee casing may become hot to the touch.
-  Avoid strong magnetic fields, sources of electrical interference, atmospheres containing liquids and/or powders.
-  Do not place near any heat source. Do not leave in direct sunshine or inside a car in hot weather.
-  Linx is not intended for use when immersed in water or as a shower prosthesis. If the limb comes into contact with water wipe dry immediately.
-  Ensure no water enters the charger plug socket and the cover is on at all times.
-  Never plug the charger into Linx whilst still wearing the limb.
-  Be aware of finger trap hazard at all times.
-  Linx is not suitable for extreme sports, running or cycle racing, ice and snow sports, extreme slopes and steps. Any such activities undertaken are done so completely at the users' own risk. Recreational cycling is acceptable.
-  Contact your service provider for advice if Linx is to be used with a pacemaker or any other electronic medical device.
-  Assembly, programming and repair of Linx must only be carried out by a suitably qualified clinician, certified by Endolite.
-  The user must not adjust or tamper with the setup of Linx.
-  The user should be advised to contact their clinician if their condition changes.
-  Linx detects standing and walking on inclines, declines and walking at various speeds. Non-identified movements may cause unexpected behaviour.
-  Only charge the batteries in Linx using the charger supplied with the product. Do not use this charger for charging any other device.
-  Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.
-  Make sure that no liquids enter the device during donning or normal use.
-  To minimise the risk of slipping and tripping, appropriate footwear that fits securely onto the footshell must be used at all times.

---

## 2 Key points

Linx **must not** be used with large variations in heel height.

**Do not** attempt stairs descent leg-over-leg on a Linx limb unless you have been trained for stairs descent.

High flexion resistance settings may induce some noise during stairs descent, this will not affect function.

Any changes in performance of this device must be reported to the clinician.

Changes in performance may include:

- Increase or decrease in knee and/or ankle stiffness
- Instability
- Knee starting to extend too aggressively
- Reduced ankle/knee support (free movement)
- Any unusual noise
- Any unrecognized beeps or warning lights

The practitioner must also be informed of the following:

- Any changes in body weight and/or activity level.
- Significant wear or excessive discoloration from long term exposure to UV.

### Cleaning:

Use a damp cloth and mild soap to clean outside surfaces, **DO NOT** use aggressive cleansers. Ensure water/liquids do not enter the charging socket. Thoroughly dry before use.

### Environment:

Avoid exposing Linx to corrosive elements such as water, acids and other liquids. Also avoid abrasive environments such as those containing sand for example as these may promote premature wear.

Exclusively for use between -10 °C and 50 °C (14 °F to 122 °F).

Suitable for outdoor use

---

## 3 Transport and storage

1. Ensure that Linx is in a straight (fully extended) position when not in use.
2. If the limb is to be stored upright then it must be secured to prevent it falling over or flexing.
3. If Linx is shipped either on its own or built into a limb it should be maintained in an extended position and packaged in a suitable box to allow this.
4. If a unit is to be returned, contact Endolite prior to sending if any obvious physical damage has occurred to the battery pack.
5. For transport purposes the unit must be switched off, and packed securely so that
  - a) it cannot be inadvertently switched on and become operative while in transit;
  - b) the unit is thoroughly protected from damage in transit.

## 4 Battery Charging


Linx contains Li-Ion type rechargeable batteries. Please read and comply with the safety information. Failure to do so may cause the battery to become hot, explode, or ignite and cause serious injury.

The batteries are not field replaceable and will be replaced as necessary when serviced.

If the limb is subjected to temperatures lower than  $-10^{\circ}\text{C}$  ( $14^{\circ}\text{F}$ ) it must be returned to be inspected for possible damage to the battery pack.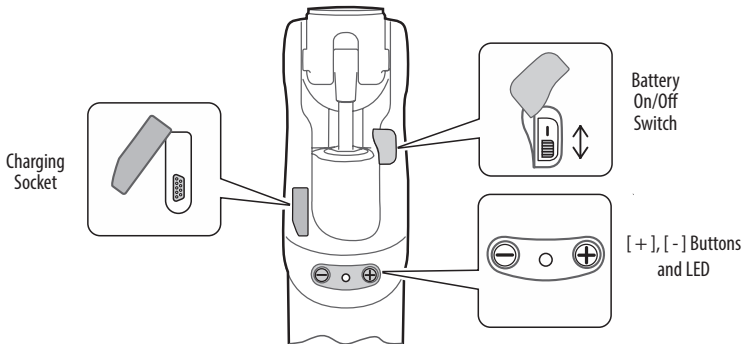

1. The battery should only be charged in temperatures between  $10^{\circ}\text{C}$  to  $40^{\circ}\text{C}$  ( $50^{\circ}\text{F}$  to  $104^{\circ}\text{F}$ ).

2. Only charge the batteries in Linx using the charger supplied with the product.

Do not use this charger for charging any other device.

3. The battery charger can be used with an electrical outlet that is rated between 100V and 240V AC, 50/60Hz.

4. Never plug the charger into Linx whilst still wearing the limb.

5. During charging no power is delivered to the Linx control system.

6. The knee module may become warm during charging but should never be too hot to touch. There should be no unusual smell, if this is the case immediately switch **Off** the battery, unplug the charger, from the electrical supply and contact your service provider.

7. Do not refit the limb until the charger has been disconnected.

8. Always recharge Linx before use if it has not been used for an extended period of time.


## Battery Charging (continued)

Fully charge the battery before first use.

We recommend that the Linx limb should be fully charged every night. It should take 8 hours to charge fully. We also recommend battery charging whenever convenient.

Step by Step Guide:

1. Lay the limb on a flat surface.
2. Switch the Battery On/Off switch to ON.
3. Connect the charger plug to the charging socket on the posterior part of the knee module.
4. Check the Battery Charger LED is flashing green to confirm the battery is charging.
5. Refer to table below to ensure limb is charging.
6. Disconnect from the charger when charged sufficiently.
7. Wait for 30 seconds while observing the Linx Limb LED changing color as indicated:
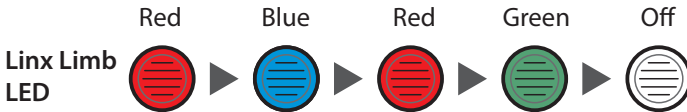


8. **1.0N+** After a successful reset sequence, 2 short beeps will be heard.
9. If reset sequence is unsuccessful or the LED remains Red, switch the battery off then on and repeat from step 7.

Battery Charger LED	Charging state
Yellow	Standby
Green > fast flashing	Fast charging
Green > slow flashing	Close to full charge
Green	Maintain (trickle charge)
Yellow > fast flashing	Error (Ensure Battery Switch is ON)

### NOTE:

If the charger LED is yellow and the limb is making a clicking noise check that the Battery switch is in the ON position.

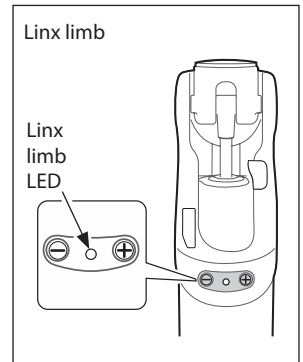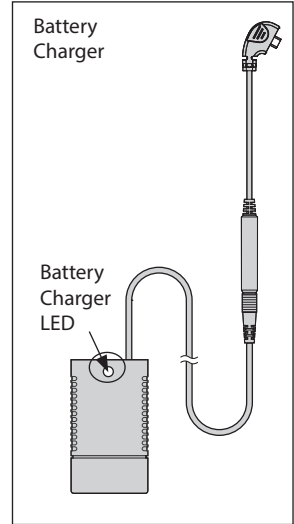
8. **1.0N+** You can enable or disable the audible reset confirmation by pressing and holding the [ + ] and [ - ] buttons simultaneously for 2 seconds. The change will be confirmed by 2 short beeps.

---

## Battery Status:

To check the battery condition press and *hold* the [-] button on the knee module, one short beep will be heard and the LED will indicate the battery condition.

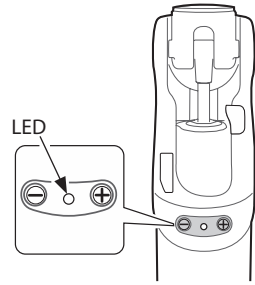Battery Charge Indicator			
LED		Beeps	Battery Condition
Green	3 flashes	--	fully charged
Green	2 flashes	--	good
Green	1 flash	--	ok
Yellow	slow flash	8 x Short beeps	low charge
Red	slow flash	5 x Long beeps	no charge

When fully charged the Linx battery will provide 3 days of normal use (conditional on the type of use). We recommend that the limb be fully charged every night. Switching off Linx when not in use will also prolong the period of use (Note: Linx will take 30 seconds to reset after it is switched back ON, refer to Battery Charging for reset sequence).

---

## 5 User Modes

Linx has 2 user modes. Please note that these modes are only available if they have been enabled by your clinician.

**Warning:**

Care must be taken to only use those modes as instructed by your clinician.

### 1 Flexion Lock

Sets the limit of flexion to a predetermined angle.

The maximum lock angle is 45 degrees from fully extended.

### 2 Cycling (Free swing)

Releases stance resistance, the Linx limb will then swing freely.

**Warning:**

In Cycling mode there is no stance control active, this mode must be de-activated before resuming walking to avoid falling and possible injury.

### Flexion Lock mode

User activation of Flexion Lock mode with [ - ] button on the Linx knee module

To activate Flexion Lock:

Fully extend the knee.

Press the [ - ] button 3 times, a short beep confirms each button press. **1.0B** 6 or **1.0N+** 3 medium beeps confirm that Flexion Lock is ON.

To de-activate Flexion Lock:

Press and hold [ - ] you will hear 2 long beeps.

### Cycling mode

**Warning:**

In cycling (free swing) mode there is NO stance resistance.

User activation of Cycling mode with [ - ] button on the Linx knee module

To activate Cycling mode:

Flex the knee to 90 degrees. Press the [ - ] button 4 times, a short beep confirms each button press. **1.0B** 5 or **1.0N+** 4 medium beeps confirm that Cycling mode is activated.

To de-activate Cycling mode:

Press and hold [ - ] you will hear 2 long beeps.

---

This device complies with part 15 of the FCC Rules.  
Operation is subject to the following three conditions:-

- 1) This device may not cause harmful interference.
  - 2) This device must accept any interference received, including interference that may cause undesired operation.
  - 3) Contains pre-approved Bluetooth module:
- FCC ID: PVH0946 IC: 5325A-0946
- 

### **Liability**

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

### **CE Conformity**

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

### **Warranty**

This device is warranted for 36 months, the foot shell for 12 months, and the glide sock for 3 months.

Extended warranties of an additional 12, 24, or 36 months are available.

A complimentary service between 30 and 36 months is available on request, and is required for all extended warranties.

For the maximum 36-month extended warranty, a second complimentary service must be carried out between 54 and 60 months.


We reserve the right to void extended warranties if the device is not serviced at the required intervals.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions.

See the Endolite website for the current full warranty statement.

### **Environmental Aspects**

This symbol indicates that the product contains electrical/electronic components and/or batteries that should not be disposed of in general waste or be incinerated at the end of the product's life.

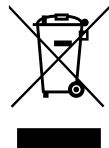At the end of the product's life, all electrical/electronic components and/or batteries should be recycled or disposed of in accordance with the current regulations for the handling of WEEE (Waste Electrical and Electronic Equipment), or equivalent local regulations. The remainder of the product should also be recycled where possible in accordance with local waste recycling regulations.

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