Elanic

User Guide

ELANIC22L1S - ELANIC30R8S
ELANIC22L1SD - ELANIC30R8SD

EN	User Guide	3
FR	Guide de l'Utilisateur	15
DE	Bedienungsanleitung für Anwender	27
IT	Guida per il Paziente	39
ES	Guia de Usuario	51
NO	Brukerhåndbok	63
RU	Руководство пользователя	75
PL	Podręcznik użytkownika	87
AR	دليل المستخدم	99

Contents

Contents	2
1 Description and Intended Purpose	3
2 Safety Information	4
3 Construction	5
4 Key points and precautions for use	6
5 Battery Charging	7
5.1 Battery Level Indication	9
5.2 Device Status Indication	9
Appendix 1 Quick Guide to LED Indications	12

1 Description and Intended Purpose

EN

The term *device* is used throughout these instructions for use to refer to ElanIC.

Application

The device is a biomimetic foot and ankle system with microprocessor controlled dorsiflexion and plantarflexion. It is designed to adapt dynamically (via motorized valves) to provide assistance when walking on a variety of surfaces, when changing gait speeds and increased levels of safety and security when walking and standing, up to the maximum user weight of 125 kg (275 lbs).

Features

- Dynamic damping resistance aims to improve gait characteristics enabling safer walking
- Microprocessor control of resistance in real time
- The device Software adapts to individual user gait characteristics
- Standing Support mode
- Aims to reduce stress at the socket interface
- PC interface setup with **Bluetooth**® link and direct communications
- Power is supplied via an integral rechargeable battery pack
- Recharging through easy-to-use magnetically self-aligning inductive charger
- Waterproof according to the parameters described in this manual

This device provides limited self-alignment of the prosthesis on varied terrain and following footwear changes. It is intended to improve postural sway, balance stability and inter-limb loading symmetry while easing high pressures at the socket interface.

The device and its charger must not be used by users of pacemakers or implantable cardioverter-defibrillators (ICDs).

2 Safety Information

The caution symbol highlights safety information which must be followed carefully.

1. The device and its charger must not be used by users of pacemakers or implantable cardioverter-defibrillators (ICDs). Contact your service provider for advice if the device is to be used with any other electronic medical device.

2. Any changes in performance of the ankle motion e.g. restricted movement, non-smooth motion should be immediately reported to your service provider.

3. Always use a hand rail when descending stairs and at any other time if available.

4. Any excessive changes in heel height after programming will adversely affect ankle function and should be immediately reported to your service provider for reprogramming and calibration.

5. After continuous use or charging the ankle casing may become hot to the touch.

6. Avoid strong magnetic fields, sources of electrical interference.

7. Avoid exposure to extreme heat and/or cold.

8. The device is sealed for water resistance. However the water resistance of the device is limited by environmental factors and exposure time as detailed in this manual.

9. The inductive battery charger kit is not waterproof. Do not charge in atmospheres containing liquids and/or powders.

10. The device is not suitable for extreme sports, running or cycle racing, ice and snow sports, extreme slopes and steps. Any such activities undertaken are done so completely at the users' own risk. Recreational cycling is acceptable.

11. Assembly, programming and repair of the device must only be carried out by suitably qualified practitioners, certified by Blatchford.

12. The user must not adjust or tamper with the setup of the device.

13. The user should be advised to contact their clinician if their condition changes.

14. The device detects standing and walking on inclines, declines and walking at various speeds on normal ground. Non-identified movements such as treadmills and travelators may cause unexpected ankle behaviour.

15. Ensure only suitably retrofitted vehicles are used when driving. All persons are required to observe their respective driving laws when operating motor vehicles.

16. To minimise the risk of slipping and tripping, appropriate footwear that fits securely onto the footshell must be used at all times.

17. Be aware of finger trap hazard at all times

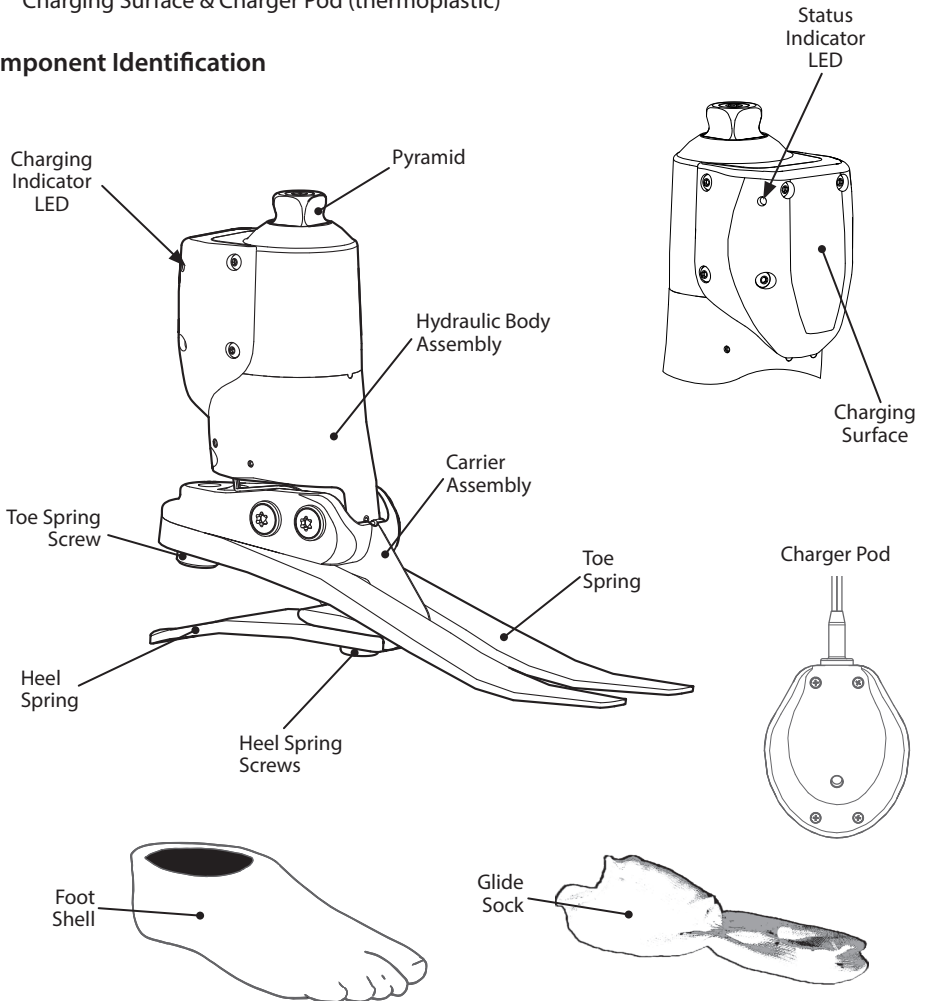
See further warnings relating to the charging of the device in Section 5 Battery Charging/Warnings.

3 Construction

Principal parts:

- Hydraulic Body Assembly including pyramid (aluminum/St. Stl./titanium incorporating batteries (NiMH), electric motors and PCB)
- Carrier Assembly (aluminum/St. Stl.)
- Heel & Toe Springs (e-Carbon)
- Spring Attachment Screws (titanium/St. Stl.)
- Glide Sock (UHM PE)
- Foot Shell (PU)
- Charging Surface & Charger Pod (thermoplastic)

Component Identification

4 Key points and precautions for use

Maintenance must be carried out by competent personnel.

It is recommended that maintenance is carried out annually and a service carried out as a condition of warranty every 20 months.

A regular visual check of the foot is recommended, signs of wear or corrosion that may affect function should be reported to the service provider.

Key points

Any changes in performance of this device must be reported to the practitioner

Changes in performance may include:

- Increase in ankle stiffness
- Reduced ankle support (freer movement)
- Any unusual noise (silent buzz from the device during charging is normal)
- Recurring rapid flashing of Charging Indicator/Status Indicator LEDs.

At low battery power levels the device will go into sleep mode, switch off dynamic control and use only the basic resistance settings and may affect **Bluetooth**® connectability. Charge the device batteries as soon as possible to restore Dynamic mode.

The practitioner must also be informed of the following:

- Any changes in body weight and/or activity level.
- Significant wear or excessive discoloration from long term exposure to UV.

Cleaning

Use a damp cloth and mild soap to clean outside surfaces, **DO NOT** use aggressive cleansers.

Transport and Storage

Storage temperature range between -15°C and 50°C (5 °F and 122 °F).

If the device is shipped, send the device and the charging pod in the packaging supplied with the unit. Always ship the device with the charger pod.

When built into a limb it should be stored securely so as not to damage the unit.

Lifting loads:

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

Environment

This device is IP67 rated: waterproof to a maximum depth of 1 meter. Occasional submersion should be limited to a max. of 30 minutes. Dry after using in water.

Suitable for submersion

Avoid exposing the device to abrasive or corrosive environments, such as those containing sand, acids, salt, chlorinated, or chemically treated water e.g. swimming pools. Failure to comply will void the warranty. After any accidental exposure to such environments rinse the device in fresh water and dry thoroughly.

Exclusively for use between -15°C and 50°C (5°F and 122°F).

Charge only between 0 °C to 35 °C (32 °F to 95 °F)

5 Battery Charging

i Please familiarize yourself with the charging information in this section.
The batteries are not field replaceable and will be replaced as necessary when serviced.

Read the user instructions supplied with the charger before using for the first time.

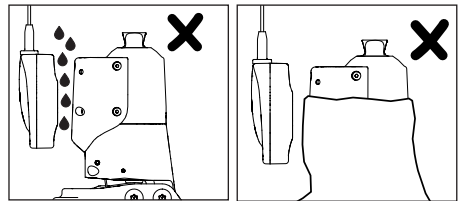
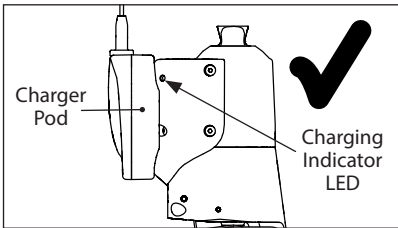
We recommend the device is charged daily; it should take 5.5 hours to charge fully, depending on residual charge. An exhausted or flat battery may take longer.

Setting Up

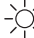
Lay the limb on a flat surface and connect the inductive charger pod to the charging surface on the back of the device.

Use the Charging Indicator LED on the right hand side of the device to confirm that charging has initiated correctly:

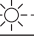
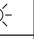
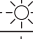
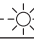
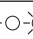
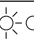
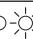
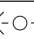
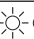
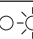
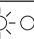
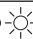
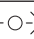
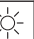
! Do not allow anything (e.g. stocking, clothing) to intrude in between the charging surface and the charger pod while charging is in progress. This will result in decreased charging performance and buildup of heat.

Charging

One initial flash  x1 — Connection Established

Followed By

LED	Charging state
Single Flash Repeated  —  —  — 	Charging - Battery Empty
Double Flash Repeated   —   —  	Charging - Charge Level Medium
Triple Flash Repeated    —   	Charging - Charge Level High
Solid ON/Solid OFF  / —	Charging Completed
Rapid Flashing           	High Temperature Shut Down*


*Charging continues when normal charging temperature is reached. (See Section 4.)

Reset

The device will take several seconds to reset when the charger is disconnected and a single blue flash from the Status Indicator LED on the left hand side of the device will confirm a correct startup.

Further information is provided by the charger pod LED (blue):

LED	Error	Corrective Action
Slowly Pulsating	No Error/Normal charging	Not needed
Solid Blue Light	Charger operational but not mated correctly to the charging surface of the device.	Try reattaching and check if anything prevents perfect mating of charging surfaces. Remove any intruding material.
No Light	The charger is not powered.	Check that charger is connected to the mains.
Rapid Pulsating	Possible overheating of the charger.	Charging may resume when normal charging temperature is reached. If recurring contact your service provider.

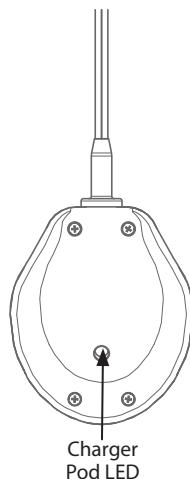Charging Error

Charging error is always indicated by a non-responding Charging Indicator LED on the right hand side of the device (no initial flash or charging state indication).

Warnings

- ⚠ 1. **The device and its charger must not be used by users of pacemakers or implantable cardioverter-defibrillators (ICDs). Contact your service provider for advice if the device is to be used with any other electronic medical device.**
- ⚠ 2. **Only charge the batteries in the device using the charger supplied with the product. Do not use this charger for charging any other device.**
- ⚠ 3. **The battery charger can only be used with an electrical outlet that is rated between 100 V and 240 V AC, 50/60 Hz.**
- ⚠ 4. **Never connect the charger to the device while still wearing the limb.**
- ⚠ 5. **During charging no power is delivered to the ankle control system.**
- ⚠ 6. **The ankle and the charger may become warm during charging but should never be too hot to touch. There should be no abnormal smell. If this is the case immediately unplug from electrical supply and contact your service provider.**
- ⚠ 7. **Do not refit the limb until the charger has been disconnected.**
- ⚠ 8. **Always recharge the device before use if it has not been used for an extended period of time.**
- ⚠ 9. **The inductive battery charger kit is not waterproof. Do not charge in atmospheres containing liquids and/or powders.**
- ⚠ 10. **Do not exceed the charging temperature limitations set out in this manual.**
- ⚠ 11. **When charging an empty battery the charger may terminate prematurely (after a few minutes). If this happens, reconnect the charger. If this problem recurs contact your service provider.**
- ⚠ 12. **Do not leave the charger plugged to electrical supply after use.**

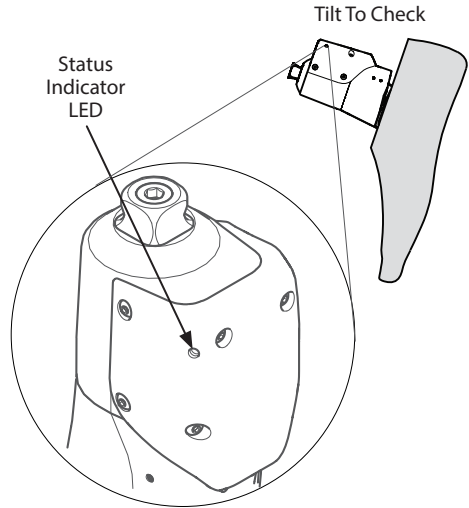
5.1 Battery Level Indication

To check the battery charge level, tilt the foot down vertically and hold stationary for 4 seconds. The Status Indicator LED on the left hand side of the device will flash blue, 1-3 times (see table) and then repeat to indicate the charge status. Tilt the foot again to repeat the battery status check.

Note: This feature can be switched off.

Note: After charging the device fully the Status Indicator LED may indicate 2 or 3 flashes. This will depend on the charging time, the age and condition of the battery.

LED	Battery Charge
3 flashes	Maximum
2 flashes	High
1 flash	Low

5.2 Device Status Indication

The same LED on the left hand side of the device is also used to indicate device status:

1. Single Flash
When the charger is disconnected it will flash once to indicate correct start up.
2. Solid Light
If after the charger is disconnected it stays on it indicates that an error was detected by your device.
3. Flashing rapidly
If it flashes rapidly after charger is disconnected, it indicates that **Bluetooth®** is active - flashing should go away after a few minutes.

Note: The device Status Indicator LED is also used for indicating stages of a **Bluetooth®** connection. The Status Indicator LED may flash rapidly after disconnecting the charger. This capability is to be used only by the clinician. The flashing will cease after a few minutes.

See Appendix 1 for a full summary of all possible LED indications of the device and the charger pod.

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for any adverse outcome caused by any component combinations that were not authorized by them.

CE Conformity

This product meets the requirements of the European Regulation EU 2017/745 for medical devices. This product has been classified as a class I device according to the classification rules outlined in Annex VIII of the regulation. The EU declaration of conformity certificate is available at the following internet address: www.blatchford.co.uk

Medical device

Single Patient – multiple use

Warranty

The device is warranted for – 36 months – foot shell 12 months - glide sock 3 months.

See Blatchford website for the current full warranty statement.

Scheduled service every 20 months is required, for maintenance of the warranty.

For extended warranty contact service provider.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licences and exemptions.

This device complies with part 15 of the FCC Rules.

Operation is subject to the following three conditions:-

1. This device may not cause harmful interference.
2. This device must accept any interference received, including interference that may cause undesired operation.
3. Contains pre-approved **Bluetooth**® module FCC ID: T9J-RN41-1

Reporting of Serious Incidents

In the unlikely event of a serious incident occurring in relation to this device it should be reported to the manufacturer and your national competent authority.

Environmental Aspects

This symbol indicates that the product contains electrical/electronic components and/or batteries that should not be disposed of in general waste or be incinerated at the end of the product's life.

At the end of the product's life, all electrical/electronic components and/or batteries should be recycled or disposed of in accordance with the current regulations for the handling of WEEE (Waste Electrical and Electronic Equipment), or equivalent local regulations. The remainder of the product should also be recycled where possible in accordance with local waste recycling regulations.

To help prevent potential harm to the environment or to human health from uncontrolled waste disposal, Blatchford offers a take-back service. Please contact Customer Services for details.

Manufacturer's Registered Address

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH, UK.

Trademark Acknowledgements

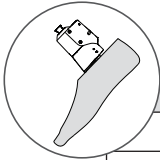
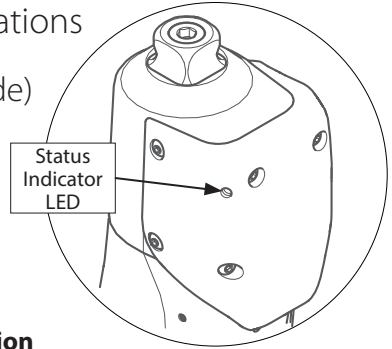
Bluetooth® is the registered trademark of **Bluetooth**® SIG.
Elan and Blatchford are registered trademarks of Blatchford Products Limited.

Appendix 1 Quick Guide to LED Indications

Device Status Indicator LED (Left Hand Side)


Note: Take into consideration that the battery level indication feature may be disabled.

Note: The built-in **Bluetooth**® device can be activated only when the ankle is in a fully dorsiflexed position.

Tilted Position

LED	Indicating	Action
Single Flash x2	Battery level low	Charge to use
Double Flash x2	Battery level high	Charge for maximum performance
Triple Flash x2	Battery level high	Ready for use
Solid after battery level indication (foot dorsiflexed)	Bluetooth ® is ready for turning ON	Turn upright to turn Bluetooth ® ON
Solid without battery level indication (foot dorsiflexed)	Bluetooth ® is ready for turning ON (Battery level indication Disabled)	Turn upright to turn Bluetooth ® ON

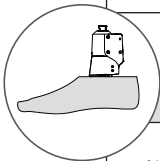
Upright Position

LED	Indicating	Action
Single flash (After disconnecting charger)	Correct start up after charging	None
Solid (After disconnecting charger)	Error detected	Contact service provider
Continuous flashing after disconnecting charger	Bluetooth ® is ready for connection	Wait few minutes for deactivation
Continuous flashing after Bluetooth ® turned On	Bluetooth ® is ready for connection	Connect PC software or Wait for deactivation
Solid after Bluetooth ® connection initiated	Bluetooth ® connection to PC software successful	Use PC software
Continuous flashing after Bluetooth ® connection established	Bluetooth ® connection is broken Bluetooth ® is ready for connection	Try reestablish connection Try reducing distance between devices

Charging Indicator LED (Right Hand Side)

Note: Always interpret the indications given by the Charging Indicator LED and the Charger Pod LED in connection with each other.

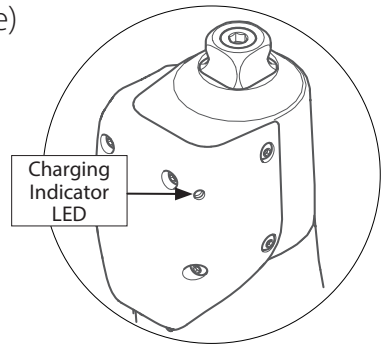Note: : The charger will turn itself off after 6 hours of operation. This will manifest itself by Solid light on Charger Pod LED and no light on Charging Indicator LED. This is normal and usually means that the device charged correctly.

LED	Indicating	Action
Start Up		
No Light	Charger connection failed	Try reconnect
Primary Indications		
Single flash	Correct start up after charger connected	See further indications below
Single flash + LED stays off	Correct start up after charger connected	Battery full
Single flash repeated	Charging empty battery	None
Double flash repeated	Charging medium level battery	None
Triple flash repeated	Charging high level battery	None
Rapid flashing	High temperature shut down	Charging resumes after cooling
End of Charging		
Solid Light	Charging completed	Ready for use

Charger Pod LED

LED	Indicating	Action
Slow pulsating	Normal Operation of charger	None
Solid	Operational but no charging	Check mating to charging surface
No light	Not powered	Check mains connection
Rapid pulsating	Possible overheating	Charging may resume after cooling down

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© Blatchford Products Limited 2020. All rights reserved.
Endolite India Ltd.