Avalon^{K2}

Instructions for Use

AV22L—AV30R
AV22LD—AV30RD

EN	Instructions for Use	2
FR	Instructions D'Utilisation	12
DE	Gebrauchsanweisung	22
IT	Istruzioni per L'Uso	32
ES	Instrucciones de Uso	42
NO	Bruksanvisning	52
RU	Инструкция протезиста	62
TR	Kullanım Talimatları	72

1 Description and purpose

EN

Application

These instructions are for the practitioner.

The term *device* is used throughout these instructions for use to refer to AvalonK2.

This device is to be used exclusively as part of a lower limb prosthesis.

This device dorsiflexes after mid-stance and remains dorsiflexed at toe-off through the swing phase to give greater toe clearance for improved confidence and security. The controlled plantar flexion provided by this device at heel contact improves time to foot flat which enhances stability on uneven surfaces and slopes.

This device is recommended for users that have the potential to achieve Activity Level 2 who may benefit from enhanced stability and an increase in confidence on uneven surfaces and slopes.

Of course there are exceptions and in our recommendation we want to allow for unique, individual circumstances. There may also be a number of users in Activity Level 1 who would benefit from the enhanced stability offered by this device, but this decision should be made with sound and thorough justification.

Contra-indications

Due to the cushioned heel and limited energy return from the foot keel this device is not suitable for Activity Level 3 or 4 individuals. This type of user will be better served by a specially designed prosthesis optimized for their needs.

This device is not suitable for users with poor balance.

Intended for a single user.

Ensure that the user has understood all instructions for use, drawing particular attention to the section regarding maintenance.


Activity Level 2

Has the ability or potential for ambulation with the ability to traverse low level environmental barriers such as curbs, stairs or uneven surfaces. Typical of the limited community ambulator.

2 Construction

Principal parts:

- Hydraulic Body Assembly including pyramid (aluminum/St. Stl./titanium)
- Carrier Assembly (aluminum/St. Stl.)
- Keel (thermoplastic composite)
- Keel Attachment Screws (St. Stl.)
- Glide Sock (UHM PE)
- Foot Shell (PU)

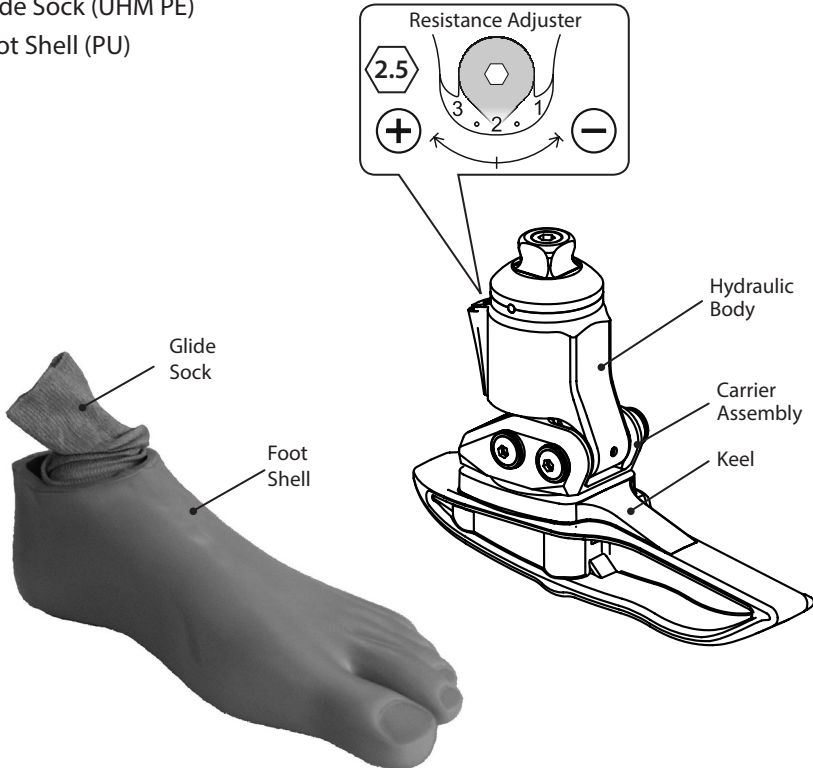3 Function

This device comprises a hydraulic body with a valve that can be adjusted to increase or reduce hydraulic resistance to plantar flexion and dorsiflexion simultaneously. The hydraulic body is connected to a carrier assembly via two pivot pins. A keel is attached to the carrier assembly using stainless steel screws. The foot is wrapped in a UHM PE sock which is in turn surrounded by a PU foot shell.

4 Maintenance

Maintenance must be carried out by competent personnel.

We recommend the following annual maintenance:

- Visually inspect the foot shell and glide sock, check for damage or wear and replace if necessary.

There are no other serviceable parts in the foot assembly.

The user should be advised:

Any changes in performance of this device must be reported to the practitioner

Changes in performance may include:

- Increase in ankle stiffness
- Reduced ankle support (free movement)
- Any unusual noise

Cleaning

Use a damp cloth and mild soap to clean outside surfaces, DO NOT use aggressive cleansers.

5 Limitations on Use

Intended life

A local risk assessment should be carried out based upon activity and usage if usage is beyond recommended activity level.

Lifting loads

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

Environment

This device is waterproof to a maximum depth of 1 meter.

Thoroughly rinse this device with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts.

Thoroughly rinse with fresh water after use in salt or chlorinated water.

Foot products must be adequately finished to prevent water ingress into the foot shell where possible. If water enters the foot shell, it should be inverted and dried before further use.


Suitable for submersion

Exclusively for use between -15 °C and 50 °C.

We recommend using Blatchford products with this device.


6 Bench Alignment

Keep the build line between pivots as shown, using shift and/or tilt devices as necessary.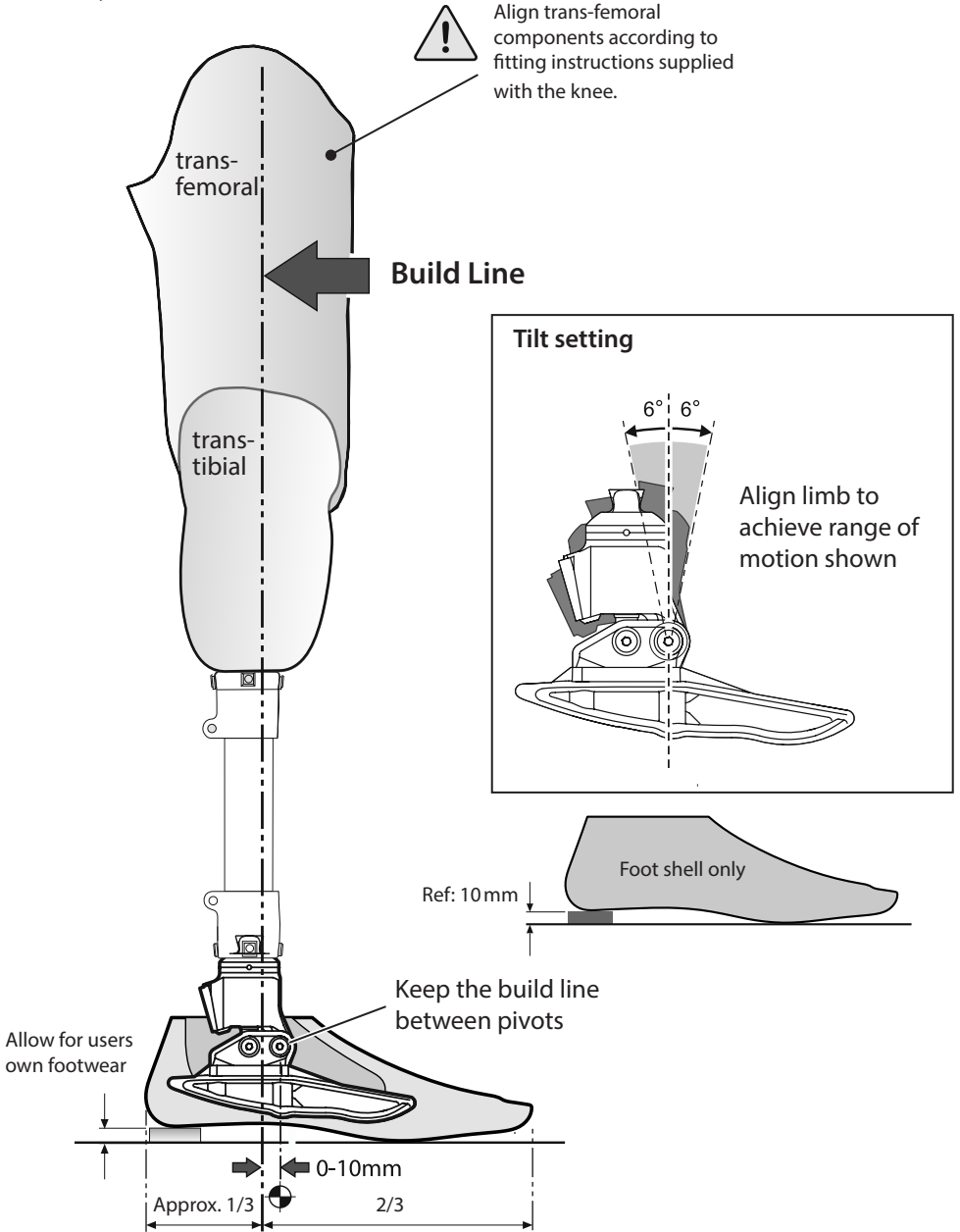

7 Biomimetic Alignment

The aim of alignment is to achieve a *balance point* while standing and set the hydraulically damped range of motion. The aim of damping adjustment is to fine tune the ankle-foot roll-over stiffness characteristics until a comfortable gait is achieved. Due to the increased range of motion provided by the ankle the user may experience the need for more voluntary control and initially find the ankle disconcerting during setup. This should quickly pass upon completion of satisfactory setup.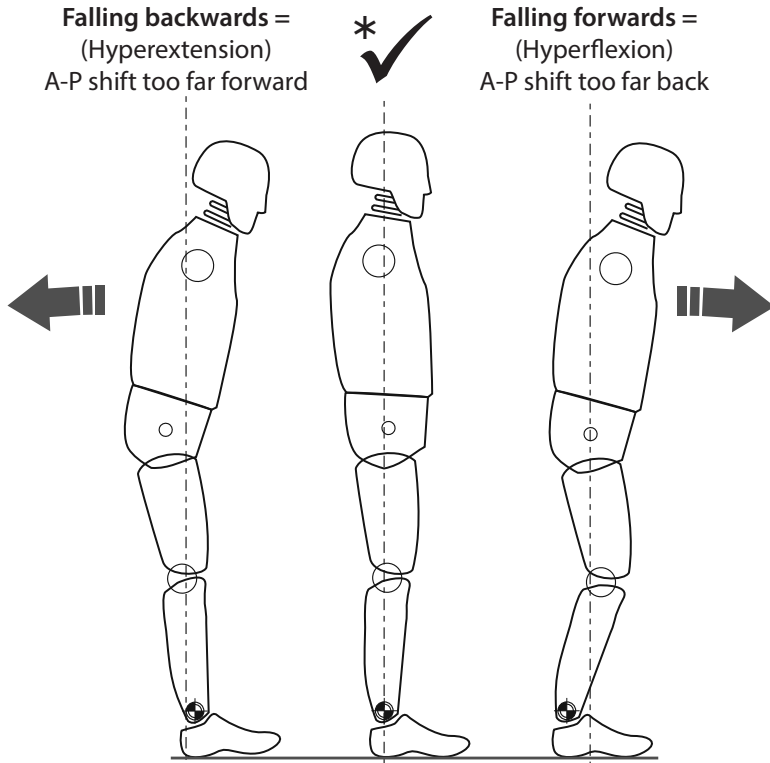


* Ensure the user is not resting on the DF (Dorsiflexion) limit when standing in an upright position.

8 Biomimetic Adjustment

Note: Carry out static alignment while ensuring the user has some means of support such as parallel bars. This is standing alignment only.

Check the Bench Alignment and the heel height
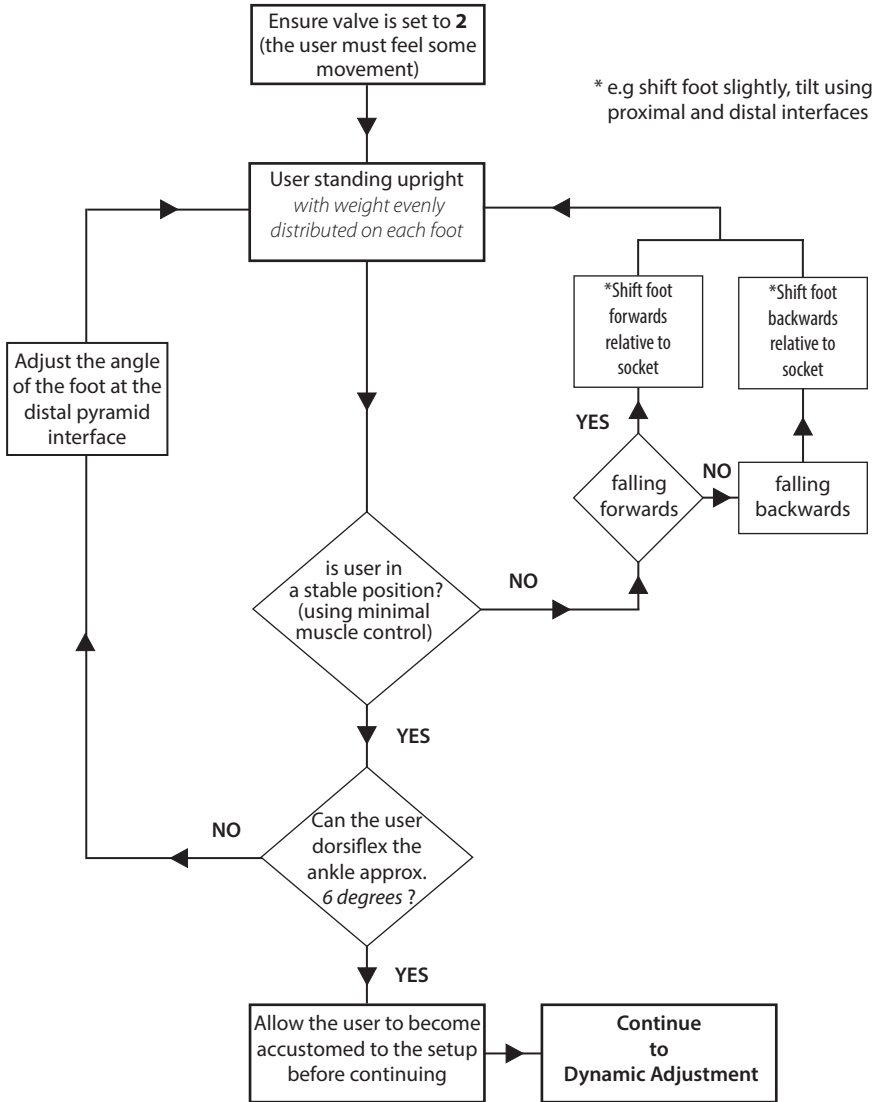


The device should encourage some degree of self adjustment to achieve a sense of balance for the user during standing.

9 Dynamic Adjustment

The user should experience the ankle moving with the body through the gait cycle. There should be no effort exerted by the user to overcome the hydraulic resistance of the ankle.

Resistance adjustment procedure: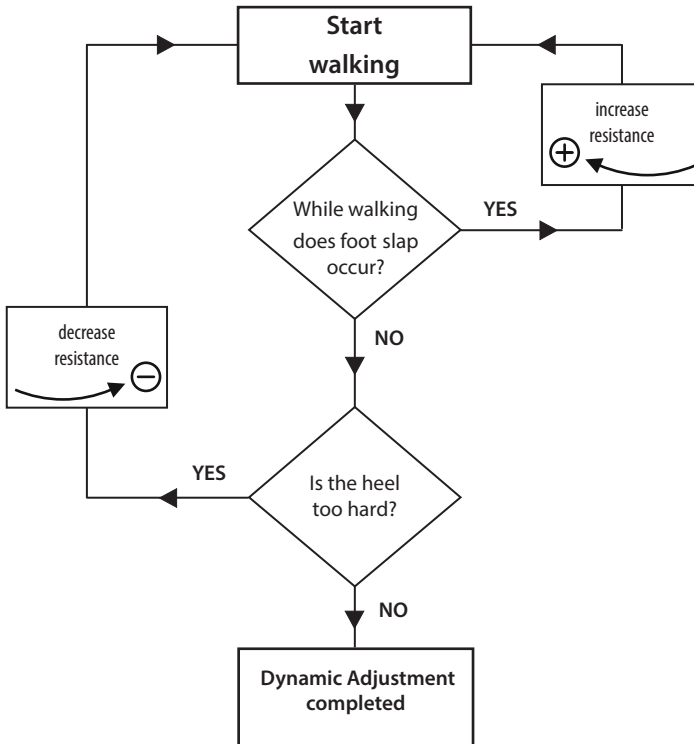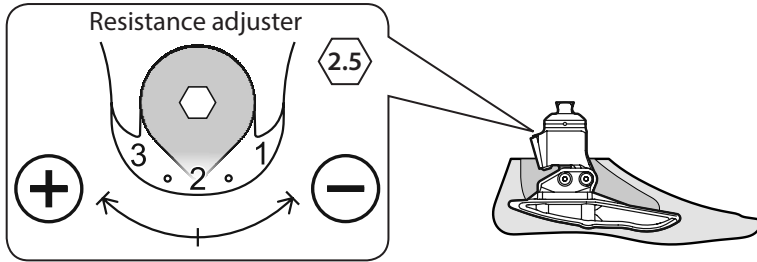


Guidance:

Following dynamic adjustment, trial the foot/ankle on ramps and stairs. Ensure the user is comfortable with the kind of terrain he/she may normally be expected to encounter. If the user reports any issues with comfort, usability or range of movement of the ankle, adjust accordingly. Caution: decreasing plantar flexion resistance may cause knee instability for trans-femoral users

10 Technical Data


Principal Materials	Aluminum, Stainless Steel, Titanium, PU, Thermoplastic Composite, UHM-PE
Component Weight (size 26)	780 g (1 lb 11 oz)
Maximum User Weight	150 kg (330 lb)
Activity Level	2
Size Range	22 to 30cm
Build Height (See diagram below)	115 mm
Heel Height	10 mm
Range of ankle movement	6 degrees plantar flexion to 6 degrees dorsiflexion
Operating and Storage Temperature Range	-15°C to 50°C (5°F to 122°F)
Proximal connection	Male Pyramid (Blatchford)

Fitting length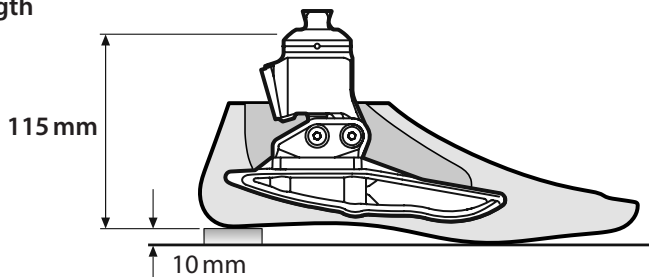


11 Assembly Instructions

1 Place foot in glide sock and slide assembly into foot shell.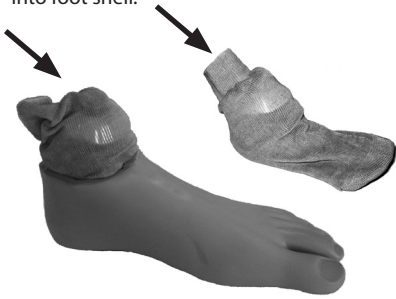


2 Ensure the heel of the keel is fully engaged in the location feature in the foot shell.

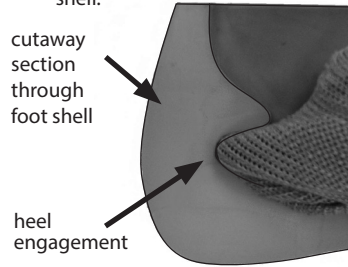3 Abraid the top surface of the foot shell. Bond 12 mm Pelite interface to foot shell and shape as appropriate. Use Evostik 528 (926213) or equivalent.


4 Bond foam cosmesis to top surface of the Pelite using Thixofix adhesive (926204), Evostik 528 (926213) or equivalent and shape to suit.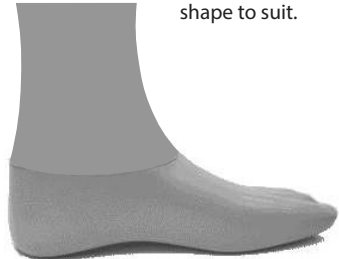

12 Ordering Information

Order example

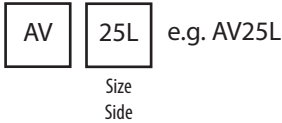Available from size 22 to size 30:
AV22L to AV30R
AV22LD to AV30RD
(add 'D' for a dark tone foot shell)

Foot Shell (for dark add 'D')

Small	Medium	Large	Extra Large
22L 539020	24L 539024	26L 539028	28L 539032
22R 539021	24R 539025	26R 539029	28R 539033
23L 539022	25L 539026	27L 539030	29L 539034
23R 539023	25R 539027	27R 539031	29R 539035
			30L 539036
			30R 539037

Glide Sock, Sizes 22–30 Part No. 405815
Allen 2.5 (Hex key, valve adjuster tool) Part No. 940234

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

CE conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty:

This device is warranted for 24 months—glide sock 3 months—foot shell 12 months. The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions. See Blatchford website for the current full warranty statement.

Manufacturer's Registered Address

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH, UK.

Avalon and Blatchford are registered trademarks of Blatchford Products Limited.

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Blatchford SAS
Endolite India Ltd