Avalon^{K2-VAC}

User Guide

AVAC24L-AVAC30R

EN	User Guide	1
FR	Guide de l'Utilisateur	4
DE	Bedienungsanleitung für Anwender	7
IT	Guida per il Paziente	10
ES	Guia de Usuario	13
NO	Brukerhåndbok	16
RU	Руководство пользователя	19
TR	Kullanım Kilavuzu	22
AR	دليل المستخدم	25

Blatchford:

AVAC24L-AVAC30R

Please keep these instructions.

The term *device* is used throughout this user guide to refer to AvalonK2VAC.

This device is recommended for users that have the potential to achieve Activity Level 2 who may benefit from enhanced stability and an increase in confidence on uneven surfaces and slopes (maximum user weight 150 kg/330 lb). In addition, it generates a vacuum in the range 406-575 mbar/12-17 inHg. It is intended to improve on postural sway and symmetry while easing abnormal pressures at the socket interface.

The device dorsiflexes after mid stance and remains dorsiflexed at toe off through the swing phase to give greater toe clearance for improved confidence and security. The controlled plantar flexion provided by the device at heel contact improves time to foot flat which enhances stability on uneven surfaces and slopes.

It is recommended that only Blatchford products be used in conjunction with the device.

After continuous use the ankle casing may become hot to the touch.

To minimize the risk of slipping and tripping, appropriate footwear that fits securely onto the foot shell must be used at all times.

Be aware of finger trap hazard at all times.

Any changes in performance of this device must be reported to the practitioner.

Changes in performance may include:

- Increase in ankle stiffness
- Reduced ankle support (free movement)
- Any unusual noise
- Lack of vacuum

Note... Check that the suspension sleeve maintains a good seal with the skin proximally and socket distally.

The practitioner must also be informed of the following:

- Significant wear or excessive discoloration of foot shell from long term exposure to UV light.
- Any changes in body weight and/or activity level.
- Discoloration of the residual limb.

Intended for single user.

Cleaning

Use a damp cloth and mild soap to clean outside surfaces, do not use aggressive cleansers. Maintenance must be carried out by competent personnel. It is recommended that maintenance is carried out annually.

Environment

Avoid exposing the device to corrosive elements such as water, acids and other liquids. Foot products must be adequately finished to prevent water ingress into the foot shell where possible. If water enters the foot shell, the limb should be inverted and dried before further use.

Suitable for outdoor use

Also avoid abrasive environments such as those containing sand for example, as these may promote premature wear.

Exclusively for use between -15 °C and 50 °C (5 °F to 122 °F).

Warranty

The device is warranted for 24 months – foot shell 12 months - glide sock 3 months.


This warranty does not apply to:

Consumable parts including the vacuum tubing, in-line filter and valves unless a failure has occurred due to a defect in materials or workmanship.


The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses and exemptions.

See Blatchford website for the current full warranty statement.

Vacuum System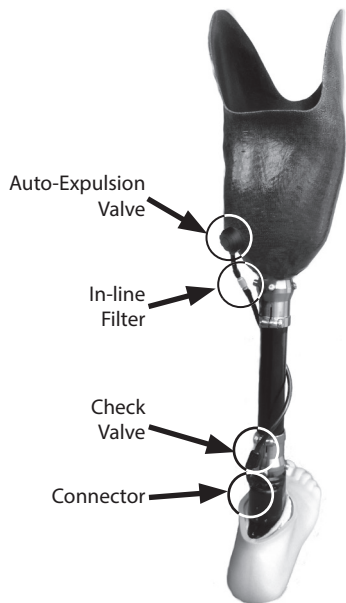

Check Valve
The check valve retains the vacuum created in the socket. It must be connected with the direction arrow pointing towards the ankle.

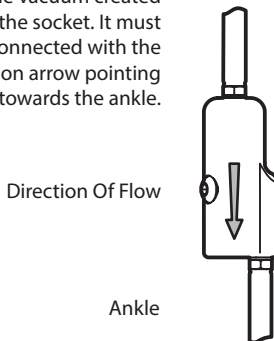Vacuum System Guidance

	Symptoms:	Cause/Remedy
1.	Unable to generate a vacuum	Vacuum tube(s) disconnected Inspect and reconnect as necessary
		Limited ankle movement due to footwear A full range of movement is required
2.	Unable to maintain a vacuum	Vacuum tube(s) disconnected Inspect and reconnect as necessary
		Leakage at socket valve/barbs Consult your practitioner
		Check the integrity of the vacuum seal at the socket/residuum interface

Environmental Aspects

Where possible the components should be recycled in accordance with local waste handling regulations.

Manufacturer's Registered Address

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH, UK.

Avalon and Blatchford are registered trademarks of Blatchford Products Limited.

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© Blatchford Products Limited 2019. All rights reserved.
Endolite India Ltd