Female Mini Pyramid Tube Adapter

Instructions for Use

189526 [25 mm] & 189727 [30 mm]

EN	Instructions for Use	2
FR	Instructions D'Utilisation	6
DE	Gebrauchsanweisung	10
IT	Istruzioni per L'Uso	14
ES	Instrucciones de Uso	18
NO	Bruksanvisning	22
RU	Инструкция протезиста	26
TR	Kullanım Talimatları	30

1 Description and Purpose

EN

These instructions are for use by the practitioner.

Application:

The Female Mini Pyramid Tube Adapter is for use exclusively as part of a lower limb prosthesis. The device connects a Male Mini Pyramid to a 25/30mm diameter tube and allows for angular and rotational adjustment.


Intended for single user only.

There are no known contra-indications if used in accordance with these instructions.

Ensure that the user has understood any instructions for use, drawing particular attention to the maintenance information.

2 Construction

Principal Parts	Part No. 189526	Part No. 189727
Adapter	Titanium - \varnothing 25mm	Titanium - \varnothing 30mm
Mini Pyramid Screws	Zinc Plated Steel	Zinc Plated Steel
Clamp Screw	Zinc Plated Steel	Zinc Plated Steel
Washer	Zinc Plated Steel - Lock	Stainless Steel - Cup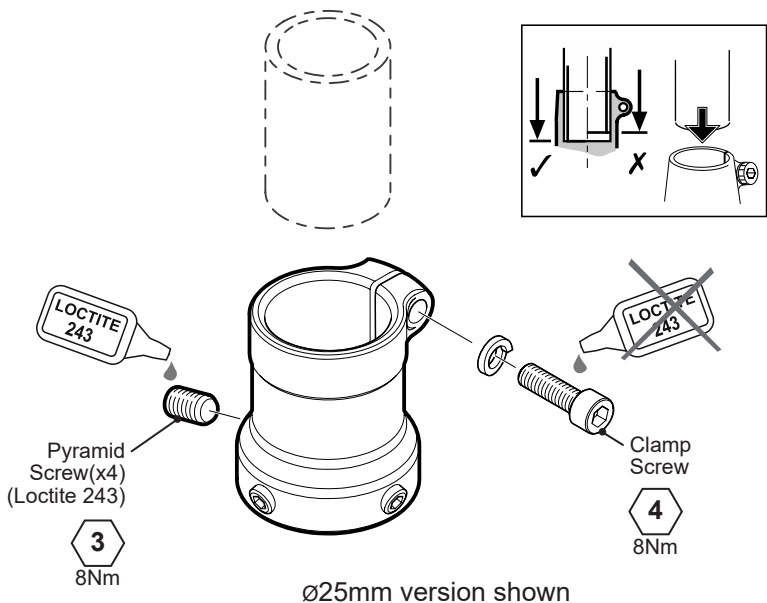

3 Function

The angular movement and rotation prior to tightening the screws allows for alignment of the limb to suit the individual user.

When tightened securely using Loctite on the pyramid screws only and torque settings shown, the device offers a secure way to connect two parts of the limb.

4 Maintenance

Maintenance is to be carried out by competent personnel only.

An annual visual inspection is recommended

- Ensure all screws are secure. If not remove and clean screws, reapply Loctite to the pyramid screws only, tighten all screws to the correct torque settings.
- Check for visual defects that may affect proper function.

The user should be advised:

To contact their practitioner if their condition changes.

Any changes in performance of the device must be reported to the practitioner.

Changes in performance may include:

- Any unusual noises
- Excessive play or loss of alignment

5 Limitations on use

It is recommended that only Blatchford products be used in conjunction with the Female Mini Pyramid Tube adapter, see Technical Specification. Blatchford Mini Pyramid components must only be used in conjunction with other Blatchford Mini Pyramid components.

Intended life:

A local risk assessment should be carried out based upon activity and usage.

Lifting loads:

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

Environment:

The product is water proof to a maximum depth of 1 metre.

Thoroughly rinse with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage to moving parts.


Thoroughly rinse with fresh water after use in salt or chlorinated water.

Exclusively for use between -15°C and 50°C (5°F to 122°F).

6 Technical Specification:

Part No.	189526	189727
Material:	Titanium, Steel	Titanium, Steel
Operating and Storage Temperature Range:	-15°C to 50°C 5°F to 122°F	-15°C to 50°C 5°F to 122°F
Product Weight:	75g (3ozs)	75g (3ozs)
Recommended Activity Level:	up to 4	up to 4
Maximum User Weight:	60kg (132lbs)	60kg (132lbs)
Range of adjustment:	360 degrees rotation 5 degrees angular	360 degrees rotation 5 degrees angular
Build Height:	42mm	48mm
Connection to:	Male Mini Pyramid Ø25mm Tube	Male Mini Pyramid Ø30mm Tube

Key Dimensions:

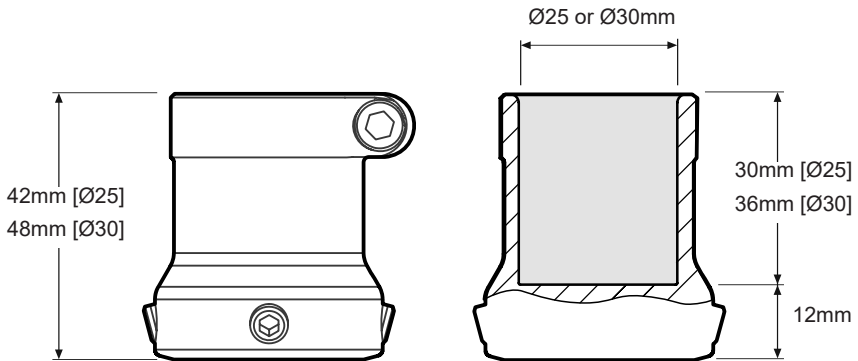Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty

The Female Mini Pyramid Tube adapter is warranted for 36 months.

See Blatchford website for the current full warranty statement.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licences and exemptions.

Manufacturer's Registered Address

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Blatchford SAS
Endolite India Ltd