Mini Blade^{XT}

Instructions for Use

MBXTAB1SM-MBXTAP3SM

EN	Instructions for Use	2
FR	Instructions D'Utilisation	9
DE	Gebrauchsanweisung	16
IT	Istruzioni per L'Uso	24
ES	Instrucciones de Uso	32
NO	Bruksanvisning	40
RU	Инструкция протезиста	47
TR	Kullanım Talimatları	54

Blatchford:

1 Description and Purpose

Application

These instructions are for the practitioner.

The term *device* is used throughout these instructions for use to refer to Mini BladeXT.

This device is to be used exclusively as part of a lower limb prosthesis.

This device is suitable for walking and is designed specifically for general sport and running.

This device is recommended for users that have the potential to achieve Activity Level 4.

Blatchford mini pyramid components must only be used in conjunction with other Blatchford mini pyramid components.

Contra-indications

This device may not be suitable for Activity Level 1 and 2 individuals.

Intended for a single user.

Ensure that the user has understood all instructions for use, drawing particular attention to the Safety Information and section regarding maintenance.

Spring set selection

Activity	User Weight			kg (lbs)
	20-35 (44-77)	36-50 (78-110)	51-60 (111-132)	
4	1	2	3	Toe and Heel Spring set
	Soft	Medium	Firm	

Important: For higher impact users, do not exceed the weight limit for individual springs.

Safety Information

Steps and stairs

The user should be advised that because of the deep 'C' shape side profile (A-P profile) of this device, care should be taken when descending stairs and steps due to the potential hazard of 'over-stepping'.


Be aware of finger trap hazard at all times


2 Construction

Principal parts:

- Heel & Toe Springs (e-carbon)
- Heel attachment Screws & Washers (St. Stl. - silver)
- Sole Cover & Wedge (PU)
- Mini Pyramid (titanium)
- Pyramid Washer and Nut (St. Stl.)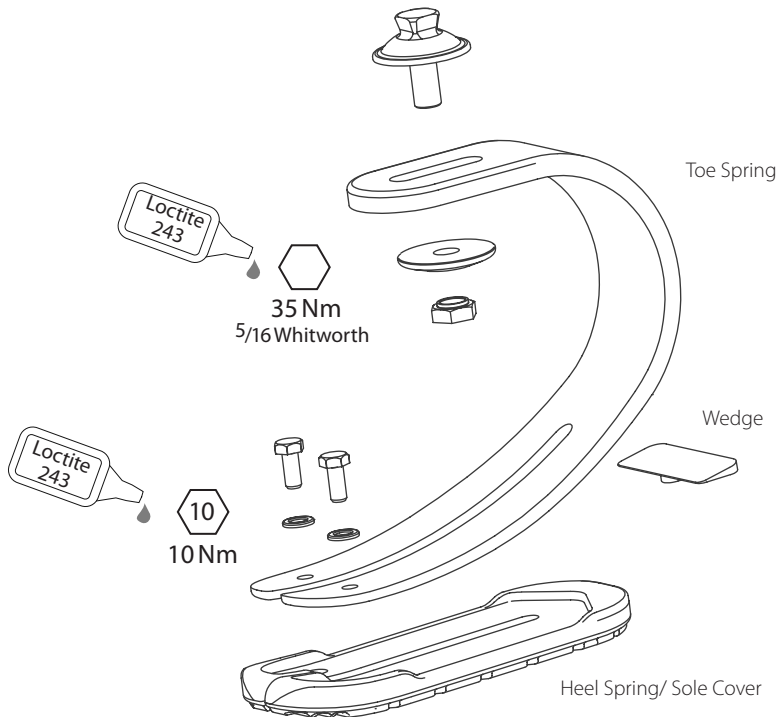

3 Alignment

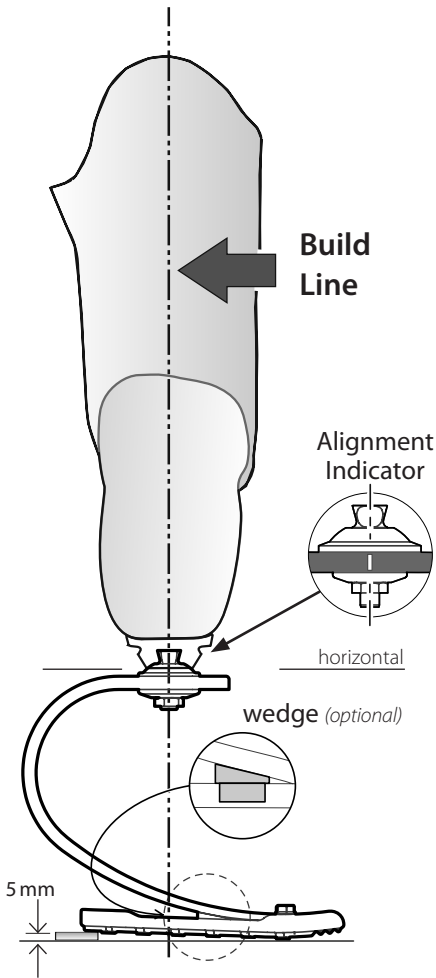Ensure the top surface of the device is horizontal to allow for compression in the main toe spring when loaded with the user's body weight.

Static alignment

Trans-femoral alignment

Align trans-femoral components according to fitting instructions supplied with the knee, keeping the build line relative to the device, as shown.

Setup length

This device should be set up with a 5 mm heel wedge so that the top surface is horizontal. The length of this device should be 5–10 mm longer than the user's everyday prosthesis with running trainer footwear to allow for spring compression.

Build line

The build line should fall in the middle of the slot on top of this device, this is defined by a vertical alignment indicator line on each side.

Dynamic alignment

Coronal plane

Ensure that M-L thrust is minimal by adjusting relative positions of socket and foot.

Sagittal plane

On standing the toe should deflect so that the heel is in contact with the ground. On walking, check for a smooth transition from heel strike to toe off.

1 Optimise plantar flexion and dorsiflexion

On running, a heel strike should always be present but this can be very small and subtle. To optimise running gait, plantarflex the foot incrementally in small amounts at a time ensuring a heel strike is maintained. If the foot is too plantar flexed the sole will 'slap' against the ground. To prevent 'slap' dorsiflex the foot.

2 Stiffness and A-P shift

When an optimal plantar flexion/dorsiflexion angle has been achieved, if when running the toe feels too soft, then shift the weight line posteriorly on the foot. If the toe feels too stiff, then shift the weight line anteriorly on the foot.

3 If the user is sinking at heel strike the heel is too soft. Fitting the wedge supplied will have the effect of stiffening the heel spring, this can be taped in place for trial (see diagram). For permanent fitting the wedge should be glued in place by application of Loctite 424 (926104) on the upper surface of the wedge.

4 Fitting Advice

Springs are supplied as matched sets i.e. the shin/toe and heel springs are designed to work together to give smooth progression for most users.

Condition	Symptoms	Remedy
Heel too soft	Sinking at heel strike	Add Heel Wedge
Heel too hard	Rapid transition from heel strike through stance phase Difficulty in controlling heel action, foot jars into mid-stance	1. Remove Heel Wedge (if fitted) 2. Order a softer Heel Spring category
Toe too soft	Rapid progression through mid-stance 'Drop-off' during running	1. Move socket posteriorly in relation to foot 2. Plantar flex foot slightly - note: heel strike must be maintained
Toe too hard	Feels like 'climbing' over toe	1. Move socket anteriorly in relation to foot 2. Dorsiflex foot slightly

Please contact your supplier if it is not possible to achieve a smooth gait after following the recommendations above.

5 Maintenance

This device should **not** be used if the sole cover is excessively worn, torn or loose. The user should be advised to visually inspect this device, especially the condition of the heel and toe springs, before any activity.

We recommend the following annual maintenance:

- Check the Sole Cover for damage or wear and replace the Heel Spring assembly if necessary.
- Check all bolts for tightness, clean and reassemble as necessary.
- Visually check both the Heel and Toe Blade Springs for signs of delamination or wear and replace if necessary. Some minor surface damage may occur after a period of use, this does not affect the function or strength of the foot.

The user must be handed the User Guide and be advised of the following:

- Beware of slip hazard at all times especially on wet/smooth surfaces.
- Any changes in performance of this device must be reported to the practitioner e.g. reduced energy return, unusual noises or softening of the toe.
- The practitioner must also be informed of any changes in body weight and/or activity level.
- Excessive wear of the sole cover may lead to loss of grip
- Excessive damage or wear to the spring elements could lead to structural failure

Cleaning

Use a damp cloth and mild soap to clean the outside surfaces.

DO NOT use aggressive cleansers. Thoroughly dry before use.

6 Limitations on Use

Intended life

A local risk assessment should be carried out based upon activity and usage.

Lifting loads

User weight and activity is governed by the stated limits.

Load carrying by the user should be based on a local risk assessment.

Environment

This device is waterproof to a maximum depth of 1 meter.

Thoroughly rinse this device with fresh water after use in abrasive environments such as those that may contain sand or grit, for example, to prevent wear or damage.

Thoroughly rinse with fresh water after use in salt or chlorinated water.

Exclusively for use between -15 °C and 50 °C (5 °F to 122 °F).

We recommend using Blatchford products with this device.

Suitable for submersion


7 Heel Spring Removal & Replacement

8 Technical Data

Material:	E-carbon, PU , titanium, St. Stl.
Operating and Storage Temperature Range:	-15°C to 50°C (5°F to 122°F)
Component Weight:	450 g (1 lb)
Activity Level:	4
Maximum User Weight:	60 kg (132 lb)
Proximal Alignment Attachment:	Male Mini Pyramid (Blatchford)
Range of Adjustment:	360° rotation ±5° angular 30 mm A-P shift
Build Height:	168 mm
Proximal pyramid to ground level (See diagram below)	

Fitting length

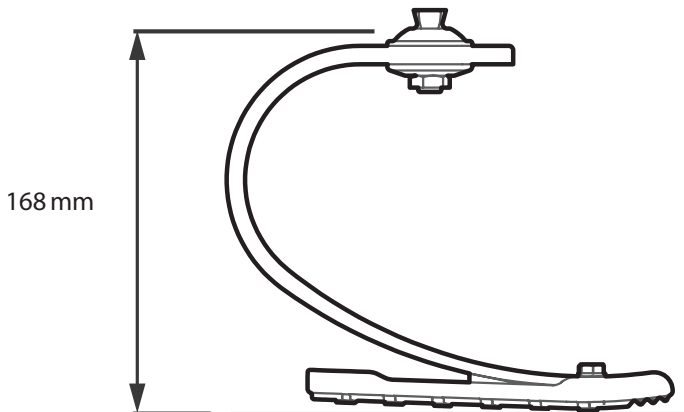9 Ordering Information

Order example

Spring Set: 1, 2, 3

Color: Blue B

Pink P

e.g. MBXTAB3SM

MBXTA

B

3

SM

Blue

Spring
Set 3

MBXT heel spring kit

Includes the overmoulded cover, fasteners and heel wedge.

Spring category	Heel type	Part no.
1	Soft	539084SS
2	Medium	539084MS
3	Firm	539084FS

Proximal adapter

Male mini pyramid kit Part No. 189627

Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

CE conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty

This device is warranted for 12 months (excludes paintwork and sole cover). The user should be aware that changes or modifications not expressly approved could void the warranty, operating licences and exemptions. See Blatchford website for the current full warranty statement.

Manufacturer's Registered Address

Blatchford Products Limited, Lister Road, Basingstoke RG22 4AH, UK.

BladeXT and Blatchford are registered trademarks of Blatchford Products Limited.

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Blatchford SAS
Endolite India Ltd