Casting Dummy

Instructions for Use

559019

EN	Instructions for Use	2
FR	Instructions D'Utilisation	4
DE	Gebrauchsanweisung	6
IT	Istruzioni per L'Uso	8
ES	Instrucciones de Uso	10
NO	Bruksanvisning	12
RU	Инструкция протезиста	14
TR	Kullanım Talimatları	16

1 Description and Purpose

EN

Application

This device is only for casting a socket that is for use in combination with Silcare Breathe Locking Liner.


2 Casting the Socket with the Casting Dummy

 Sockets with sharp proximal edges can tear the device.

When casting the socket, use the casting dummy to make sure that the distal end of the socket has space for the valve to open. Otherwise, moisture will collect inside the device.

Note: Do not use the casting dummy for any purpose other than to cast the socket.


1. Trim the edge of the casting dummy so that it covers the distal cap and does not overlap it.
2. Put the casting dummy over the distal cap.
3. Cast the socket.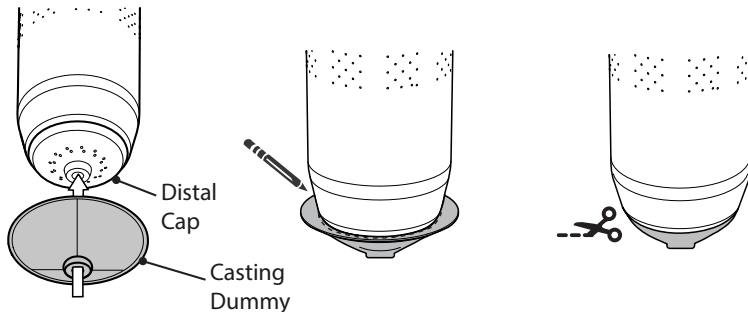

After casting the socket with the casting dummy, the valve should have enough space to open.

Stance phase: valve open

Swing phase: valve closed

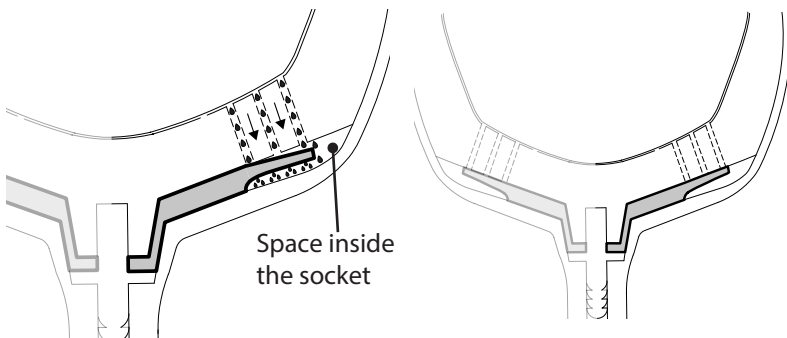Liability

The manufacturer recommends using the device only under the specified conditions and for the intended purposes. The device must be maintained according to the instructions for use supplied with the device. The manufacturer is not liable for damage caused by the component combinations that were not authorized by the manufacturer.

CE Conformity

This product meets the requirements of 93/42/EEC guidelines for medical products. This product has been classified as a Class 1 Product according to the classification criteria outlined in Appendix IX of the guidelines. The Declaration of Conformity was therefore created by Blatchford Products Limited with sole responsibility according to Appendix VII of the guidelines.

Warranty

This device is warranted for 6 months.

The user should be aware that changes or modifications not expressly approved could void the warranty, operating licenses, and exemptions.

See the Blatchford website for the current full warranty statement.

Manufacturer's Registered Address

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© Blatchford Products Limited 2019. All rights reserved.
Endolite India Ltd